

LIFETIME LIMITED WARRANTY

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. We are dedicated to providing products that will give our customers years of dependable service.

This Lumex product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failures due to normal wear and tear or device misuse. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized Graham-Field Health Products Dealer.



All illustrations and specifications in this manual are based on the latest product information available at the time of publication. Graham-Field reserves the right to make changes at anytime, without notice.
© 2005, Graham-Field Health Products, 2935 Northeast Parkway, Atlanta, Georgia 30360.
Printed in China. Doc Number: RJ4200A 03/04

LUMEX®

For The Quality Of Life®



INSTRUCTIONS & WARRANTY INFORMATION

Deluxe Four Wheeled Rollator
RJ4200A

IMPORTANT: READ BEFORE USE

INSTRUCTIONS FOR USE – PLEASE READ ALL INSTRUCTIONS BEFORE USE.

Please consult a healthcare professional for the proper adjustment.

HEIGHT ADJUSTMENT

1. Remove Rollator from case.
2. Cut plastic ties to remove basket.
3. Unfold frame; spread front and back legs apart.
4. Flip down black locking mechanism to secure back legs.
Ensure that black locking mechanism is resting firmly on back crossbar.
5. Remove the gray plastic end caps from the frame and discard them.
6. Insert chrome tube sections of left and right handle bars into their respective vertical frame sections.
7. Adjust handle bars to user's height. DO NOT extend the handle bars beyond the security mark. Tighten adjustment knobs on both vertical frame sections until handle bars are secure.

OPERATION

HAND BRAKE: Squeeze hand brakes to stop; release hand brakes to walk. Ensure that both hand brakes are squeezed at the same time to ensure proper braking. If only one hand brake is squeezed, only one wheel will brake.

TO LOCK: Push hand brakes down to lock. Ensure that both left and right hand brakes are locked. Always lock hand brakes when user is immobile; never sit on rest seat if hand brakes are not locked.

TO UNLOCK: Lift hand brakes up to unlock.

FOLDING ROLLATOR: Remove any attached accessories, such as the wire shopping basket and food tray. Flip up black locking mechanism. Press legs together.



ATTACH ACCESSORIES

SHOPPING BASKET: Attach wire shopping basket hooks to front side of Rollator top crossbar on both sides of rest seat

FOOD TRAY: Place food tray on rest seat; ensure that food tray is firmly seated.

MAINTENANCE

- For dust and dirt, wipe Rollator gently with damp, softy cloth.
- Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners or scouring pads on any part of Rollator or food tray
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that Rollator is safe to use.
- To guarantee the best service from your Rollator, always specify Lumex replacement parts, available from your Lumex dealer.

SAFETY GUIDELINES – PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 500 lb.
3. Graham-Field Health Products assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.
5. Before use, ensure that all assembled parts are secure and all hardware is tightened.
6. Always use caution when using this product on wet or slippery surfaces. Do not use Rollator on stairs or steep inclines.
7. Inspect Rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Graham-Field Health Products dealer for repair and/or appropriate replacement parts.
8. Assemble Rollator as instructed to ensure safe use.
9. If the hand brakes fail to stop the Rollator, contact your Graham-Field Health Products dealer immediately for adjustment.