

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

- ⚠ WARNING:** **Important: Read and understand these instructions before assembling or using the PVC hamper. Ensure that the PVC hamper is installed as described in the following paragraphs before use. If the PVC hamper is not properly installed, personal injury and/or property damage could result.**
- ⚠ WARNING:** **GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**
- ⚠ WARNING:** **If components are damaged or missing, contact your Graham-Field Health Products dealer immediately. DO NOT use substitute parts.**

## HAMPER ASSEMBLY - REFER TO PICTURES AT RIGHT

*Tools needed: Phillips screwdriver or cordless drill*



1. Remove partially assembled hamper from shipping container / box. Box should contain the following items:
  - a. Partially assembled hamper
  - b. Four caster wheels (two locking, two non-locking) in brown box
  - c. Lid handle (one for single unit; two for double unit)
  - d. Appropriate mesh/vinyl retainer bag/s
  - e. Appropriate wire bag holder/s for optional liner/s

*Note: The appropriate lid is attached to each hamper frame. To fit properly, each lid must be used on its own frame. DO NOT MIX lids on double Hampers.*

**Double units must be assembled before continuing:** Connect the fittings to the PVC extensions. Install 1/2" screws to fasten them. Use Phillips screwdriver to tighten.

2. Ensure that double units are assembled as described above. Install the four casters in the openings of the four-way fittings: Each caster fitting has a pre-drilled pilot hole on the inside. Secure the casters to the fittings with 1/2" screws. Use a Phillips screwdriver or cordless drill to tighten. **Ensure that casters are securely fastened to fittings, or casters will fall out.**
3. Secure the lid/s to the paddle/s. Use the nylon screws and bolts to fasten. Hand-tighten. Ensure that lid and paddle are securely fastened, or the hamper lid will not pivot properly.
4. Attach handle/s to lid/s: Install handle on top of lid. Insert screw through lid and into handle. Use a Phillips screwdriver or cordless drill to tighten.

- ⚠ WARNING:** **Before use, ensure that hamper is correctly assembled as described and that all components and hardware are securely fastened.**

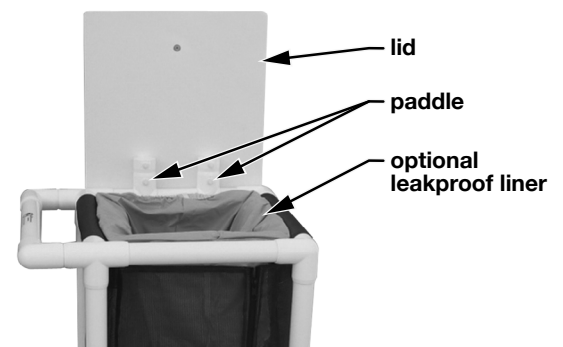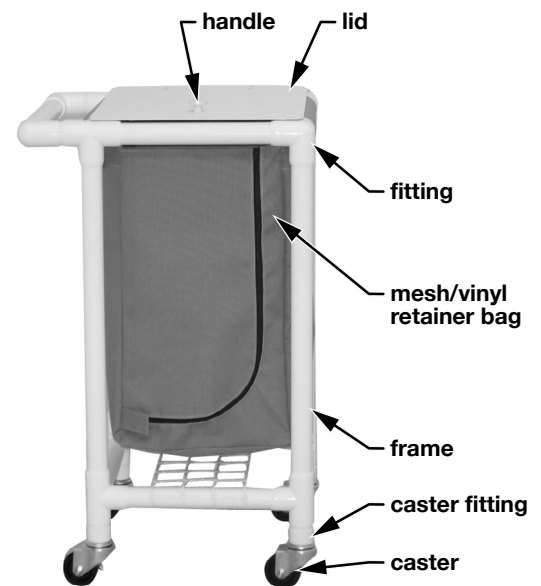

## WARRANTY

This warranty is extended only to the original purchaser and is not transferable. GF Health Products, Inc. warrants the PVC hamper and its components to be free from defects in workmanship and material integrity, with the exception of casters, as listed below:

PVC frame: one year

Mesh bag: six months

Solid vinyl components: six months

This warranty excludes casters and all other components not listed above. The warranty period for the consumer commences on the invoice date from GF Health Products, Inc. This limited warranty does not cover defects of malfunction caused by misuse, improper installation, alteration, exposure to hazardous or corrosive chemicals, shipping damage, or other accident or casualty.

