Note: The most current version of these instructions can be found online at www.grahamfield.com


- ⚠ WARNING: Important! Read and understand these instructions before installing or using the hand-held shower head. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the hand-held shower head is not properly assembled and installed, personal injury and damage to the hand-held shower head could result.**
- ⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the hand-held shower head.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION

1. Remove all contents from packaging.
2. Attach the mounting bracket to the end of the shower arm on the wall. Ensure the mounting bracket threaded rod is facing down when tight as shown in picture at right.
3. Place one rubber washer in the end of the hose as shown at right.
4. Attach the end of the hose in which you just placed a rubber washer to the bottom of the mounting bracket. Turn hose nut clockwise until tight. **Caution - Do not overtighten.**
5. Insert the other rubber washer into the other end of the hose as shown.
6. Attach the remaining end of the hose to the end of the shower head by turning the threaded collar clockwise. **Caution - Do not overtighten.**

Note: Plumber's tape (not included) may be needed to seal threaded areas.

- ⚠ WARNING: Ensure that the hand-held shower head is installed as described. If the hand-held shower head is not properly assembled and installed, personal injury and damage to the hand-held shower head could result.**

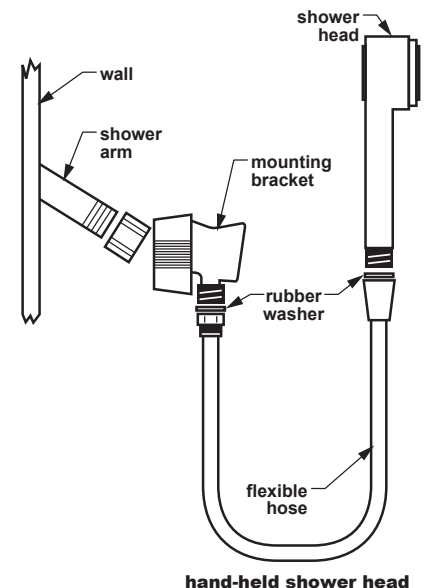MAINTENANCE

To clean the hand-held shower head, wash it with soap and water or a mild detergent, then rinse it with water.

- ▲ Caution: Never use abrasive cleaners or scouring pads on any part of the hand-held shower head.**

WARRANTY

GF Health Products, Inc. warrants the Lumex 12038/12039 Hand-Held Shower Head for three years against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© March 2009 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com

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