

Note: The most current version of these instructions can be found online at www.grahamfield.com

Tools needed: Phillips screwdriver, 9/16 hex wrench, 5/16 hex wrench

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the GF8905-1A Overbed Table. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the GF8905-1A Overbed Table is not properly assembled and adjusted, personal injury and/or damage to the Overbed Table could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and/or damage to overbed table.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**



UNPACKING

1. Check for obvious damage to the carton or its contents. If damage is evident, notify the carrier and your Graham-Field® authorized distributor.
2. Remove all loose packing from the box. Carefully remove all the components from the carton. Components: Table top, mast, 6 machine screws, base with 4 casters, 2 hex head bolts, 2 flat washers, and 2 metal spacers.

ASSEMBLY (SEE PICTURES AT RIGHT)

- ⚠ **WARNING: Do not remove the transit lock screw from the mast before the table is completely assembled; the table top could rise suddenly and cause serious personal injury.**
- ⚠ **WARNING: Do not disassemble the mast; this could cause serious personal injury.**

1. Place the table top upside down on work surface.
2. Position the mast above the table top, oriented as shown at right. Place mast mounting plate on table top, matching mast mounting plate holes with table top bracket holes.
3. Mount mast to table top: Install machine screws through mast mounting plate holes and table top bracket holes. Install nuts on screws. Tighten.
4. Place the base upside down above the mast, oriented as shown at far right.
6. Install the two hex head bolts through the washers, spacers, and base, and into the end of the mast column as shown. Tighten the bolts.

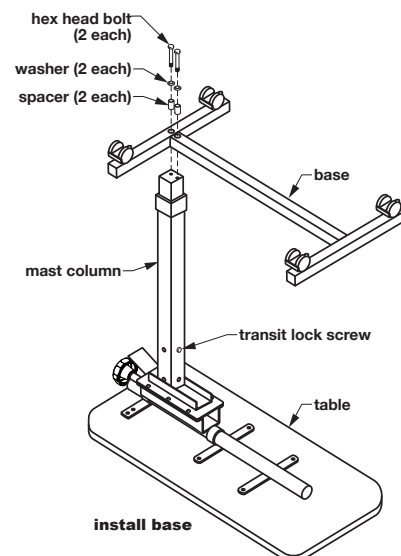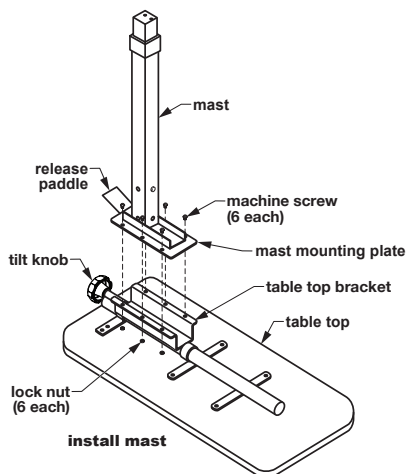

7. Turn the table upright and place it on the floor, as shown at right. While holding table top down with one hand, **carefully** — **table could rise suddenly** — remove the transit lock screw with the other hand. Discard transit lock screw.

⚠ **WARNING: Ensure that the table is assembled as described, and that all components are securely attached, before use.**

OPERATION INSTRUCTIONS

⚠ **WARNING: Do not lean on the suspended end of the table top; this may cause the table to tip.**

⚠ **WARNING: The table supports up to 30 lb of dispersed weight. Do not exceed this limit.**

To raise the table top

To raise the table top, apply light pressure upward on the bottom of the table top and stop when the table top has reached the desired position.

To lower the table top

To lower the table top, squeeze the release paddle upward while pushing the table top down; release the paddle when the table top has reached the desired position.

To adjust tilt

To tilt the table top in either direction, pull out the tilt knob, tilt the table top, and release the knob when the the table top is in the desired position.

⚠ **WARNING: Ensure the table top is locked in place before use.**

Lock and unlock casters

The two mast-end casters have step-on locks. Step on locks to lock casters; pull locks up to unlock.

MAINTENANCE

Check the fit and tightness frequently of all nuts and bolts to ensure that the table is securely assembled.

Cleaning


1. To clean: Gently wipe the table with a soft, clean, damp cloth.

▲ **Caution: Do not use cleansers that may damage the table's finish.**

2. To wash more thoroughly: Clean with mild soap and water.
3. Dry table with a clean, soft cloth.

WARRANTY

GF Health Products, Inc. offers a limited three-year warranty against manufacturer's defects on the Lumex® GF8905-1A Deluxe Tilt Overbed Table. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

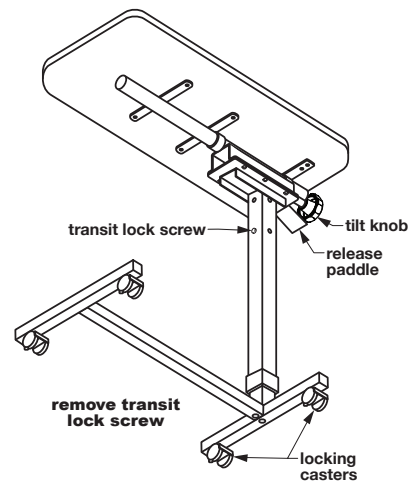

GRAHAM-FIELD

www.grahamfield.com