

***Please call Tech Support
for Model 577G Parts***



577RG

TO ORDER: Tel 800-347-5678 • Fax 800-726-0601 • www.grahamfield.com

Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.


GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Lumex® and Graham-Field® are registered trademarks of GF Health Products, Inc.



© June 2009, GF Health Products, Inc., All Rights Reserved


GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0602 • www.grahamfield.com

577RG Recliner: Replacement Parts List

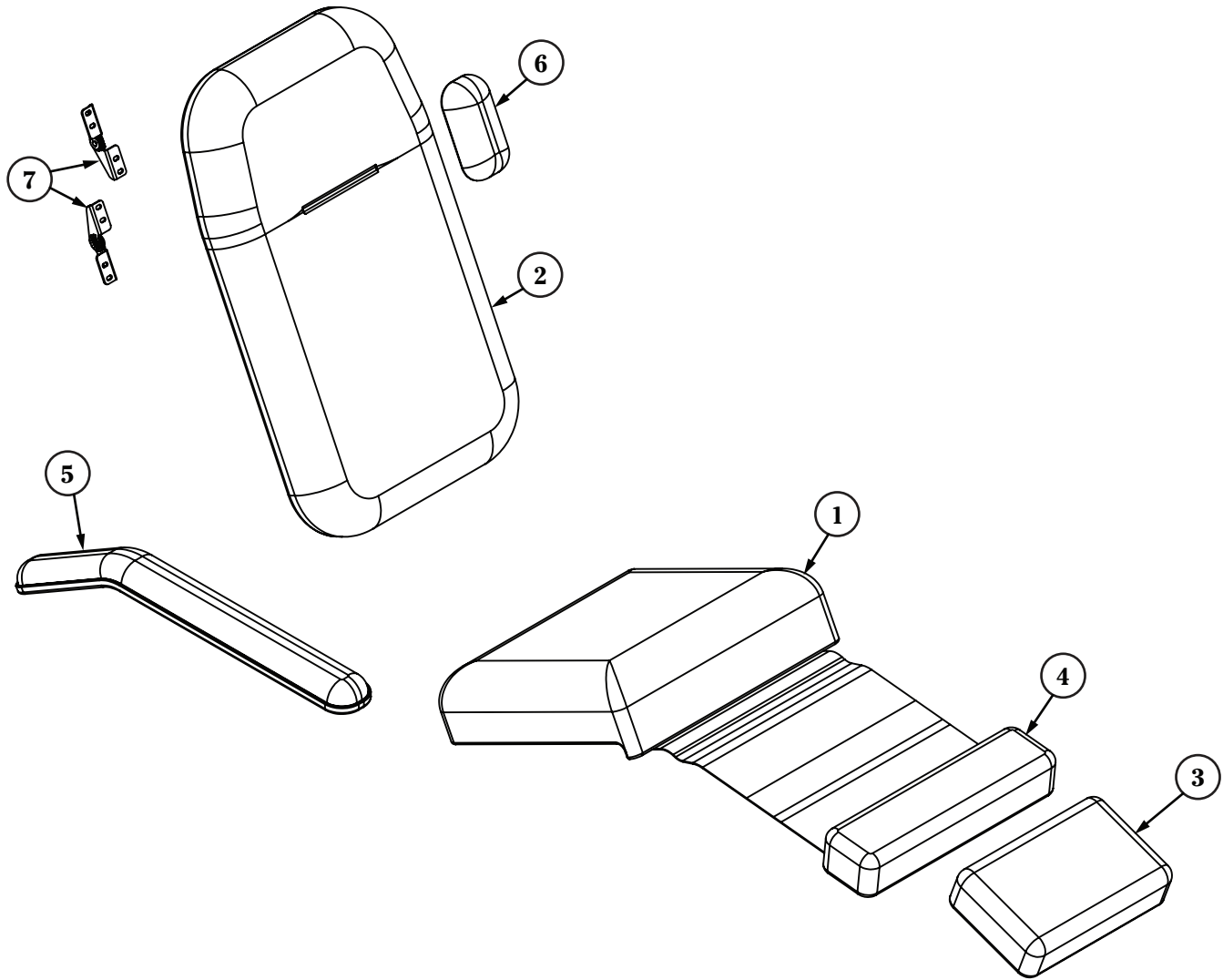

	ITEM NUMBER	DESCRIPTION	QTY/UNIT
1A	57701UXXX	Seat Upholstery	1 each
1B	57701UXXXFR	Seat Upholstery (CA-133 Fire Rated)	1 each
2A	57702UXXX	Back Upholstery	1 each
2B	57702UXXXFR	Back Upholstery (CA-133 Fire Rated)	1 each
3A	57403UXXX	Footrest Upholstery	1 each
3B	57403UXXXFR	Footrest Upholstery (CA-133 Fire Rated)	1 each
4A	57404UXXX	Legrest Upholstery	1 each
4B	57404UXXXFR	Legrest Upholstery (CA-133 Fire Rated)	1 each
5A	58735UXXX	Armrest Upholstery	1 each
5B	58735UXXXFR	Armrest Upholstery (CA-133 Fire Rated)	1 each
6A	57706UXXX	Upper Arm Support	1 pair
6B	57706UXXXFR	Upper Arm Support (CA-133 Fire Rated)	1 each
7	5740A020	Back Hinge with Hardware	1 pair

577RG Recliner: Replacement Parts List

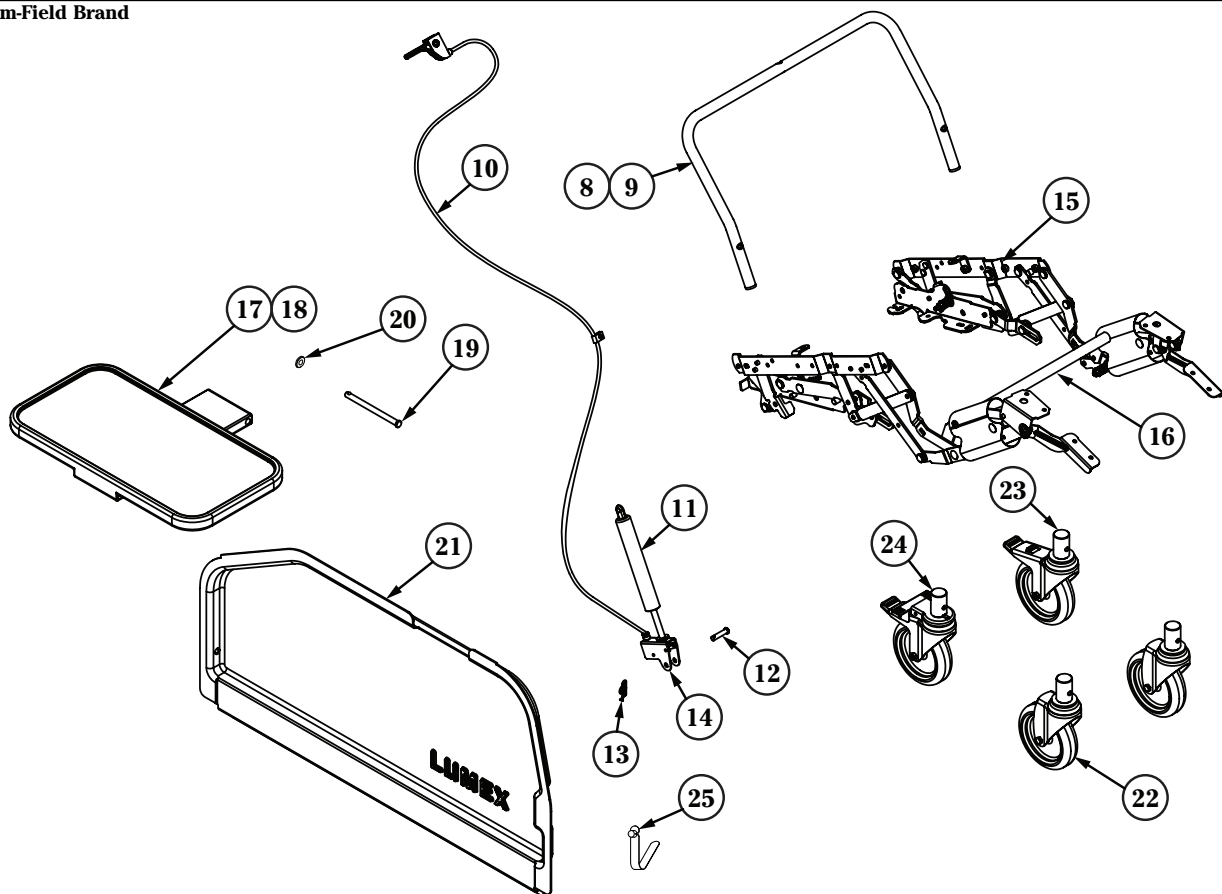

	ITEM NUMBER	DESCRIPTION	QTY/UNIT
8	5772A216G	Push Handle with Grip	1 each
9	9205	Grip, Push Handle	1 each
10	5772A001	Cable Assembly	1 each
11	E1000001	Gas Spring	1 each
12	HP327220	Clevis Pin for Gas Spring	1 each
13	HP349160	Rue Ring Cotter Clip	1 each
14	5770A073	Lower Bracket for Gas Spring (Zinc Plated)	1 each
15	5730F200	Recliner Mechanism	1 each
16	5740A023	Torque Tube Assembly	1 each
17	5792A001G	Side Table Assembly	1 each
18	PP830008G	Side Table Top Only	1 each
19	HP340750	Clevis Pin, Side Table	1 each
20	HS347702	Washer, Side Table	1 each
21	5660A122FR	Side Panel	1 set
22	WC170016	Front Caster	1 each
23	WB170017	Left Rear Caster	1 each
24	WB170034	Right Rear Directional Locking Caster	1 each
25	AP020004-8	Release Push Pin for Side Panel	8/package