
*Note: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com)*

### **IMPORTANT SAFETY INFORMATION: PLEASE READ**

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the 3-in-1 Folding Steel Commode. If you do not understand any part of these warnings, cautions or instructions, contact a health-care professional for direction in the use of this product. If the 3-in-1 Folding Steel Commode is not properly assembled and installed, personal injury and damage to the 3-in-1 Folding Steel Commode could result.**
- ⚠ WARNING: If components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts. Use only Lumex® replacement parts. Non-Lumex® replacement parts could cause personal injury and damage to the 3-in-1 Folding Steel Commode.**
- ⚠ WARNING: This product has a maximum user weight capacity of 300 lb.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

### **INTENDED USE**

The Lumex® 7108R 3-in-1 Folding Steel Commode may be used as a bedside commode, raised toilet seat, and toilet safety frame. It is intended to provide assistance to those who have difficulty sitting down or standing up from the toilet or need additional support and stability during toileting.

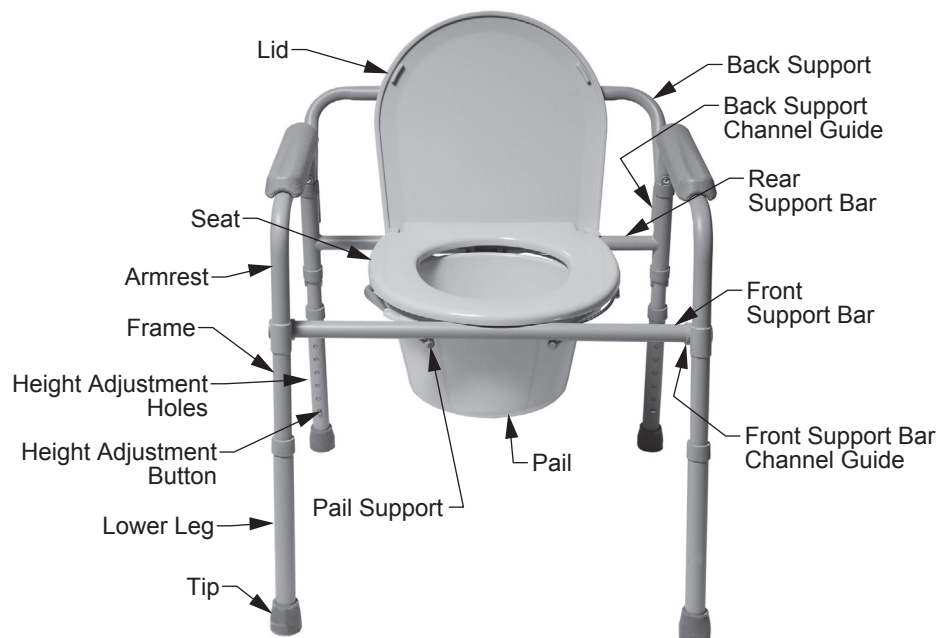**7108R Folding Steel Commode**

## **SETUP**

Refer to picture on previous page.

### **▲ CAUTION: DO NOT use excessive force when installing the seat and lid.**

1. Remove frame, seat, lid, pail, cover, and splash guard from carton.
2. Attach front support bar to armrest by depressing the locking button and sliding into the front support bar channel guide until it clicks into place. Repeat for other side. Ensure the locking buttons protrude fully from the holes.
3. Attach seat to rear support bar. Push down firmly until the seat snaps into place.

### **⚠ WARNING: When installing the seat on the frame, ensure the seat is evenly spaced between the side frames to ensure proper stability.**

4. Attach lid to rear support bar.
5. In the upright position, place splash shield or pail and cover in frame.

## **Adjusting Commode Height**

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.

### **⚠ WARNING: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.**

## **Inserting Back Support**

1. Remove back support from carton.
2. Insert back support into channel guides until back support locking buttons click into place in channel guide holes.

### **⚠ WARNING: Ensure that locking buttons protrude fully through holes before use.**

## **Use Commode over Toilet**

1. Set commode to desired height (described in previous steps).
2. Remove commode pail and lid.
3. Position splash guard in place on pail supports.
4. Remove back support (described in previous steps).
5. Lift toilet bowl seat.
6. Place commode over toilet with splash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

## **Use as Bedside Commode**

1. Set commode to desired height (described in previous steps).
2. Lift commode seat and lid and place commode pail on pail supports. Ensure that pail is properly positioned below commode opening.
3. Place commode seat down. Commode is now ready for use.
4. When user is finished using commode, place commode pail lid over commode pail to contain odors.
5. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

## **MAINTENANCE**

- ⚠ **WARNING: Inspect frequently to ensure that all attaching hardware is tight.**
- ⚠ **WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.**
- ▲ **CAUTION: Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.**

**After each use:** Dry with a soft cloth.

**Cleaning:** Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.

To guarantee the best service from your product, always specify Lumex® replacement parts, available from your GF Health Products, Inc authorized distributor.

## **LIFETIME LIMITED WARRANTY**

GF Health Products, Inc. (“Graham-Field”) hereby warrants specific parts of the Lumex® 3-in-1 Folding Steel Commode Model 7108R for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care.

Parts covered under the lifetime warranty include: frame, welding, screws, and frame-locking components. Specific items excluded from this warranty of any kind are: ALL other parts not specifically listed above. The warranty does not extend to non-durable components, such as tips, which are subject to normal wear and require periodic replacement.

If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.’s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

Made in China © 2010 GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, GA 30360, tel 800-347-5678, fax 800-721-0601

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