## Garantia Limitada

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# LUMEX®

### Features

- Arms drop independently below seat level for lateral transfers
- Easy-to-release drop arm mechanism may be operated by patients lacking manual dexterity
- Arms automatically lock in raised position
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame
- Welded steel construction increases strength and stability
- Cushioned armrests enhance user comfort
- Easy to attach plastic seat [6433A, 6433A-2]
- Easy to attach padded seat [6437A]
- Comes complete with commode pail, cover and splash shield
- 300 lb Weight Capacity EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0165

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2014, GF Health Products, Inc.  
2935 Northeast Parkway, Atlanta, Georgia 30360  
Made in China    [www.grahamfield.com](http://www.grahamfield.com)

GF0500036RevD14

# LUMEX®

## Silver Collection Steel Drop Arm Commode

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Items 6433A, 6433A-2 & 6437A

### User Instructions and Warranty Information

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Importante: Lea Antes de Uso

## Instructions for Use – Please Read All Instructions Before Use.

### Setup

- Remove frame, seat, lid, pail, cover, and splash shield from carton.
- Attach seat and then lid to rear support of frame. Place splash shield or pail and cover in frame.  
**NOTE:** When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.  
**NOTE:** Push down firmly until the seat snaps into place.  
**NOTE:** DO NOT use excessive force when installing the seat and lid.

### Adjusting Leg Height

- Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.  
**NOTE:** An audible “click” will be heard.
- Repeat for other 3 legs.  
**CAUTION:** Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

### Inserting Back Bar

- Remove back bar from carton.
- Insert back bar into channel guides on back of commode until back bar push buttons protrude fully through channel guide holes ( you will hear an audible click).

### Use Commode over Toilet

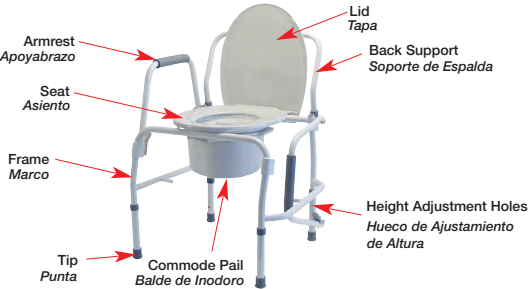
- Set commode to desired height (described in previous steps)
- Remove commode pail and lid
- Position splash shield in place onto pail supports
- Remove back bar (described in previous steps)
- Lift toilet bowl seat
- Place commode over toilet with slash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

### Use as Bedside Commode

- Set commode to desired height (described in previous steps).
- Lift commode seat and lid and place commode pail onto pail supports. Ensure that pail is properly positioned below commode opening.
- Place commode seat down. Commode is now ready for use.
- When use is finished using commode, place commode pail lid over commode pail to contain odors.
- Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

### Arm Adjustment

- Drop Armrest:** to drop the armrest, push the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

## Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

## WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Users with limited physical capabilities should be supervised or assisted when using the commode.

**WARNING:** Item has a maximum weight capacity of 300 lb. EVENLY DISTRIBUTED.

**WARNING:** GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

**WARNING:** Leg extensions with rubber tips MUST be in contact with the floor at all times.

**WARNING:** Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

**WARNING:** Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

**WARNING:** Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

**WARNING:** The commode seat MUST be installed before sitting on the commode.

**WARNING:** Notice for California Customers  
**California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** If components are damaged or missing, contact your GF Health Products Inc. distributor immediately.

## Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

### ENTIRE WARRANTY. EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

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