

ONE-YEAR LIMITED WARRANTY
SCOPE OF WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original purchaser only that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF’s customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. The warranted components and time period are set forth below:

GF13700: Pull-Cord Patient Alarm

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

A GF Customer Service Representative must authorize warranty service. Please contact the GF Customer Service department by calling 678-291- 3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the Bill of Lading and must be made with immediacy. The ICC regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within thirty (30) days of the invoice date.

Manufactured for
GF Health Products, Inc.
Atlanta, GA 30360



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

Made in China

© 2014 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360
tel 800-347-5678 • fax 800-721-0601



GF13700: PULL-CORD PATIENT ALARM
SETUP & OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Note: The most current version of these instructions can be found online at www.grahamfield.com

IMPORTANT SAFETY INFORMATION: PLEASE READ

⚠ WARNING: Important! Read and understand these instructions before installing or using the GF13700 Pull-Cord Patient Alarm. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the patient alarm is not properly set up and adjusted, personal injury and/or damage to the patient alarm could result.

⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex® replacement parts. Use ONLY the included magnet and pull-cord with this alarm; DO NOT use other types or lengths of cord. Non-Lumex® replacement parts could cause personal injury and/or damage to patient alarm.

⚠ Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

UNPACKING

Check for any obvious damage to the carton or its contents. If damage is evident, DO NOT USE. Contact carrier / distributor for further instruction.

INTENDED USE

The intended use of the GF13700 Pull-Cord Patient Alarm is to alert caregivers when a patient attempts to exit a wheelchair, chair or bed without assistance. The alarm activates when the magnet is removed from the metal disk on the monitor.

Contraindications

⚠ WARNING: This device is not a substitute for routine visual monitoring by caregiver, and is intended to be used in conjunction with a comprehensive fall management program.

⚠ WARNING: No claim is made that this device will prevent patient falling or reduce the likelihood of injury as a result. The caregiver is responsible for ensuring that this product is properly installed, tested, and functioning before each use.


⚠ WARNING: When placed nearby, the patient alarm’s magnet may affect the patient’s pacemaker or ICD (Implantable Cardioverter Defibrillator). Ensure the magnet and pull-cord are always more than 8” from these devices.

▲ CAUTION: The garment clip may have difficulty gripping slippery, thin fabric; ensure the garment clip always securely grips the patient’s garment.

▲ CAUTION: Ensure the pull-cord is always free of interference from environmental factors such as sweaters, blankets and pillows.

▲ CAUTION: This device is not suitable for all patients, including but not limited to, those wishing to defeat the alarm system.

ALARM SETUP

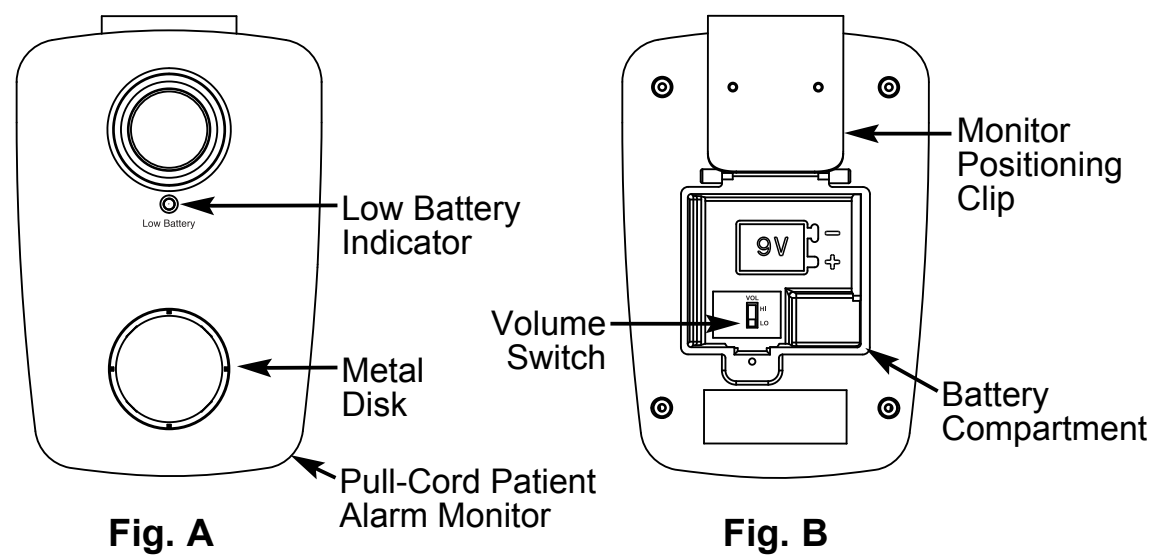

1. Remove the screw that holds the battery compartment cover in place.
2. Inside the battery compartment (see Fig. B):
 - a. Set the volume switch to the HI or LO position depending on desired alarm volume (see Fig. B).
 - b. Install a new 9V alkaline battery (not included).
 - c. Replace battery compartment cover and reinstall screw.

Alarm Testing

⚠ WARNING: Test alarm before each use.

1. Attach magnet to metal disk on front of monitor (see Fig. A).
2. Remove magnet from metal disk on front of monitor; when magnet is removed from the monitor the alarm will sound.
3. To silence the alarm, place magnet on metal disk on front of monitor.

Patient Setup

1. Follow this procedure before each use.
 - a. If using alarm with a chair or wheelchair, use the monitor positioning clip on the back of the monitor (see Fig. B) to mount the monitor securely on the chair or wheelchair.
 - b. If using alarm with a bed (optional), clip the monitor to the optional bed mounting bracket.
2. Attach the garment clip to the patient’s clothing between the shoulders near the back of the neck.
3. Use the adjuster to modify the pull-cord length based upon the amount of movement you wish the patient to have before alarm activates.
4. Place magnet on metal disk on front of monitor (see Fig. A).
5. When magnet is removed from the monitor the alarm will sound. To silence alarm, replace magnet on metal disk on front of monitor.

⚠ WARNING: If alarm fails to sound when magnet is removed from monitor, immediately discontinue use and review alarm setup.

MAINTENANCE

⚠ WARNING: Test the alarm before each use. If alarm fails to sound, remove it from service immediately. Check before each use that all alarm components are functional and battery is charged. To replace battery, see ALARM SETUP section on previous page.

Note: The Low Battery indicator on the front of the alarm will flash when battery is low and needs to be replaced. Replace battery immediately if indicator flashes (see Fig. A).

Cleaning

1. To clean: Gently wipe alarm components with a soft, clean cloth.
- ▲ CAUTION: Do not use cleansers that could damage the alarm's finish. Do not submerge any alarm components in liquid.**