



527 SERIES GOLDEN CROWN TRUSS



INSTRUCTIONS FOR USE

Read these instructions before using the Pipeer Golden Crown Truss. Save these instructions for future reference.

The most current version of these instructions can be found online at www.grahamfield.com.

Safety

- ⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

Intended use

The intended use of the Truss is the treatment of reducible inguinal hernia. The Truss may be worn at any time.

Info: Truss may be right-sided, left-sided or double-sided. Double-sided Truss is shown on front cover; single-sided Truss is shown in instructions.

Application

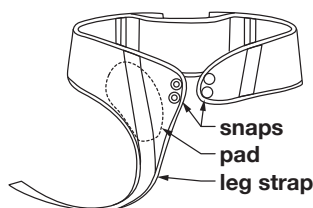


fig. 1

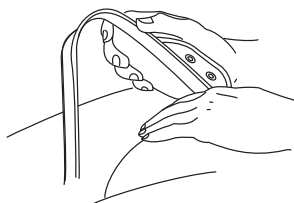


fig. 2

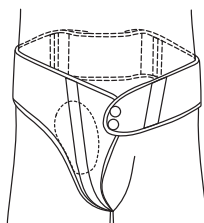


fig. 3

1. Open the front snaps.
2. Unbuckle the leg strap(s).
3. If rupture is protruding, lie down. Reduce rupture size to normal by using the palm of your hand to gently push rupture upward and in as shown in fig. 2. If necessary, cough to locate exact position of rupture opening. **BE SURE TO PLACE THE PAD AT THIS EXACT OPENING.**
4. Snap the Truss around your abdomen as shown in fig. 3. **ENSURE THE PAD IS PLACED AT THE POINT OF RUPTURE.**

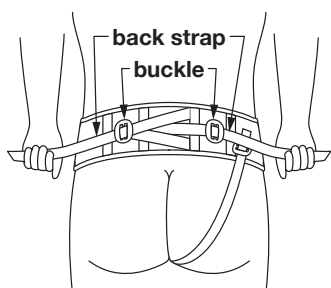


fig. 4

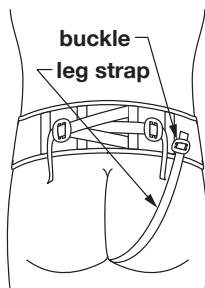


fig. 5

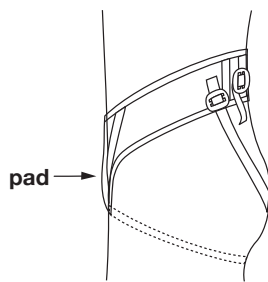


fig. 6

5. Pull the adjustable back straps outward as shown in fig. 4 until the Truss fits snugly. When the Truss feels secure, release the strap ends, as shown in fig. 5. The buckles will automatically prevent the straps from slipping.
6. Adjust the leg strap with the buckle at back of Truss as shown in fig. 5 until the fit is comfortably snug. Test for proper adjustment of Truss by sitting down. You should be able to sit down comfortably wearing the Truss. If not, readjust.

Info: When the Truss is properly adjusted, Truss back is above hip line as shown in fig. 6. The pad must be below abdomen at point of rupture as shown in fig. 6.

Info: One adjustment is usually sufficient. After Truss is adjusted properly, put it on as you would any other undergarment.

Removal

To remove Truss, unsnap the front and step out of the Truss. Check the Truss after each laundering to ensure buckles are correctly positioned.

Cleaning and Maintenance

Hand wash Truss gently, using mild soap. Air dry.

▲ **NOTICE: DO NOT use detergent or bleach to clean Truss. DO NOT iron Truss.**

Info: Repeated laundering may remove the wax lubricant from snaps. If snaps become difficult to operate, rub a little candle wax on the snaps.

Thirty-Day Limited Warranty

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Pipeer 527 Series Golden Crown Truss is warranted for thirty days. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.



Manufactured by:
GF Health Products, Inc.
Atlanta, Georgia 30360
Proudly made in the U.S.A.

GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
telephone: 770-368-4700
fax: 770-368-2386
www.grahamfield.com

© 2012 GF Health Products, Inc.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com. **Graham-Field** and **Pipeer** are registered trademarks of GF Health Products, Inc.