




BASIC AMERICAN
MEDICAL PRODUCTS

ZENITH
SERIES

Zenith 5000

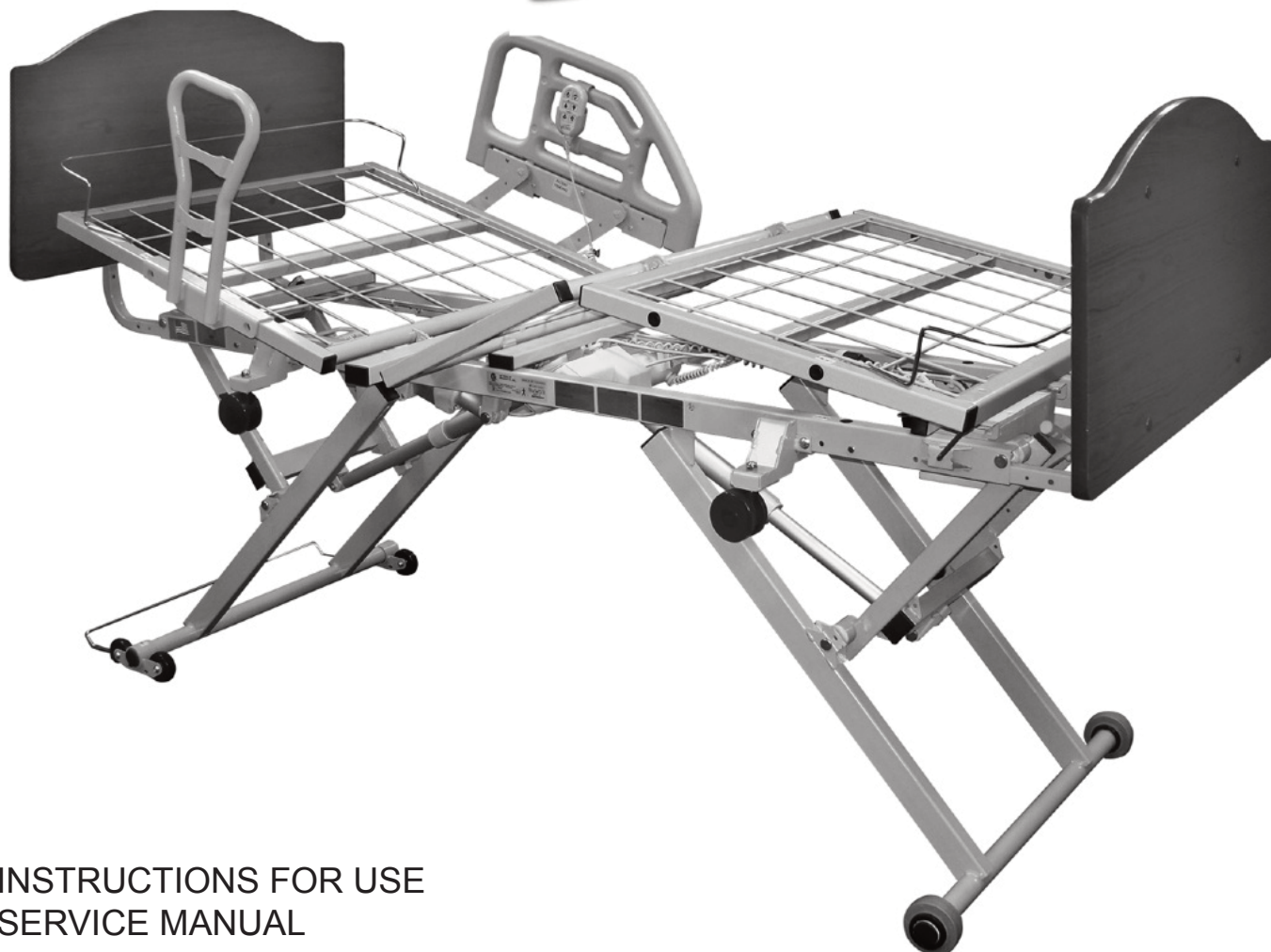

INSTRUCTIONS FOR USE SERVICE MANUAL

*To avoid personal injury or damage to bed,
please read all sections pertaining to
the bed model before use.*

ETL/UL/CSA APPROVED
UL 60601-1
IEC 60601-2-38
CSA 22.2 601.1

This service manual covers the following Zenith Models:

76" and 80" Full Electric with Low Roll Only and Grid Decks = ZG58174



Basic American Medical Products

336 Trowbridge Drive
Fond du Lac, WI 54937

For Zenith 5000 Bed service parts,
please contact our Customer Service Department
at 770-368-4700

Manufactured by:
GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
www.grahamfield.com

Made in the USA

To order a Zenith 5000 bed,
please contact a GF Health Products, Inc. sales representative
at 770-368-4700

IMPORTANT NOTICE

GF Health Products, Inc. is not responsible for typographical errors.
All illustrations, specifications, packaging, and warranties contained in this Service Manual
are based on the latest product information available at the time of printing.
The most current product information can be found online at www.grahamfield.com.

Please check all parts for shipping damage and test before using.
In case of damage, DO NOT USE.



TABLE OF CONTENTS

IMPORTANT SAFETY AND WARNING INFORMATION	4
IMPORTANT SAFETY AND WARNING INFORMATION	5
ENTRAPMENT AND COMPLIANCE INFORMATION	5
RECOMMENDED MAINTENANCE	6
ENVIRONMENTAL SPECIFICATIONS AND RFI INFORMATION	7
SECTION A - BED SET-UP	8
UNPACKING THE BED	9
HEADBOARD AND FOOTBOARD ASSEMBLY	10
STANDARD MATTRESS RETAINER INSTALLATION	11
STANDARD WIREFORM WALLSAVER INSTALLATION	12
HAND CONTROL PENDANT OPERATION	13
BED OPERATIONS - STATIONARY/LOW ROLL POSITIONING	14
SECTION B - ASSIST DEVICES	15
OPTIONAL PIVOTING ASSIST BAR INSTALLATION	16
PIVOTING ASSIST BAR OPERATION	16
OPTIONAL HALF RAIL INSTALLATION	17
HALF RAIL OPERATION	17
OPTIONAL TRAPEZE SUPPORT INSTALLATION	18
SECTION C - MECHANICAL AND ELECTRICAL INFORMATION	19
SECTION D - SERVICE/REPLACEMENT PARTS	23
SECTION E - DECK/MECHANICAL SERVICE PARTS	28
SECTION F - TROUBLESHOOTING	32
LIMITED WARRANTY	35

LABEL SYMBOL DEFINITIONS



Consult
Accompanying
Documents



Safe
Working
Load



Double
Insulated



Protected
Grounded
Device



Type B
Equipment and
Applied Parts



IMPORTANT SAFETY AND WARNING INFORMATION



This product is a variable height, adjustable mattress platform, which will provide comfort and convenience for residents/patients and caregivers in long term care settings.



The **MAXIMUM SAFE WORKING LOAD** for the Zenith 5000 Bed with weight evenly distributed, including bedding, resident/patient, support surface, and all accessories, is 500 lb.



NEVER operate the bed if a Power Cord or Plug is damaged or not working properly. Contact qualified Service Personnel for examination and repair. Always unplug the Power Cord when performing any maintenance on the bed.



DO NOT open assemblies such as the Actuators, Hand Control, or Control Box. If unauthorized personnel perform work on these components, the manufacturer's warranty becomes void.



DO NOT use unauthorized parts, accessories, or adaptors other than those specified/authorized by GF Health Products, Inc.



When operating the HI/LO, Knee, or Back Functions of the bed, **ALWAYS** ensure that the confined individual is positioned properly within the confines of the bed.



DO NOT let any extremities protrude over the side or between the bed rails when performing these functions.



DO NOT lower the bed when objects are beneath it. Failure to inspect under the bed can result in damage to property or personal injury.



The bed's Hand Control Pendant Cable **MUST BE ROUTED AND SECURED PROPERLY** to ensure it does not become entangled and eventually severed during use. Also make sure all electrical cords **DO NOT** get tangled around the bed, side rails, or legs during transport or normal operation of the bed.



When using nasal-type or masked-type administering equipment, all oxygen or air tubing **MUST BE ROUTED AND SECURED PROPERLY** to ensure that the tubing does not become entangled and eventually severed during the normal operation of the bed.



Keep all moving parts free of obstructions (i.e. blankets/sheets, heating blankets/pads, wiring, etc.).



Beds manufactured by Basic American Medical Products are designed for use within an institutional healthcare environment (i.e. assisted living, skilled nursing, transitional care, rehabilitation care, etc.).



DO NOT use the assist devices as push handles for moving the bed. Assist devices can be deformed or broken if excessive side pressure is exerted. Assist devices are not meant for patients considered as high risks for entrapment (i.e. patients with pre-existing conditions such as confusion, restlessness, lack of muscle control, altered mental status, either organic or medicinal, or a combination thereof). Additional safety measures should be considered for such high-risk patients.



NEVER permit more than one (1) person on/in the bed at any time.



Body weight should be evenly distributed over the sleeping surface of the bed. **DO NOT** allow the patient to lie, sit, or lean in such a way that their entire body weight is placed only on the raised head or foot sections of the bed. This especially applies when repositioning or transferring a patient in or out of the bed. Increased risk may occur when the patient's size and/or weight are inappropriate for the bed's dimensions or weight capacity.



Risk of entanglement or injury may occur if the mattress used with mattress retainers does not fill the entire width between stops or which compresses to less than 1.50 inches under user's weight.



Mattress must be properly sized to fit the mattress support platform and must remain centered on the support platform relative to State and Federal guidelines. Recommended minimum dimensions of mattress is 35 inches wide and 6 inches deep. Length should match the mattress support platform. Use of an improperly fitted mattress could result in injury or death.



IMPORTANT: Powered air mattress surfaces may pose a risk of entrapment. Prior to use, ensure the therapeutic benefits outweigh the risk of entrapment.



The bed is intended for use within a temperature range of 10°C to 40°C. It has a water resistance rating of IPX4 and **IS NOT** to be power washed or submerged.



IMPORTANT SAFETY AND WARNING INFORMATION



The head/back and knee/foot decks can be lifted freely by hand for easy cleaning access when patients are not in the bed. If you lift the head/back or knee/foot deck for any reason, please take great care when lowering back down to the prone position - make sure that all body parts are clear of the space between the deck and the bed prior to slowly lowering any deck manually. To avoid injury, **DO NOT LET DECKS FALL FREELY FROM ANY ANGLE.**



Notice for California Customers - California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.



Proper routing and tie-off of electrical cabling, especially the power cord, is essential for proper operation and to ensure safety from electrical shock. In the event you are replacing any electrical cabling on your bed, you must make sure the cables are free from any pinch points, obstructions, or stretched so tight that they may come loose or become damaged. In addition, cables should be tied off in such a way to secure them and keep them free from tangling on any part of the bed during normal operation. Refer to your service manual's electrical section for proper cable routing.

ENTRAPMENT AND COMPLIANCE INFORMATION

On April 10, 2006, the FDA (U.S. Food and Drug Administration) released long-awaited guidelines for reducing the risk of bed entrapment: "Hospital Bed System Dimensional and Assessment Guidance to Reduce Entrapment". The new Guidance identifies potential entrapment areas and those body parts most at risk for entrapment; provides design criteria for manufacturers of new hospital/convalescent beds; recommends particular test methods to assess the conformance of existing hospital/convalescent bed systems; and answers frequently-asked questions about entrapment issues.

The new Guidance was a result of a long-standing collaboration between the FDA and the Hospital Bed Safety Workgroup (HBSW), formed in 1999. GF Health Products, Inc.'s Long Term Care Bed division: Basic American Medical Products, is an HBSW charter member. As a result of our commitment to product safety, **all our current long-term care beds have been strictly tested and conform to the new FDA Guidance.**

The guidelines set forth by the FDA Guidance layout specific dimensional limitations on potentially injury-threatening gaps and spaces that can occur between bed system components, such as rails, when not properly installed. GF Health Products, Inc. and Basic American Medical Products have conformed to these guidelines from a manufacturing aspect. However, entrapment issues can often arise when a healthcare provider/facility has not correctly assembled the components on a bed. It is essential that the provider/facility fully understand their responsibility in complying to the guidelines set forth by the FDA in order to avoid injury. To that end, we have provided the FDA's web address at right as a resource for understanding and following these guidelines for the safety of patients/residents.

It is also essential to have the correct bed components/accessories that correspond with the needs of the patient/resident and the particular bed you have purchased. Matching the correct bed component that correlates with the regulatory guidelines can be a daunting task. Our sales team at GF Health Products, Inc. and our friendly Customer Service Representatives at Basic American Medical Products can help you sift through the wide array of compliance and bed options. We will help you determine which bed/bed part is best for the patient's/resident's particular needs and help you with any compliance issues.

The Zenith 5000 bed and accessories listed in this manual are in full compliance with FDA guidelines for reducing the risk of bed entrapment: "Hospital Bed System Dimensional and Assessment Guidance to Reduce Entrapment".

Details can be found at www.fda.gov.



*Regular maintenance of the Long Term Bed is necessary to ensure continuing proper and safe operation.
Please read and observe the following recommended maintenance schedule.*

ITEM	RECOMMENDED INSPECTION PERIODS		
	Inspect on Receipt	Every 3 Months	Every 6 Months
Maintenance Inspection of All Components at Receipt of Shipment			
Make sure all parts/components are included (Please see "Unpacking The Bed").	X		
Check all bed components for obvious damage.	X		
Inspect the power cord for any cuts and/or damage.	X		
Check to see all actuator/motor cables are routed and connected properly to the control box.	X		
Mechanical Inspection of Assemblies			
Inspect all welds on the sleeping surface, frame, and base assemblies for stress fractures.			X
Inspect all fasteners for wear and looseness.			X
Mechanical Inspection of Casters and Pedal Locking Mechanism			
Check the stationary foot pads on both the head end and foot end for any damage or wear. Replace if needed.		X	
Check all rolling casters to ensure that they roll properly and are unobstructed.		X	
Electrical Inspection of Control Box and Hand Control Pendant			
Check the external power cord that plugs into the control box for any chafing, cuts, or wear. Replace if damaged.		X	
Make sure all attaching hardware is securely tightened.		X	
Check all electrical connections for any wear or fractures.		X	
Check the external backup battery (if you have one). Replace if needed.		X	
Check the hand control pendant cable for any chafing, cuts, or wear.		X	
Check all hand control pendant functions - check to make sure each button and associated function work properly (i.e. head section rises when the HEAD UP button is activated).		X	
Electrical Inspection of Actuators/Motors			
Check the actuator/motor cables for any chafing, cuts, or wear.		X	
Check the range of movement on all motors to ensure they do not bind in the Full Up or Full Down positions.		X	



The following warning labels have been placed on the bed to help protect you and the bed from damage. Please do not remove any labels from the bed.

WARNING!

DO NOT LOWER BED WHEN OBJECTS ARE BENEATH BED. FAILURE TO INSPECT UNDER BED CAN RESULT IN DAMAGE TO PROPERTY OR PERSONAL INJURY.

ATTENTION:

S'assurer de ne pas faire descendre le lit lorsque des objets se trouvent sous le lit. Ne pas inspecter le dessous du lit pourrait entraîner des dommages matériels et des risques de blessures.



WARNING!

DO NOT LOWER BED WHEN OBJECTS ARE BENEATH BED. FAILURE TO INSPECT UNDER BED CAN RESULT IN DAMAGE TO PROPERTY OR PERSONAL INJURY.

CAUTION

THIS BED IS SUITABLE FOR USE ONLY WITH OXYGEN ADMINISTERING EQUIPMENT OF THE NASAL OR MASK TYPE OR A TENT COVERING ONLY THE UPPER HALF (HEAD END) OF THE BED. OXYGEN TENT CANOPIES SHOULD NOT EXTEND BELOW BED SPRING LEVEL. LOCK HAND CONTROL AT FOOT OF BED WHEN USING OXYGEN ADMINISTERING EQUIPMENT.

ATTENTION:

CE LIT PEUT ETRE UTILISE UNIQUEMENT AVEC UN EQUIPMENT DESTINE A L'ADMINISTRATION D'OXYGENE DE TYPE NASAL OU MASQUE OU AVEC UNE TENTE RECOUVRANT SEULEMENT LA MOTTIE AVENT (TETE) DU LIT. LES COTES DE LA TENTE OXYGENE NE DOIVENT PAS SE PROLONGER PLUS QUE LA SOMMIER DU LIT.

ENVIRONMENTAL SPECIFICATIONS AND RFI INFORMATION

OPERATING CONDITIONS

Ambient Temperature	10°C to 40°C
Relative Humidity	30% to 75% Non-Condensing
Atmospheric Pressure	700 to 1060 hPa
Protected Against Splashing Water	IPX4

STORAGE AND TRANSPORT CONDITIONS

Temperature	-40°C to 70°C
Relative Humidity	10% to 100%
Atmospheric Pressure	500 to 1060 hPa

RFI (RADIO FREQUENCY INTERFERENCE)

RFI influences most electronic equipment. Caution should be exercised with regard to the use of portable communications equipment in the area around such equipment. If RFI causes erratic behavior, shut the bed off immediately. Leave it off while the transmission is in progress.



SECTION A

ZENITH 5000

Bed Set-up Instructions



UNPACKING THE BED

- Make sure all parts/components are included.
- Check all bed components for obvious damage.
- Inspect the Power Cord for any cuts and/or damage.
- Check to see all actuator/motor cables are routed and connected properly to the control box.

DISCARD


1. Cable Tie on Bundle
2. Large Foam Block
3. Large Cable Tie - CUT
4. Notched 2 x 4 Board
5. Foam Wrapping
6. End Caps with Foam
7. Small Cable Tie - CUT
8. Notched Leg Foam

KEEP

9. Wireform Wallsaver
10. Two Mattress Retainers
11. Service Manual/Documents
12. Power Cord, Hand Control Pendant, and Pendant Holster Bundle

NOTE:
END OF POWER CABLE IS COILED FOR SHIPPING AND TIED, WITH CABLE TIE, TO GRID WIRE ALONG WITH HAND CONTROL PENDANT, PENDANT CABLE AND HOLSTER.

PLEASE CUT AND DISCARD CABLE TIE AROUND CABLES WHEN YOU UNPACK THE BED.

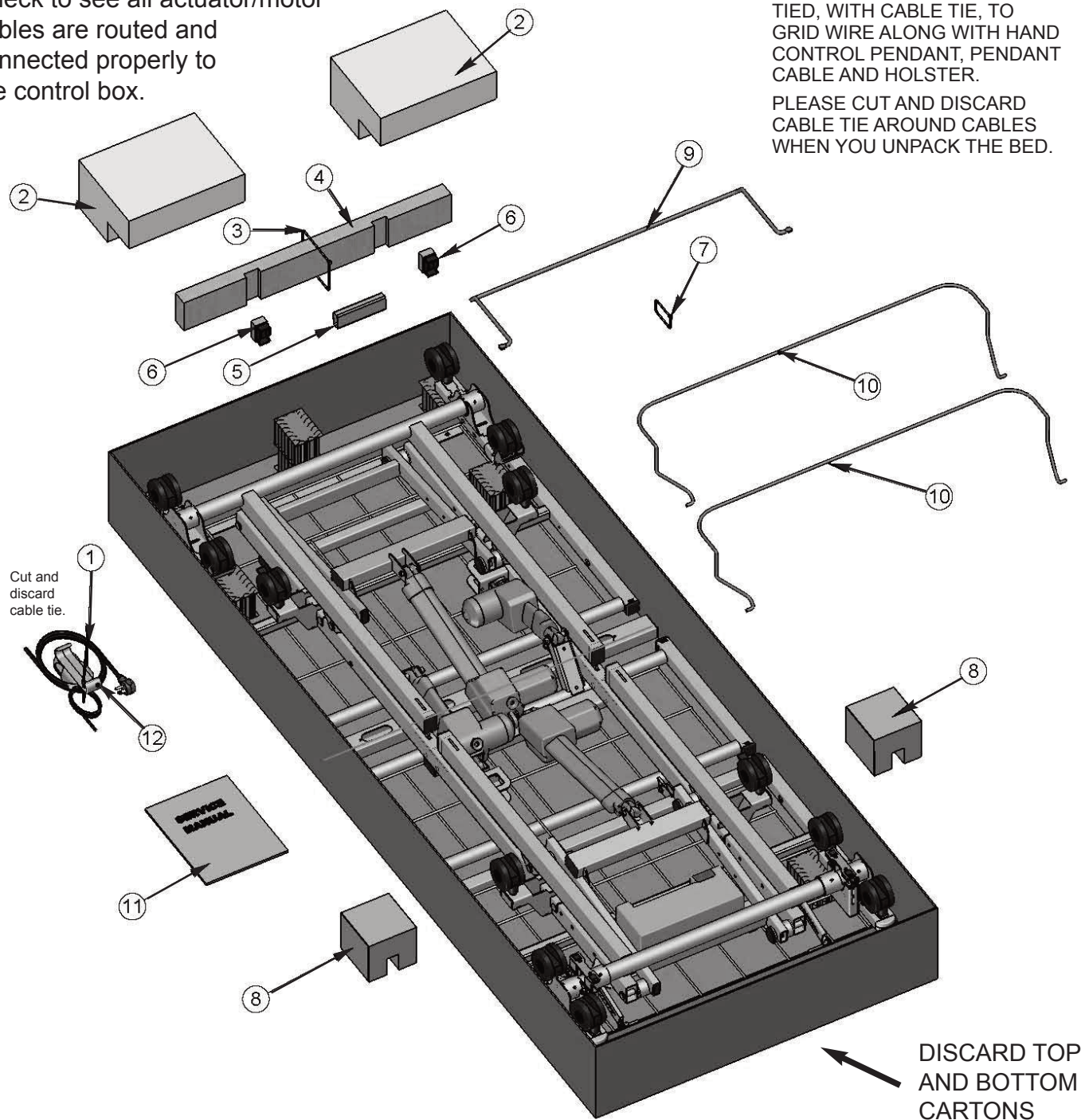




HEADBOARD AND FOOTBOARD ASSEMBLY

1. HEADBOARD INSTALLATION

- The headboard comes with four pre-installed inserts - consider this the inside of the headboard.
- Position two mounting tubes on the outside of the headboard with "L" facing inward.
- Align the mounting tube top holes with the headboard top holes.
- Insert a 40mm long socket head cap screw through each of the top holes and bottom holes and screw into each insert. Tighten with the hex key included in the kit.
- At the head deck end, slide the mounting tube "L" portions into the main frame rail hollow ends.
- **FOR 80" BEDS:** Slide the mounting tubes in until the first tube hole aligns with the first rail hole. See sample below.
- **FOR 76" BEDS:** Slide the mounting tubes in until the first tube hole aligns with the second rail hole. See sample below.


HEADBOARD

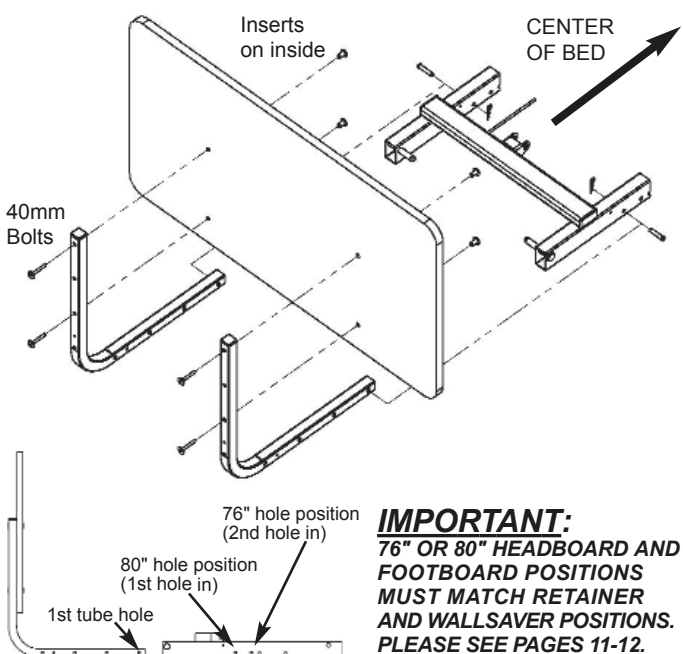

2. FOOTBOARD INSTALLATION

- The footboard comes with four pre-installed inserts - consider this the outside of the footboard.
- Position two mounting tubes on the inside of the footboard with "L" facing outward.
- Align the mounting tube top holes with the footboard top holes.
- Insert a 40mm long socket head cap screw through each of the top holes and bottom holes and screw into each insert. Tighten with the hex key included in the kit.
- At the foot deck end, slide the mounting tube "L" portions into the main frame rails hollow ends.
- **FOR 80" BEDS:** Slide the mounting tubes in until the first tube hole aligns with the second rail hole. See sample below.
- **FOR 76" BEDS:** Slide the mounting tubes in until the first tube hole aligns with the third rail hole. See sample below.

FOOTBOARD

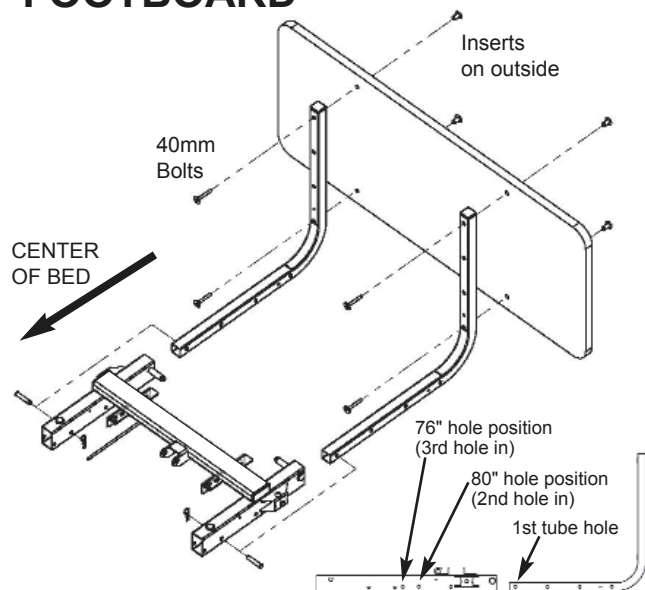


NOTE: The 1st hole at the foot end is reserved for attaching the Optional 4" pan extension.



STANDARD MATTRESS RETAINER INSTALLATION (Using 2 Retainers 999-0844-124)

1. Determine if you need to position your bed in an 80" or 76" configuration (80" outside holes; 76" inside holes - see Details A and B).
2. On the foot end, carefully squeeze the Mattress Retainer ends inward, toward the center of the retainer, and insert the ends into the proper holes (see Details A and B).
3. Lay the Mattress Retainer down so that the elbows rest on the decks and the long cross rod is on the foot end side as shown.
4. On the head end, carefully squeeze the Mattress Retainer ends inward, toward the center of the retainer, and insert the ends into the proper holes (see Details A and B).
5. Lay the Mattress Retainer down so that the elbows rest on the decks and the long cross rod is on the foot end side as shown.

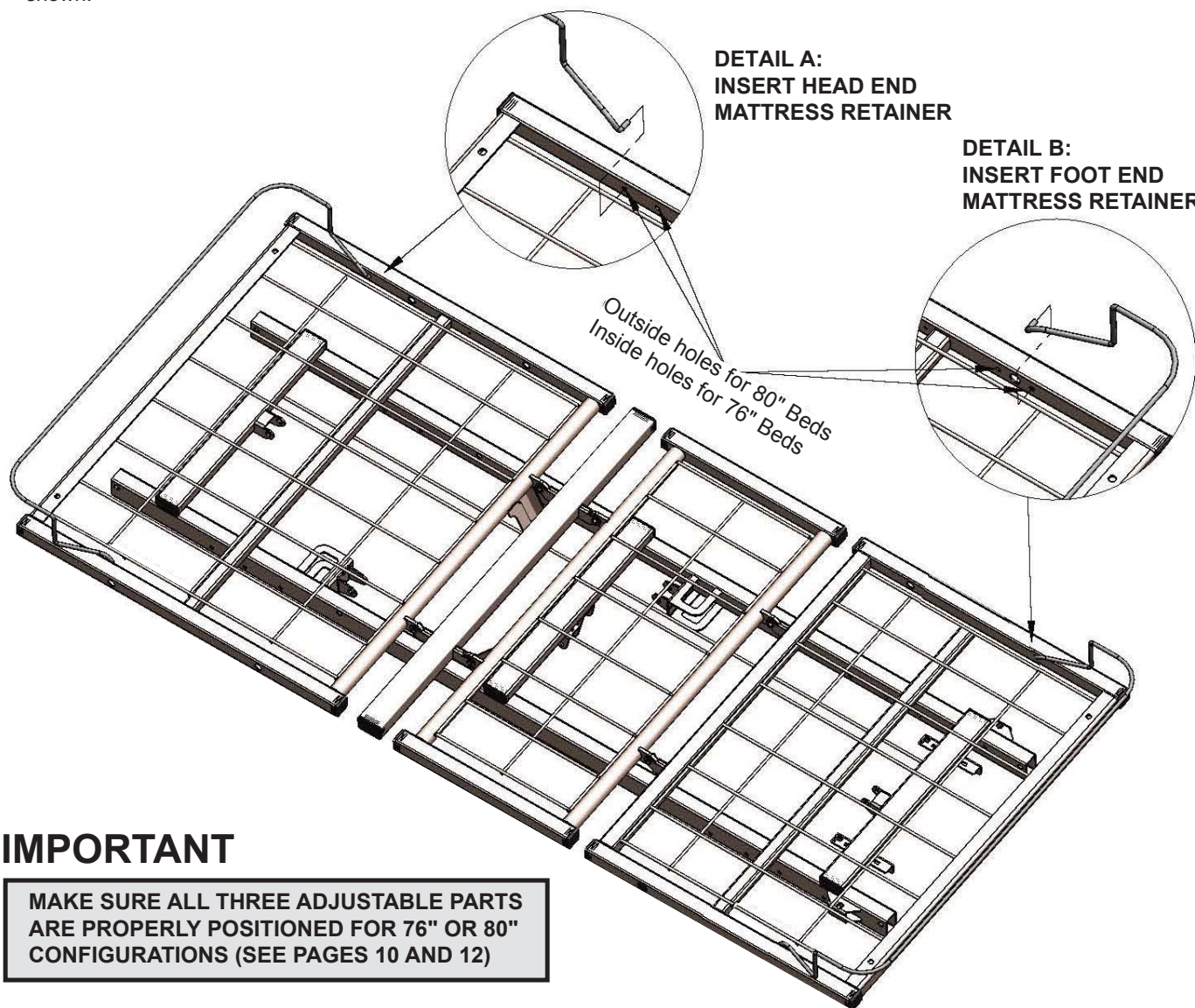

Be sure to use a mattress that is properly sized to fit the sleep deck, which will remain centered on the deck relative to State and Federal Guidelines. Use of an improperly fitted mattress could result in injury or death.






Use a properly sized mattress in order to minimize the gap between the side of the mattress and assist devices. This gap must be small enough to prevent resident/patient from getting their head or neck caught in this location.



STANDARD WIREFORM WALLSAVER INSTALLATION

WALLSAVER ASSEMBLY

1. Position the Wireform Wallsaver (Part no. **999-0856-180SP**) with bent end facing upward and tab ends facing inward as shown at right.
2. Determine the position desired (see bottom of page).
3. Gently squeeze the wallsaver tab ends inward toward the center of the wallsaver and, holding the tabs parallel with the caster base holes (see DETAIL A), slide the tabs into the holes while letting the wallsaver gently expand outward.
4. Turn the wallsaver downward until it rests on the floor (See Detail B).

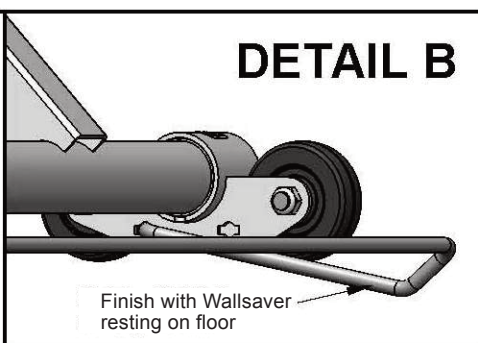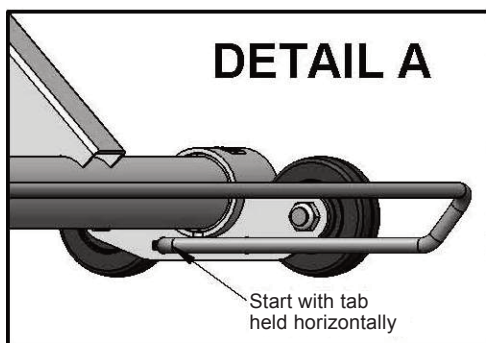
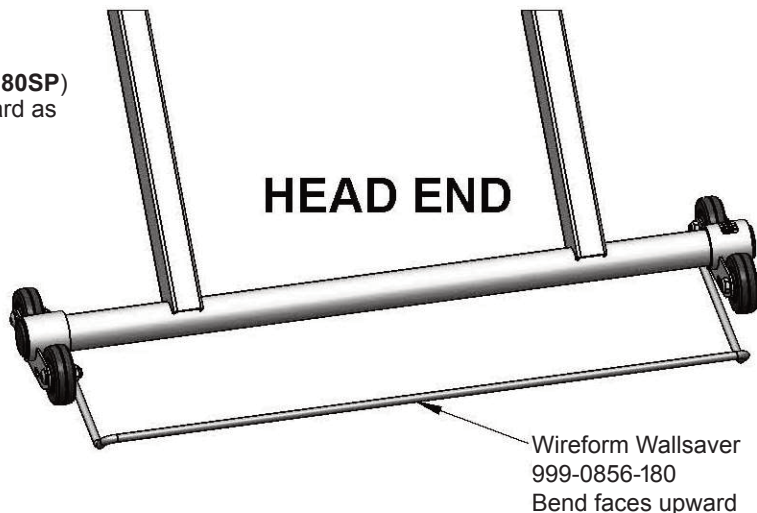


WALLSAVER REMOVAL

1. To remove or move the wallsaver to a new position, raise the wallsaver off the floor until the end tabs are horizontal.
2. Squeeze the ends toward the center of the wallsaver until the end tabs slide out of the caster base holes.

WALLSAVER POSITIONS

1. The Zenith 5000 wallsaver features two standard positions for easy bed/mattress length reconfigurations - 76" and 80".
2. For 76" beds, position the wallsaver ends in the BACK caster base holes (toward the foot end). For 76" beds using an optional Trapeze unit, position the wallsaver ends in the FRONT caster base holes (toward the head end.)
3. For 80" beds, position the wallsaver ends in the FRONT caster base holes (toward the head end).

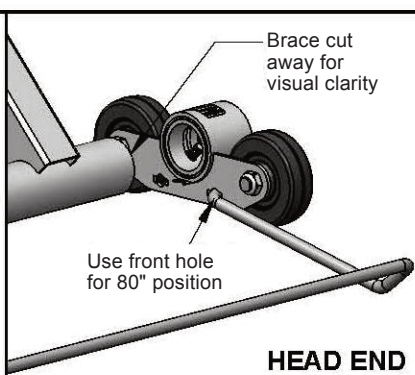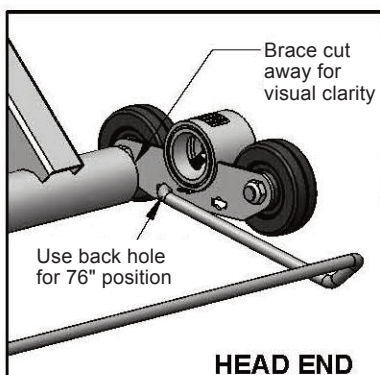

IMPORTANT NOTE: WHEN POSITIONING THE BED'S WALLSAVER FOR A PARTICULAR BED LENGTH (I.E. 76" OR 80"), ENSURE THE OTHER BED ADJUSTMENTS (HEADBOARD AND FOOTBOARD POSITIONS AND MATTRESS RETAINER POSITION) ARE IN THE SAME CONFIGURATION. PLEASE REFER TO PAGES 10 AND 11.



ZENITH 5000 HAND CONTROL PENDANT OPERATION

The Hand Control Pendant comes standard on the Zenith 5000 series.

The plug features a lock-on cap.

A pendant holster for attaching to optional assist devices is also provided.



HAND CONTROL PENDANT OPERATION	
1	HEAD DECK ANGLE UP BUTTON
2	HEAD DECK ANGLE DOWN BUTTON
3	HI/LO UP BUTTON (RAISE ENTIRE BED)
4	HI/LO DOWN BUTTON (LOWER ENTIRE BED)
5	KNEE AND FOOT DECK ANGLE UP BUTTON
6	KNEE AND FOOT DECK ANGLE DOWN BUTTON


The Hand Control Pendant Holster 999-0791-000SP comes standard with the Zenith 5000 bed.



HAND CONTROL PENDANT

The Zenith 5000 bed comes with Alligator Clip attached to Hand Control Pendant Cable. To order a replacement Hand Control Pendant, please see page 25.

The Hand Control Pendant Cable has a phone jack connector.

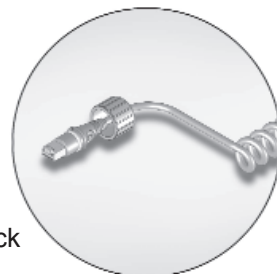

NOTE:

THE ZENITH 5000 HAND CONTROL PENDANT CABLE RUNS ALONG THE SEAT PAN, THEN DOWN TOWARD THE CONTROL BOX, AND PLUGS INTO A SMALL BLACK CONTROL CABLE WHICH PLUGS DIRECTLY INTO THE CONTROL BOX.


IN ORDER TO MAXIMIZE THE EASE OF USE AS WELL AS MAINTAIN THE LONGEVITY OF THE ZENITH 5000 BED, THE MICROPROCESSOR-CONTROLLED CONTROL SYSTEM MONITORS ALL HAND CONTROL PENDANT INPUTS. THE HAND CONTROL PENDANT IS INTENDED FOR USE BY THE PATIENT TO MAXIMIZE COMFORT OR BY THE CAREGIVER FOR REPOSITIONING.

TO DETER MISUSE BY A PATIENT, IF THE SYSTEM DETECTS REPEATED BUTTON OPERATIONS, IT WILL BLOCK THE NEXT SIGNALS TO ENSURE THE WELLBEING OF THE PATIENT AND TO PROTECT THE SYSTEM. SHOULD THIS OCCUR, THE USER WILL NOTICE A TWO (2) SECOND DELAY WHEN DEPRESSING A FUNCTION, OR MUST WAIT FOR TWO SECONDS BEFORE OPERATING THE BED AGAIN.




BED OPERATIONS - STATIONARY/LOW ROLL POSITIONING

The Zenith 5000 is sold as a "Low Roll" bed to give the user the option of placing it in a stationary (non-moving) position or lowering it into a "low roll" position. This is accomplished by lowering the bed until the stationary foot pads rise off the floor and the rolling casters contact the floor surface for easy mobility.

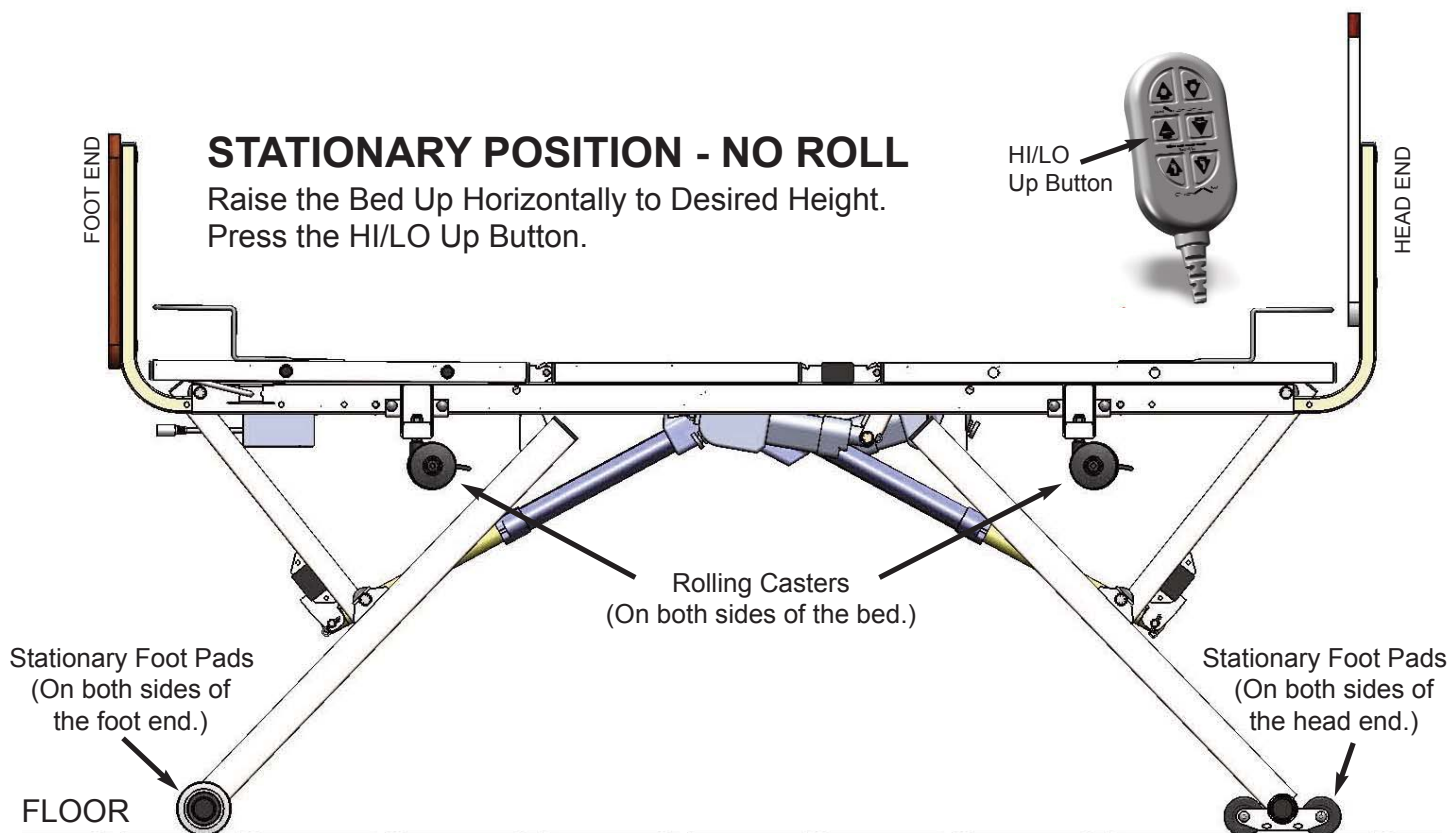

MOBILE POSITION - LOW ROLL

Lower the Bed Down Until the Rolling Casters Engage the Floor.
Press the HI/LO Down Button.

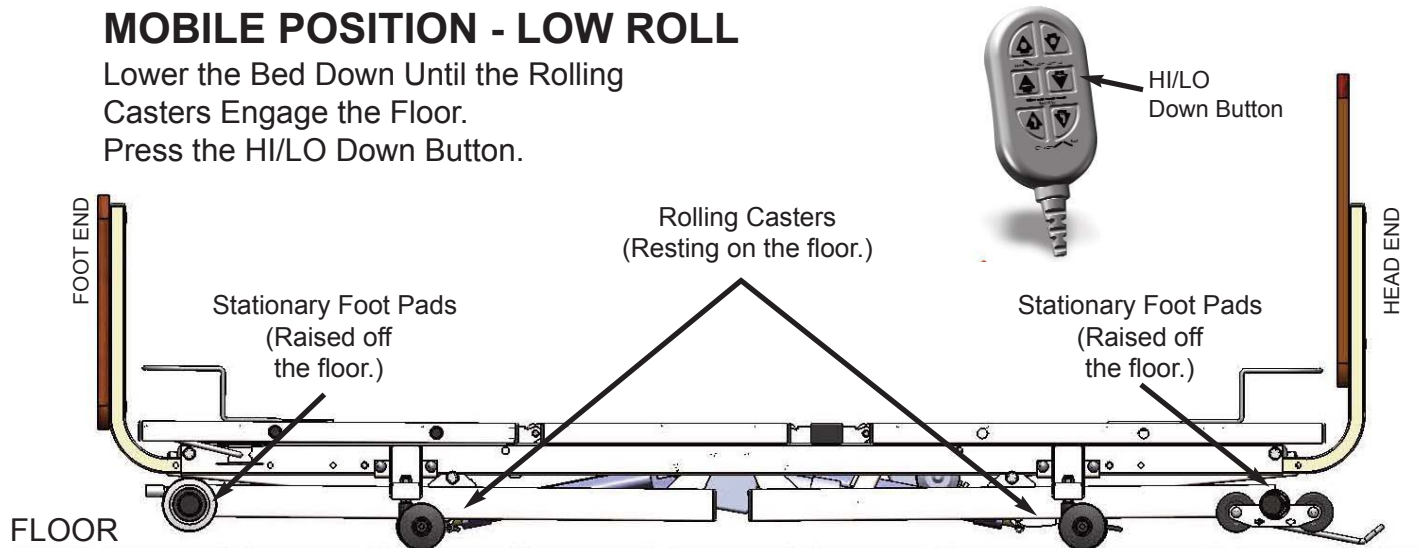



SECTION B


ASSIST DEVICES

Model ZG58174



OPTIONAL PIVOTING ASSIST BAR INSTALLATION AND OPERATION KIT ZA85400 - (Sample shows bar positioned on left side of bed)

1. At the head end of the bed, position the Pivoting Assist Bar (#2) bracket over the Head Deck (#1) frame and align the two Assist Bar bracket holes with the two frame rail holes on either side of the head deck.
2. From the outside, insert the Clevis Pins (#3) through the pivot bracket and deck frame holes.
3. From the inside, insert a Hair Pin Clip (#4) through each of the clevis pin small holes.

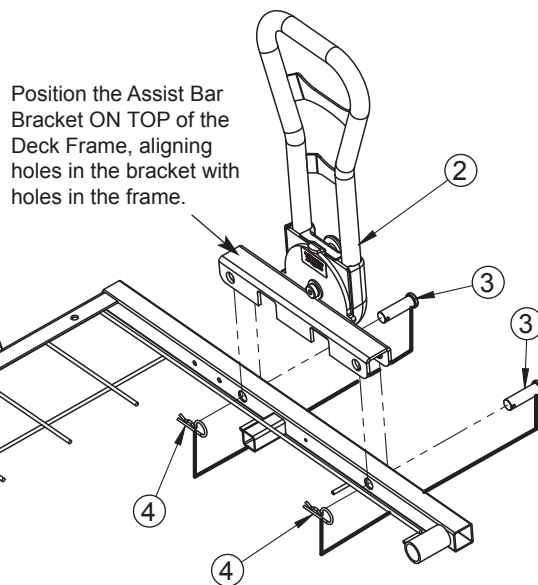


PLEASE ORDER KIT # ZA85400.

THE KIT INCLUDES ONE PIVOTING ASSIST BAR ASSEMBLY AND TWO LANYARD ASSEMBLIES WITH CLEVIS PINS AND HAIR PIN CLIPS. (A STATIONARY ASSIST BAR IS ALSO AVAILABLE AS AN OPTION - KIT # ZA85000)

THE PIVOTING AND FIXED ASSIST BARS CAN BE POSITIONED ON EITHER THE RIGHT OR LEFT SIDE OF THE HEAD DECK.

ASSIST BARS ARE NOT INTENDED FOR USE AT THE FOOT END OF THE BED.

PIVOTING ASSIST BAR OPERATION

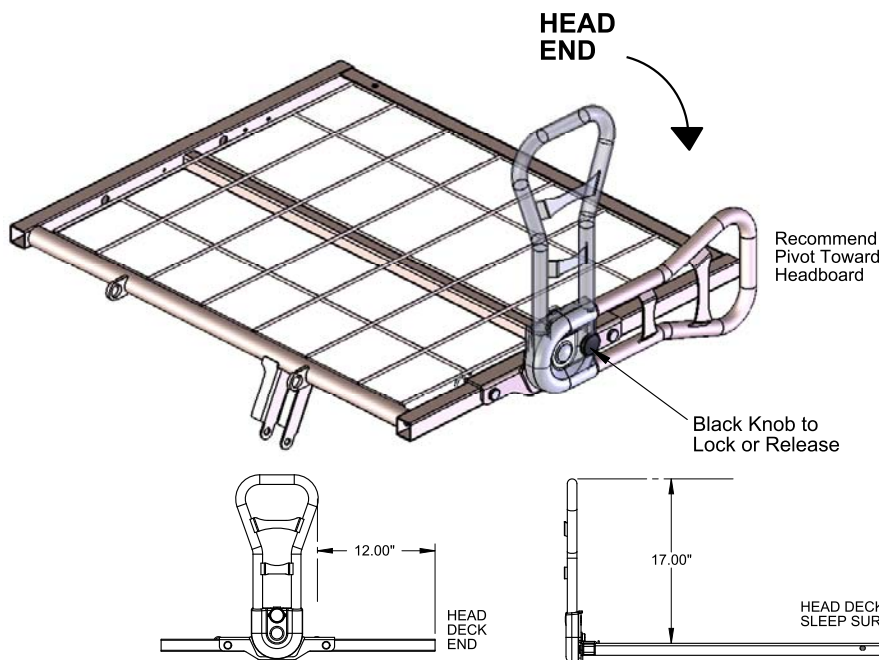

1. To release the pivoting assist bar from its vertical lock position, hold onto the top of the bar with one hand and slightly pull out the black knob on the outside of the pivot assembly with the other hand to release the locking mechanism. For ease of patient access, it is recommended that you pivot the rail toward the headboard until it stops and rests in place.
2. To return the assist bar to its upright locked position, grab the top of the assist bar with one hand and pivot the assembly upward until the black knob mechanism snaps into place, locking the assist bar vertically.



TO AVOID PINCHING FINGERS, ENSURE FINGERS ARE CLEAR OF SIDES OR UNDERSIDES OF THE PIVOTING ASSEMBLY WHEN POSITIONING THE ASSIST BAR UP OR DOWN.



TO AVOID PERSONAL INJURY, ENSURE THE PIVOTING ASSIST BAR IS ALWAYS IN THE FULL, VERTICAL, LOCKED POSITION WHENEVER RESIDENT/PATIENT IS LEFT UNATTENDED.




POWERED AIR MATTRESS SURFACES MAY POSE A RISK OF ENTRAPMENT. PRIOR TO USE, ENSURE THE THERAPEUTIC BENEFITS OUTWEIGH THE RISK OF ENTRAPMENT.



OPTIONAL PLASTIC HALF RAIL INSTALLATION AND OPERATION

KIT ZA84200 - (SAMPLE SHOWS RAIL POSITIONED ON RIGHT SIDE OF BED)



1. Position the Plastic Half Rail (#2) as shown, with the taller, straight edge toward the head end of the bed and arms angled toward the center of the bed.
2. From the outside, insert the two welded grooved pins on the rail arms into the two large holes in the side of the head deck.
3. From the inside, install the metal hair pin clip end of the Lanyard Assembly (#3) onto the exposed grooves at the end of the pins.
4. Position the other terminal end of the Lanyard Assemblies (#3) over the small hole on the inside of the deck frame. Secure with a small Phillips Screw (#4).

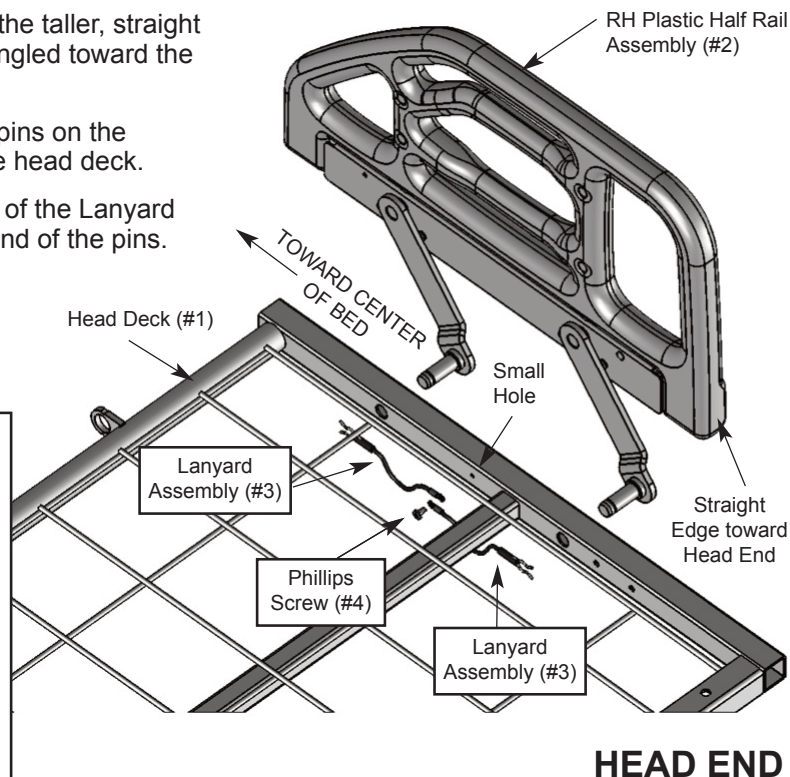

PLEASE ORDER PLASTIC RAIL KIT #ZA84200, WHICH INCLUDES A RIGHT AND LEFT HALF RAIL, FOUR LANYARD ASSEMBLIES, AND TWO PHILLIPS SCREWS (OR TWO OPTIONAL HEX HEAD SCREWS). ORDER BAG #999-0842-905 FOR JUST THE REPLACEMENT HARDWARE.

THE PLASTIC RAIL CAN BE INSTALLED ON EITHER THE HEAD OR FOOT END OF 80" BEDS. **IT SHOULD ONLY BE INSTALLED ON THE HEAD END OF 76" BEDS.**

PLASTIC HALF RAIL OPERATION

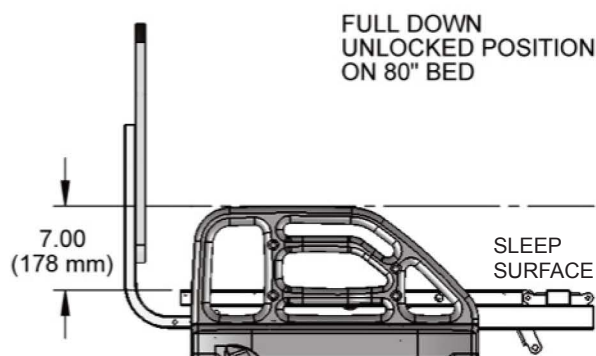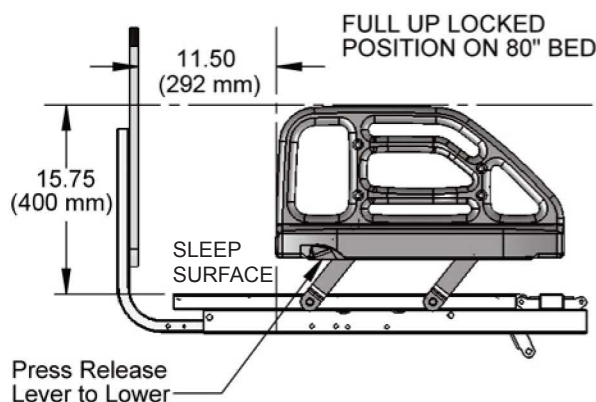

! TO AVOID PINCHING FINGERS, HOLD THE HALF RAIL FROM THE TOP WITH ONE HAND AND CAREFULLY DEPRESS THE BLACK RELEASE LEVER WITH THE OTHER. ENSURE THERE ARE NO OBJECTS IN THE WAY THAT MAY HINDER THE FULL MOVEMENT OF THE RAIL.


! TO AVOID PERSONAL INJURY, ENSURE THE HALF RAIL IS ALWAYS IN THE FULL UP LOCKED POSITION WHENEVER RESIDENT/PATIENT IS LEFT UNATTENDED.

! POWERED AIR MATTRESS SURFACES MAY POSE A RISK OF ENTRAPMENT. PRIOR TO USE, ENSURE THE THERAPEUTIC BENEFITS OUTWEIGH THE RISK OF ENTRAPMENT.



OPTIONAL TRAPEZE SUPPORT INSTALLATION

(THE TRAPEZE SUPPORT IS ATTACHED TO THE HEAD SECTION OF THE BED.)
PLEASE ORDER KIT NUMBER ZA79000

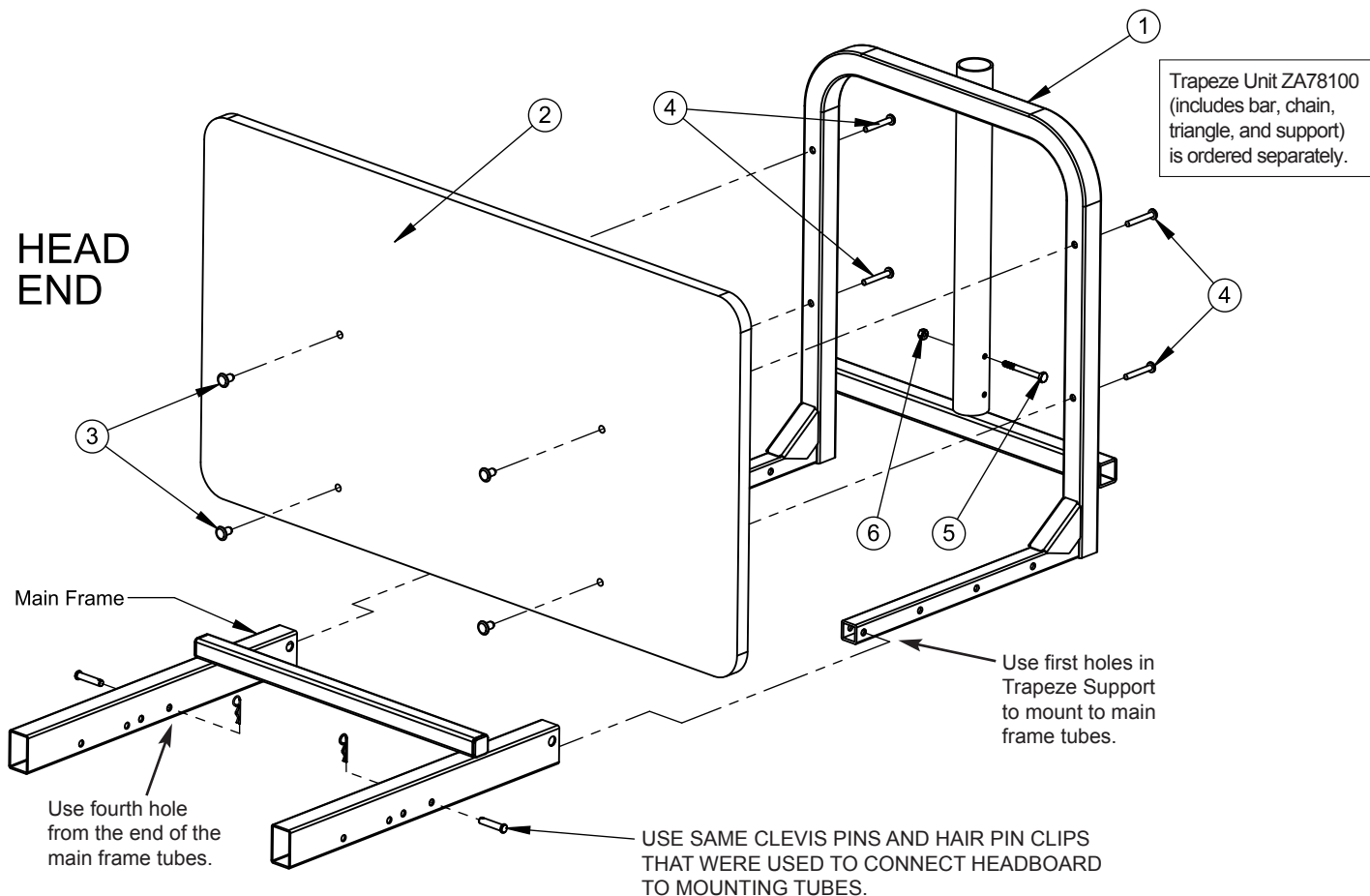

1. Begin by removing the headboard panel assembly (#2), if previously assembled to the bed, by pulling out the two Clevis Pins and two Hair Pin Clips that attach the main frame tubes to the headboard's mounting tubes. Set all pins aside for later use.
2. Detach the headboard panel from the mounting tubes by unscrewing the four 40mm long Socket Head Cap Screws that fasten the board to the tubes (these will be replaced with the longer Phillips Head Pan Screws from the trapeze adaptor hardware bag). Do not remove the four 1/4-20 inserts (#3) from the board. They will be used again to mount the board to the trapeze support.
3. Assemble the Trapeze Support (#1) to the main frame tubes using the two Clevis Pins and Hair Pin Clips you removed in Step 1. **The first holes in the Trapeze tubes must line up with the fourth hole from the end in the main frame tubes as shown; the main frame shown is for 80" deck.**
4. Insert the 1/4-20 x 2.50" Carriage Bolt (#5) into the vertical support as shown and fasten with a 1/4-20 Lock Nut (#6). Both parts should be in the hardware bag.
5. Assemble the headboard to the inside of the Trapeze Support so that the side of the headboard with the inserts is facing away from the Trapeze Support. From the outside, insert the (4) four 1/4-20 x 1.75" long Machine Screws (#4) from the hardware bag through the holes in the Trapeze Support, screwing them into the inserts in the headboard.



SECTION C

Mechanical and Electrical Information



ZENITH 5000 MECHANICS

NOTE: ALL DIMENSIONS ARE $\pm .25$ INCHES

- Overall Bed Length (with boards and wallsaver) . 82"/86"
- Overall Bed Width (with boards). 36.00"
- Maximum Height* 30.00"
- Minimum Height* 7.95"
- Maximum Head/Back Deck Angle 70°
- Maximum Knee/Foot Deck Angle 30°
- Maximum Safe Working Load 500 lb
WITH WEIGHT EVENLY DISTRIBUTED
- includes bedding, resident, support surface,
and all accessories
- Mass of bed (without assist devices or boards) =
204 lb

* Bed height calculated from floor surface to top of sleep deck.

ZENITH 5000 ELECTRICAL

- Power/Frequency 120 Volt/ 50/60 Hz
- Output Rating 12/33V IPX4
- Overall Movement Draw 4.50 Amps
- Power (Electrical) Cord #18 AWG 3 Conductor
. Type SJT
- Mode of Operations 10% Max. Duty Cycle
. (2 minutes on/18 minutes off)
- Internal Battery Backup 9V Battery

Pack and Charger can be purchased separately as an accessory.

TYPICAL ZENITH 5000 BED IDENTIFICATION LABELS with Grounded Electrical Cable

GF HEALTH PRODUCTS, INC. ATLANTA, GA 30360		MODEL NO 805-0000-000000									
DUTY CYCLE - 2 min Continuous/18 min MODE OF OPERATION - Intermittent		MFG. DATE 4/01/13									
= 500 lbs SAFE WORKING LOAD	IPX4	<table border="1"><tr><td>VOLT</td><td>HZ</td><td>AMP</td><td>TYPE</td></tr><tr><td>110-120</td><td>50/60</td><td>4.5</td><td>AC</td></tr></table>	VOLT	HZ	AMP	TYPE	110-120	50/60	4.5	AC	SERIAL NO 000000000000
VOLT	HZ	AMP	TYPE								
110-120	50/60	4.5	AC								

Bed labels are an important part of identifying the bed's make and model when ordering replacement parts. The Serial Number is essential if you are claiming parts or service under warranty. These helpful labels can be located on the main frame rails, immediately below the sleep decks on either side of the bed.


Please have this IMPORTANT information ready when calling our Customer Service or Technical Support staff at 770-368-4700; it will allow us to better assist you and quickly answer your questions and concerns.

ZG58174		ZENITH 805 SERIES PATENT PENDING ZENITH 5000		SERIAL ZG5
MODEL NO. 805-0000-000000		MFG DATE 4/01/13		
ZG58174				000000000000




ZENITH 5000 SERIES ELECTRICAL COMPONENTS


Plug-in for
Optional
Battery
Backup

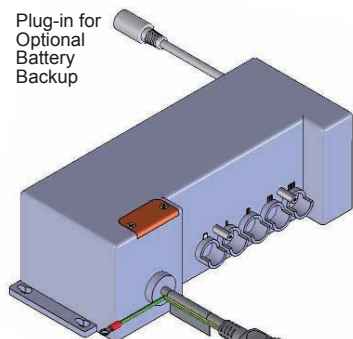


Control/Junction Box
Part # 999-0844-300SP
Quantity = 1
Box and cable color = White

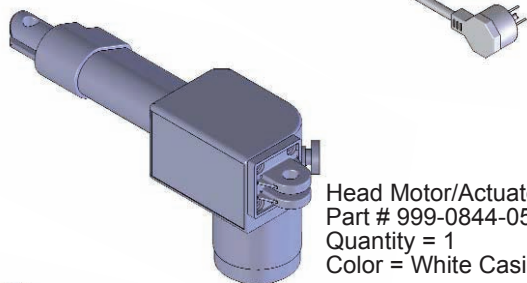


Head Motor/Actuator
Part # 999-0844-052SP
Quantity = 1
Color = White Casing

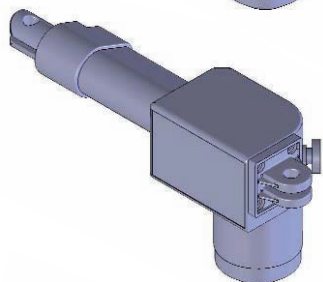


Foot Motor/Actuator
Part # 999-0844-053SP
Quantity = 1
Color = White Casing

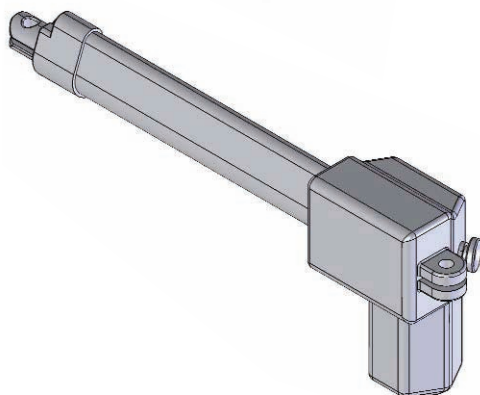

HI/LO Motor/Actuator
Part # 999-0843-251SP
Quantity = 2
Color = White Casing

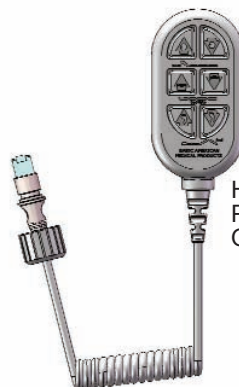


Hand Control Pendant
Part # 999-0775-305SP
Quantity = 1

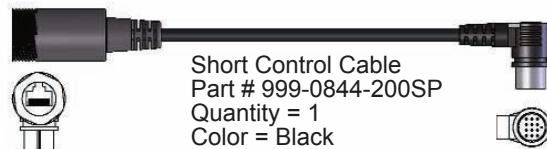

Short Control Cable
Part # 999-0844-200SP
Quantity = 1
Color = Black

Phone Jack end for
Hand Control Pendant/
Staff Control

9 Pin DIN Plug
for Control Box

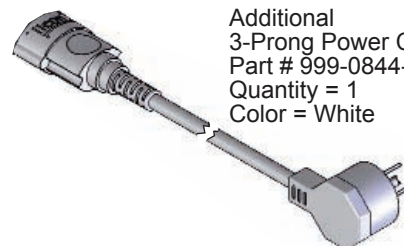

Additional
3-Prong Power Cord
Part # 999-0844-208SP
Quantity = 1
Color = White



Pendant Holster
Part # 999-0791-000SP
Quantity = 1
(included with your bed)

*Clipped on Pendant
when bed first shipped.*




DO NOT use unauthorized parts, accessories, or adaptors other than those specified/authorized by GF Health Products, Inc. DO NOT open assemblies such as the Actuators, Hand Control, or Control Box. If unauthorized personnel perform work on these components, the manufacturer's warranty becomes void. NEVER operate the bed if a Power Cord or Plug is damaged or not working properly. Contact qualified Service Personnel for examination and repair.

ALWAYS UNPLUG THE POWER CORD WHEN PERFORMING ANY MAINTENANCE ON THE BED.




ZENITH 5000 SERIES ELECTRICAL CABLING

- ❶ POWER CORD
- ❷ HAND CONTROL PENDANT CABLE
- ❸ HEAD MOTOR CABLE
- ❹ FOOT MOTOR CABLE
- ❺ HI/LO CABLE (FOOT)
- ❻ HI/LO CABLE (HEAD)
- ❼ BLACK CONTROL CABLE
- ❽ GROUND WIRE

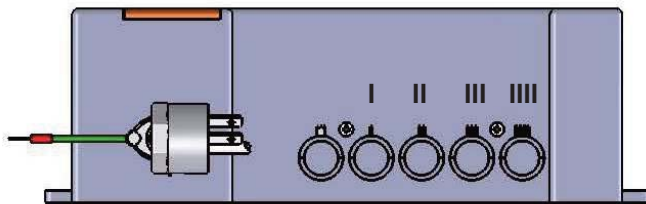


Control Cable (crown)
Head - Port I
Foot - Port II
HI/LO (Ft) - Port III
HI/LO (Hd) - Port IIII

Black Control Cable
White Head Motor Cable
White Foot Motor Cable
White HI/LO (Foot) Cable
White HI/LO (Head) Cable

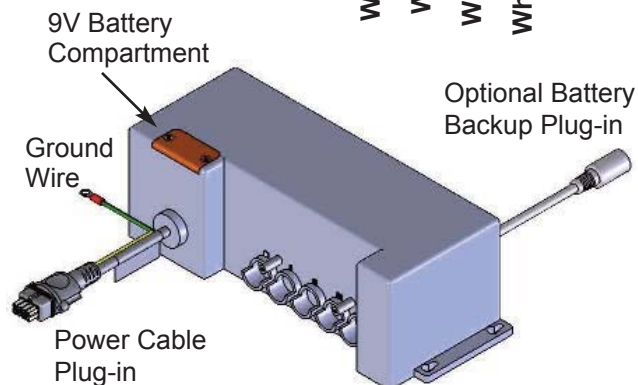

NOTE: 9V Battery included in top of box is good for only one use in event of a power outage. An optional portable Ilcon Battery Backup unit is recommended for longer periods where power is not available. Contact your sales representative for more information.

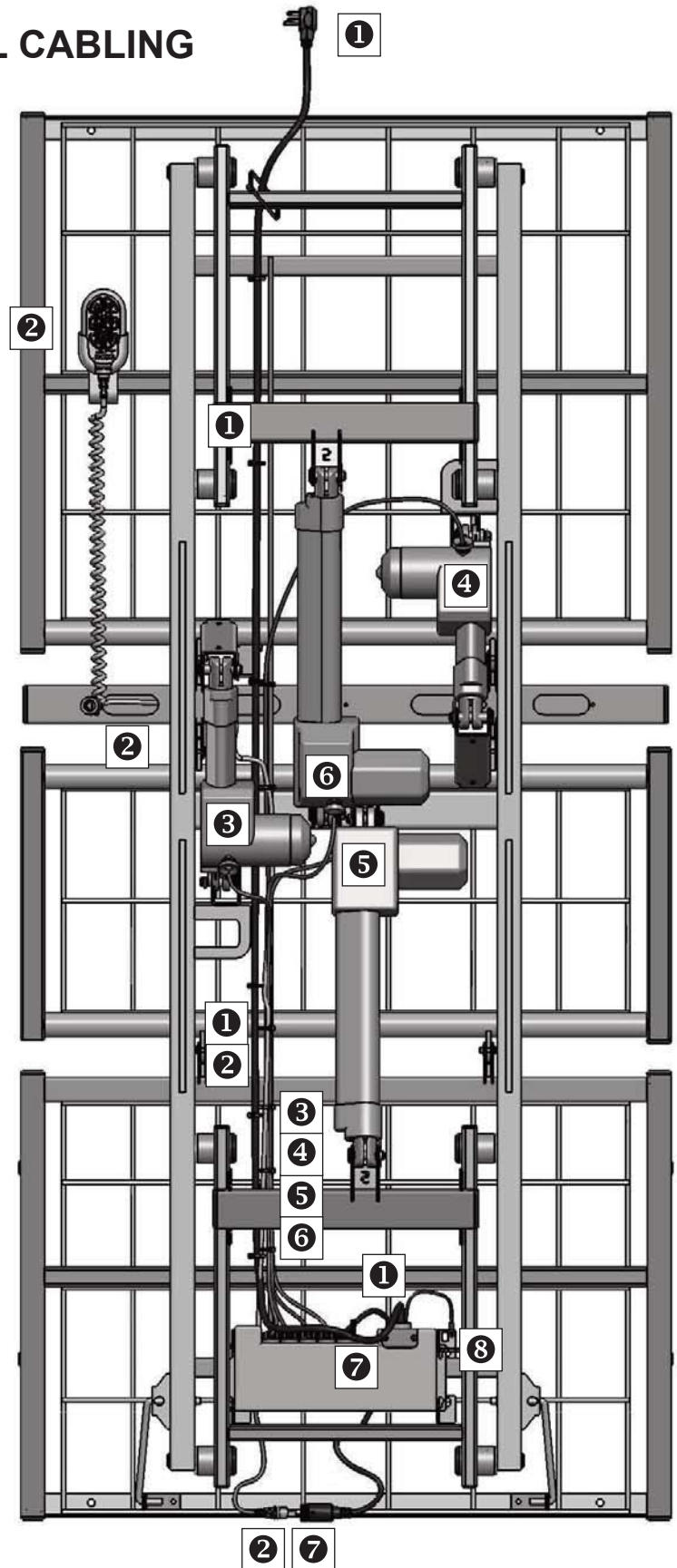



SECTION D

SERVICE/REPLACEMENT PARTS



SERVICE/REPLACEMENT PART RECORD (Page 1)

DECKING					
Part Number	Description		QTY	Order Date	Price
999-0844-920SP	Head/Back Deck and Hardware Pack		1 Deck		
999-0844-930SP	OR - Foot Deck and Hardware Pack		1 Deck		
999-0844-940SP	OR - Knee Deck and Hardware Pack		1 Deck		
	Deck Hardware Pack - INCLUDES . . .		1 Pkg		
Separate Hardware Pack 999-0844-990	4	Clevis Pins			
	5	5/16" Retaining Rings (Deck Brackets)			
	2	12mm Retaining Ring (Cntrl Arms to Decks)			
	1	Installation Instruction Sheet			
MOTORS (ACTUATORS)					
Part Number	Description		QTY	Order Date	Price
999-0843-251SP	HI/LO Motor (Cable attached) and Hardware Pack		1 Motor		
999-0844-052SP	OR - Head Motor (Cable attached) and Hardware Pack		1 Motor		
999-0844-053SP	OR - Foot Motor (Cable attached) and Hardware Pack		1 Motor		
	Motor Hardware Pack- INCLUDES . . .		1 Pkg		
Separate Hardware Pack 999-0844-991	2	12mm Retaining Rings			
	2	Hair Pin Clips			
	2	Snap Rings			
	8	Standard Cable Ties			
	1	Installation Instruction Sheet			
CONTROL BOX AND POWER CABLES					
Part Number	Description		QTY	Order Date	Price
999-0844-300SP	Control Box Service Pack (includes control cable)		1 Kit		
	Service Pack - INCLUDES . . .				
Separate Hardware Pack 999-0844-992	1	Adaptor Control Cable 999-0844-200			
	1	Ground Screw			
	8	Standard Cable Ties			
	1	Installation Instruction Sheet			
999-0844-300B	Control Box (includes 9V battery)		1 Box		
999-0844-208	Detachable Ilcon 3-Prong Power Cable		1 Cable		




SERVICE/REPLACEMENT PART RECORD (Page 2)

HAND CONTROL PENDANT					
Part Number	Description		QTY	Order Date	Price
999-0775-305SP	Hand Control Pendant (Standard Model)		1 Pendant		
999-0791-000SP	Pendant Holster (Standard for all pendants)		1 Holster		
SHORT BLACK CONTROL CABLE					
Part Number	Description		QTY	Order Date	Price
999-0844-200SP	Black Control Cable Kit		1 Pack		
	Control Cable Service Pack - INCLUDES . . .		1 Pkg		
No Separate Hardware Pack	2	Standard Cable Ties			
	1	Installation Instruction Sheet			
CAPS AND PLUGS					
Part Number	Description		QTY	Order Date	Price
NOTE: CAPS AND PLUGS CAN ONLY BE PURCHASED IN SETS OF 12					
100-4200-004PK	1.25" x 1.25" Square Caps Pack		12/pkg		
100-4700-015PK	1.00" x 1.00" Square Caps Pack (arm ends)		12/pkg		
100-4700-017PK	1.00" x 2.00" Rectangular Caps Pack		12/pkg		
100-4700-021PK	.75" x 1.50" Rectangular Caps Pack		12/pkg		
100-4700-018PK	1.25" x 2.00" Rectangular Caps Pack		12/pkg		
100-4715-011PK	1.50" Round End Caps Pack (cover arm bearings)		12/pkg		
999-0775-001PK	Half Moon End Caps Pack (caster bases)		12/pkg		
100-4762-002PK	5/8" Round Plugs Pack (both sides of foot deck)		12/pkg		
100-4738-005PK	3/8" Round Plugs (on main frame rails - foot end)		12/pkg		
FOR GENERAL CAP/PLUG DIAGRAMS PLEASE SEE PAGES 30 AND 31					



SERVICE/REPLACEMENT PART DIAGRAM - ELECTRONICS

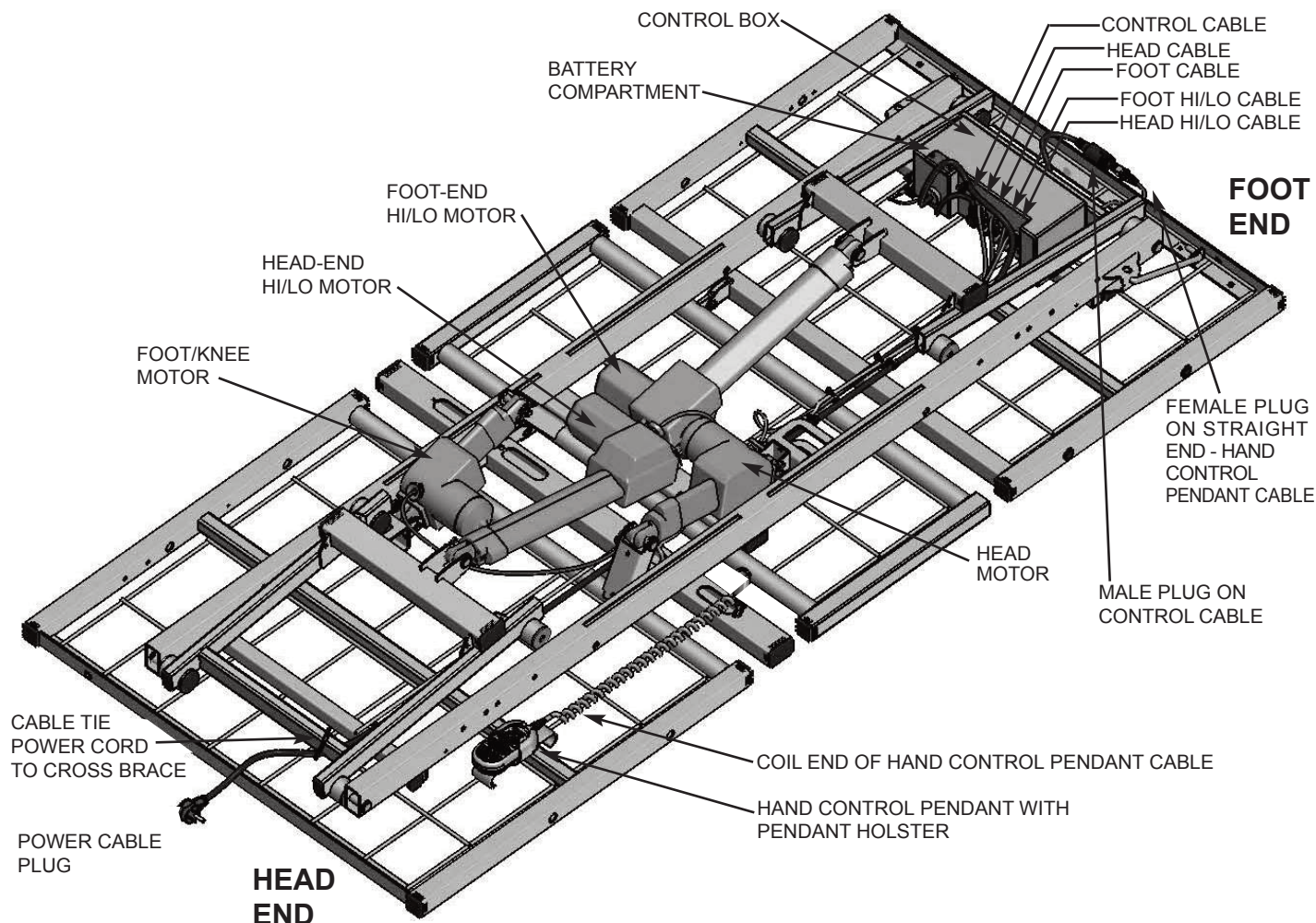

1. **TO ORDER A REPLACEMENT HI/LO MOTOR USE SERVICE PACK 999-0843-251SP** (Includes motor, mounting hardware, and installation instruction sheet).

TO ORDER A REPLACEMENT CONTROL BOX USE PART NO. 999-0844-300SP (Includes control box, detachable power cord, control cable, ground wire screw, cable ties, and installation instruction sheet).

2. **TO ORDER A REPLACEMENT HEAD MOTOR USE SERVICE PACK 999-0844-052SP** (Includes motor, mounting hardware, and installation instruction sheet).

TO ORDER A REPLACEMENT HAND CONTROL PENDANT USE PART NO. 999-0775-305SP. TO ORDER A NEW PENDANT HOLSTER USE PART NO. 999-0791-000SP.


3. **TO ORDER A REPLACEMENT FOOT MOTOR USE SERVICE PACK 999-0844-053SP** (Includes motor, mounting hardware, and installation instruction sheet).

TO ORDER A REPLACEMENT CONTROL CABLE USE PART NO. 999-0844-200SP (Includes cable, two (2) cable ties, and installation instruction sheet).

WHEN ORDERING REPLACEMENT PARTS WITH CUSTOMER SERVICE,
PLEASE HAVE THE BED SERIAL NUMBER AVAILABLE
TO CONFIRM WHETHER THE PART IS COVERED UNDER WARRANTY.
(SEE PAGE 20 FOR LOCATION OF SERIAL NUMBER ID LABEL.)



REPLACEMENT OF 9V BATTERY AND OPTIONAL BATTERY BACKUP CHARGER AND BATTERY PACK

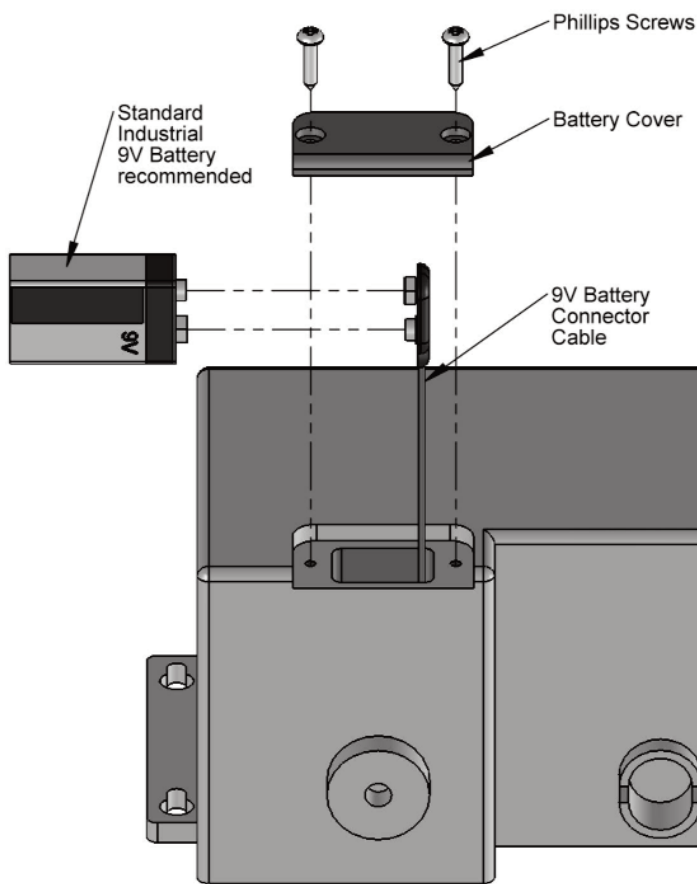

REPLACING YOUR INTERNAL 9V BATTERY:


1. Remove battery cover on the underside of your control box by unscrewing the two Phillips screws as shown.
2. Remove the existing battery connected to the Connector Cable from the battery slot.
3. Unplug the existing battery and discard. Plug in the new battery to the Connector Cable and reinsert into slot. Replace battery cover.

NOTE: THE 9V BATTERY INCLUDED WITH YOUR CONTROL BOX IS ONLY GOOD FOR ONE CHARGE - LOWERING THE BED ONCE IN THE EVENT OF A POWER OUTAGE. YOU WILL NEED TO REPLACE AFTER EACH SINGLE USAGE. A STRONGER BATTERY PACK IS AVAILABLE AS A SEPARATE PURCHASE. PLEASE ORDER WITH BACKUP CHARGER.

Battery Charger
Adaptor Cable

Plugs directly
into optional
ILCOCHARGE
Backup Battery
and Battery
Pack purchased
separately.

To plug in battery charger,
first remove the protective
cap with small screwdriver.

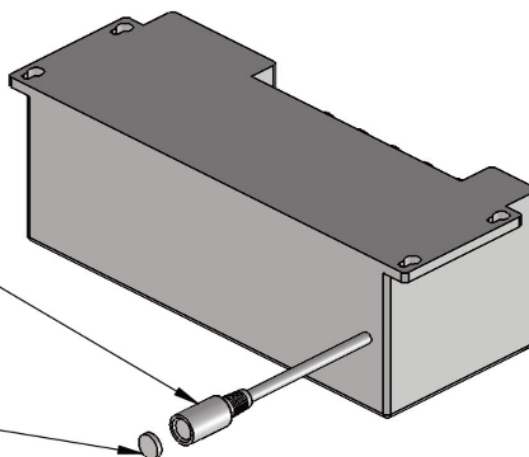




SECTION E

DECK/MECHANICAL SERVICE PARTS


For All Zenith 5000 Bed Models




SERVICE/REPLACEMENT PARTS DIAGRAM - DECKING

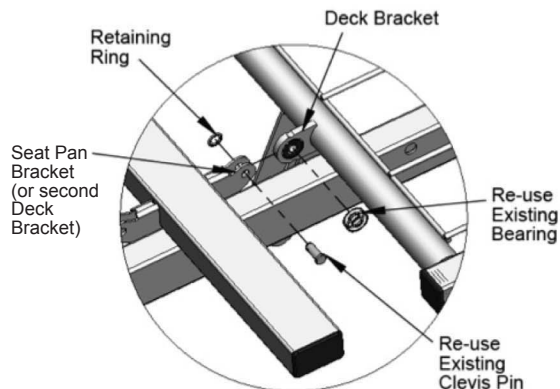


DETAIL OF DECK ATTACHMENT
(Similar for all decks)

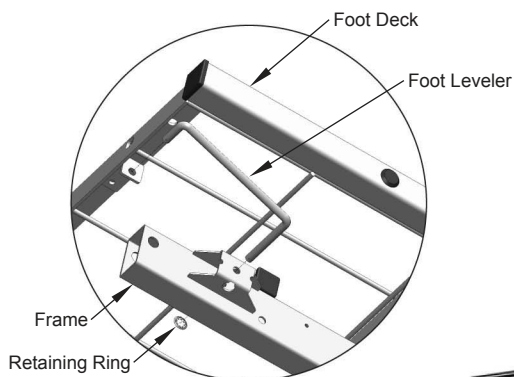


DETAIL OF FOOT DECK TO FRAME

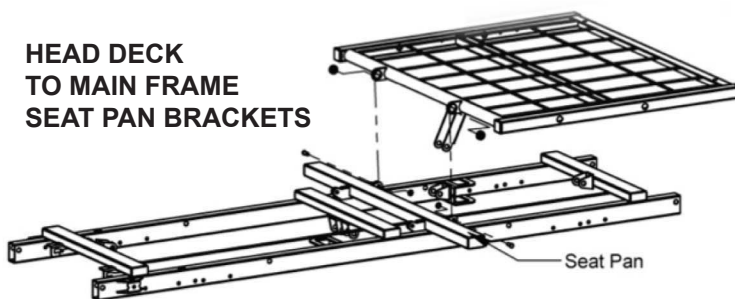

HEAD DECK TO MAIN FRAME SEAT PAN BRACKETS

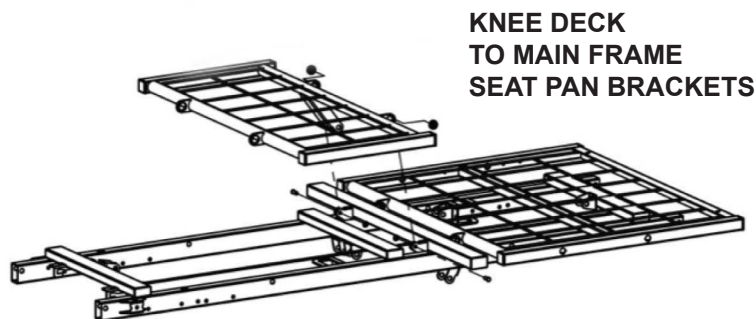

KNEE DECK TO MAIN FRAME SEAT PAN BRACKETS

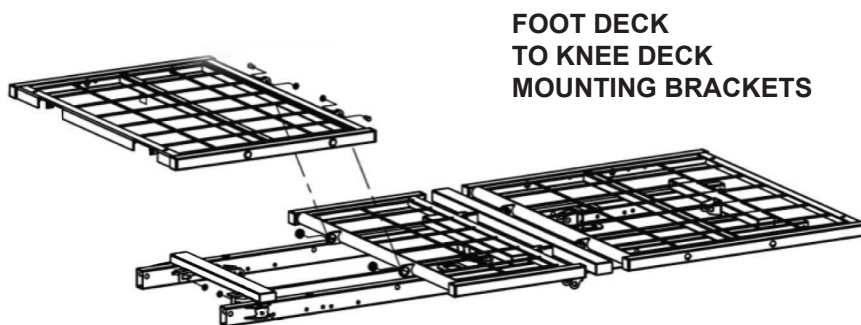

FOOT DECK TO KNEE DECK MOUNTING BRACKETS

1. **TO ORDER A REPLACEMENT HEAD DECK USE SERVICE PACK 999-0844-920SP** (includes deck, mounting hardware, and installation instruction sheet).


2. **TO ORDER A REPLACEMENT KNEE DECK USE SERVICE PACK 999-0844-940SP** (includes deck, mounting hardware, and installation instruction sheet).

3. **TO ORDER A REPLACEMENT FOOT DECK USE SERVICE PACK 999-0844-930SP** (includes deck, mounting hardware, and installation instruction sheet).

WHEN ORDERING REPLACEMENT PARTS WITH CUSTOMER SERVICE,
PLEASE HAVE THE BED SERIAL NUMBER AVAILABLE
TO CONFIRM WHETHER THE PART IS COVERED UNDER WARRANTY.
(SEE PAGE 20 FOR LOCATION OF SERIAL NUMBER ID LABEL.)




SERVICE/REPLACEMENT PARTS DIAGRAM - MAIN FRAME CAPPING

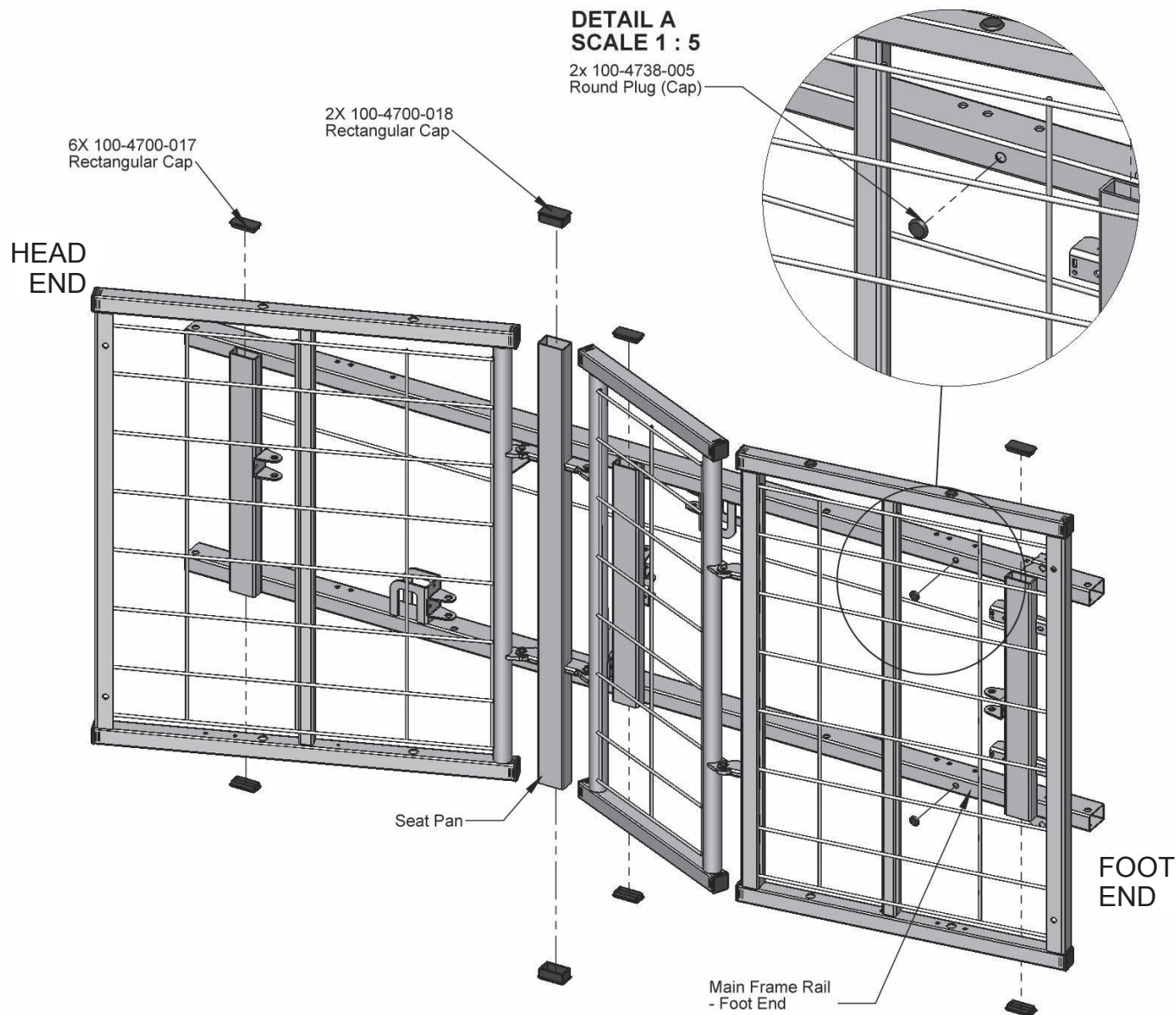



CAPS AND PLUGS ARE NOT COVERED UNDER WARRANTY
(PACKS OF 12 CAN BE ORDERED IN THE EVENT YOU DAMAGE OR LOSE
ANY BED CAPS OR PLUGS - SEE BOTTOM OF PAGE 25 FOR ORDER NUMBERS)



SERVICE/REPLACEMENT PARTS DIAGRAM - HEAD, KNEE, AND FOOT DECK CAPPING

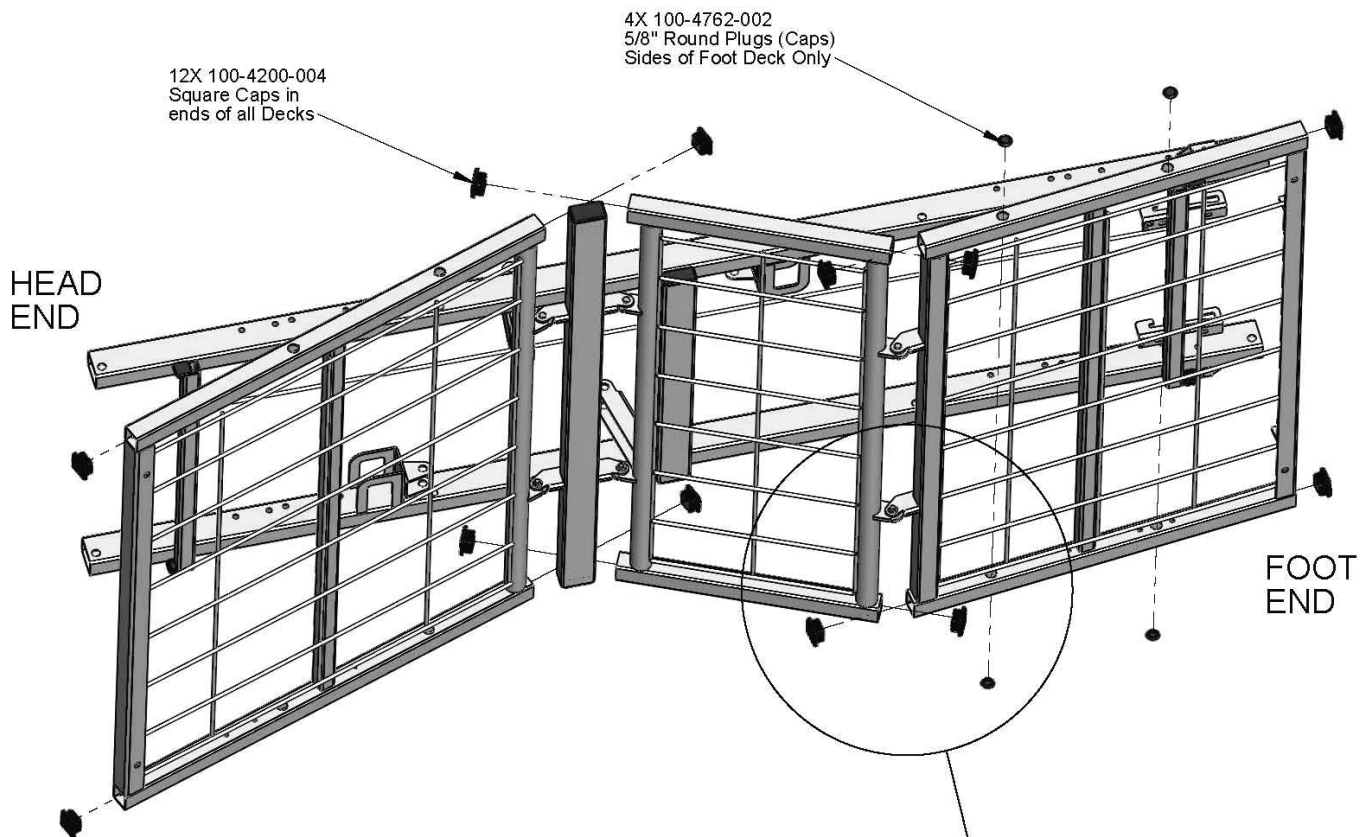

CAPPING ON ARMS AND LEGS

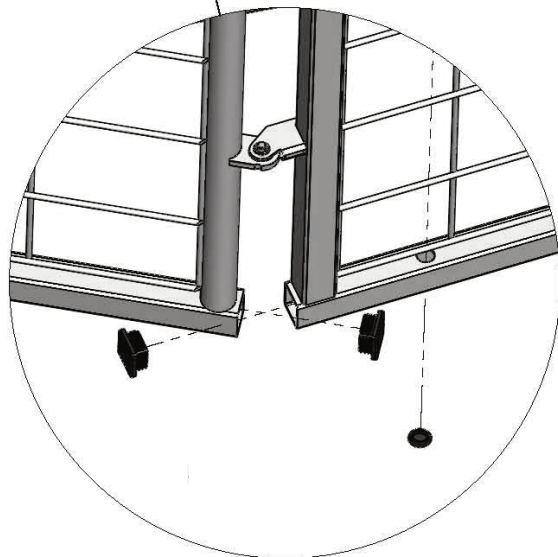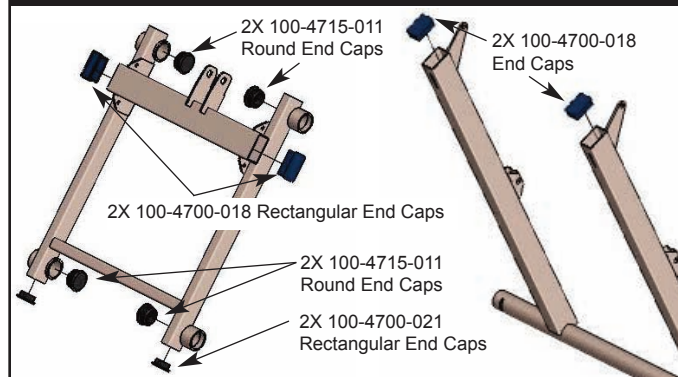

CAPS AND PLUGS ARE NOT COVERED UNDER WARRANTY
(PACKS OF 12 CAN BE ORDERED IN THE EVENT YOU DAMAGE OR LOSE
ANY BED CAPS OR PLUGS - SEE BOTTOM OF PAGE 25 FOR ORDER NUMBERS)



SECTION F

TROUBLESHOOTING

All Zenith 5000 Bed Models



ZENITH 5000 ELECTRONIC TROUBLESHOOTING QUICK REFERENCE

To troubleshoot the Zenith 5000 Series Bed, follow the steps below in the order shown to determine the cause of the problem and appropriate component replacement:

HEAD, FOOT, OR HI/LO MOTOR DOES NOT GO UP AND/OR DOWN:

1. Verify that there is power in the wall outlet.
2. Inspect all cables for pinched or damaged insulation. Check all motor and power cable connections.
3. Inspect motor condition for cracked housing, scratches, bent shaft, etc.
4. If you have another known good hand control pendant, switch out the pendant and retest all functions. If the problem continues, continue with the Troubleshooting steps below.

HEAD OR FOOT MOTOR DOES NOT FUNCTION:

1. Unplug bed power cord. Switch head and foot motor cable plug positions at control box.
2. Plug in bed power cord and retest. If same function does not work, replace that motor.
3. If opposite motor now doesn't work, replace control box.



HEAD AND FOOT DECK FUNCTION PROPERLY, BUT ONE OR BOTH HI/LO MOTORS DOES NOT:

1. This may occur if for some reason the system has lost the known position of one or both of the HI/LO motors. If so, it will need to be resynchronized to correct it. When this happens, one or both HI/LO motors may run in very small amounts if at all. To correct for lost position:
2. Unplug bed power cord from wall outlet for 15 seconds and then plug it back in.
3. ***Press Hand Control Pendant buttons only in this specific sequence:***
 - a. Press and hold the HI/LO DOWN button for 15 seconds, then release it for 2 seconds. This will put the electronics into **resynchronize** mode.
 - b. Press and hold the HI/LO DOWN button for 5 seconds; it will likely lower only a small amount.
 - c. Release the HI/LO DOWN button for 5 seconds.
 - d. Repeat this step 3 b-c sequence until both HI/LO motors have lowered the bed completely to the lowest position.


Note that pressing and releasing the buttons too quickly will cause malfunction, and if any other buttons are pressed, the system will leave resynchronize mode and you must start the sequence over.
4. Once you believe the bed is all the way down:
 - a. Wait 15 seconds.
 - b. Press and hold the HI/LO DOWN button for 15 seconds, then release it for 2 seconds.
 - c. Press and hold the HI/LO DOWN button for 5 seconds, then release it.

This is to ensure the bed is truly all the way down. If either motor moves, repeat step 4 until no further movement occurs. If no movement occurs, the system should now be back in a known position and the HI/LO UP function should work properly.

5. If HI/LO problem still continues:
 - a. Unplug bed power cord.
 - b. Unplug the two HI/LO Motor Cables at the control box and replug them into the control box opposite their original locations (see control box diagram below for correct plug locations).
 - c. Plug in bed power cord and retest.
6. If the same HI/LO motor moves and the other still does not, replace the HI/LO motor that does not move.

If only the opposite HI/LO motor moves, replace the control box. In either case, the HI/LO function will need to be resynchronized when the replacement component has been installed.

If the problem continues, please contact Technical Support at 770-368-4700.

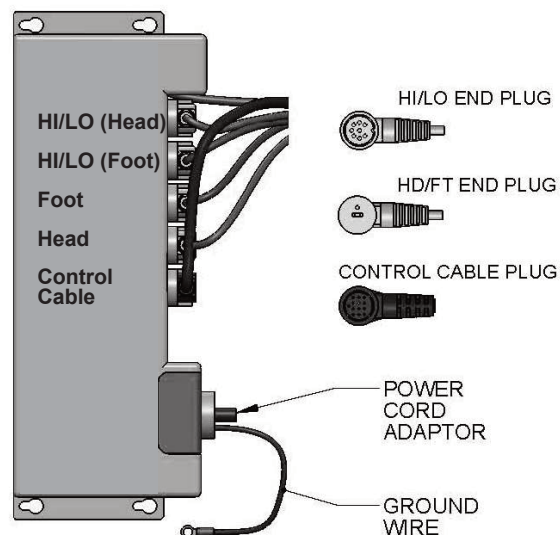



LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable.

The warranted components and time period are set forth below:

Main Frame and welds:	fifteen years
Control Box and Actuator Motors:	five years
Hand Control Pendant and Cabling	three years
Headboard and Footboard	one year
All other durable components not listed above:	six months

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.



ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within thirty (30) days of the invoice date.

GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
Phone 770-368-4700
Fax 770-368-2386
www.grahamfield.com

**If you have questions regarding a bed's warranty, please contact
Basic American Medical Products at 770-368-4700.**



NOTES:

[illegible]



NOTES:

[illegible]



NOTES:

[illegible]



GRAHAM-FIELD

www.grahamfield.com

GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360

Customer Service: phone 770-368-4700, fax 770-368-2386

© 2011 GF Health Products, Inc., All Rights Reserved

Basic American Medical Products and Zenith are
registered trademarks of GF Health Products, Inc.