

CONTENTS


SIDEFRAME AND CROSSBRACE	3
ARM, DETACHABLE	4
UPHOLSTERY	4
REAR WHEEL	5
CASTER	5
FRONT RIGGING, DETACHABLE	6
ACCESSORIES	

TO ORDER

Tel: 1-770-368-4700 · Fax: 1-770-368-2386

Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. <u>Terms and Conditions</u> for further information.

SIDEFRAME AND CROSSBRACE

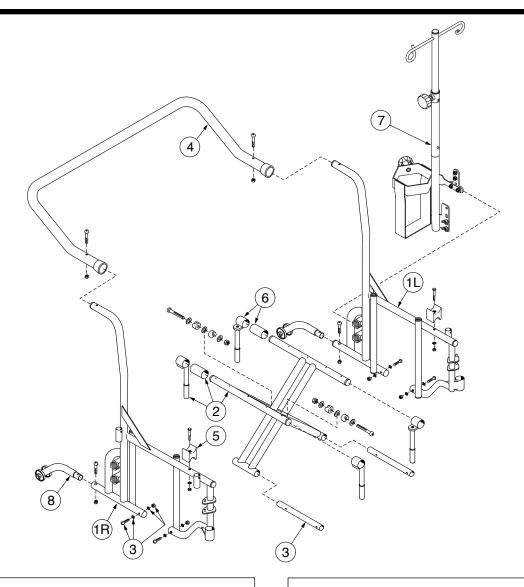

1	Sideframe.	all	widthe
	Siderrame.	all	widins

Right	90763601
Loft	00763603

2 Crossbrace, pair Includes slides, center bolt and nut

20" W	 907634C1
22" W	 907634C2
24" W	 907634C3


- 3 Bottom inner rail kit Includes bottom inner rail (pair), screw and nut that hold bottom inner rail (4 each), and sleeve (pair) ... 90763503
- 4 Push handle Includes hardware

20" W	907636B1
22" W	907636B2
24" W	907636B3

- 6 Seat slide and seat rail sleeve (2 each)90763404
- 7 O2-IV holder * B10320F
- 8 Anti-tippers, pair (Includes hardware) 90763650
- * Note: O₂-IV Holder, black-painted, is standard and included on Traveler HTC models with serial numbers after HYT039WC.


If you wish to order the O₂-IV Holder individually, please order B10320F, chrome-plated O₂-IV holder (black-painted O₂-IV holder is not available separately from wheelchair).



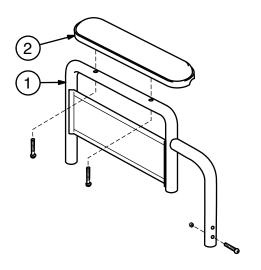

- 2 Arm pad, desk length, pair Includes two black arm pads and mounting screws 90763022

UPHOLSTERY

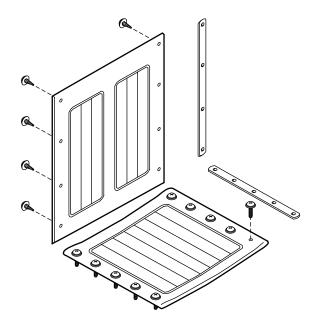


Back upholstery, black leatherette, with built-in chart holder Includes mounting strips and screws

20" W	907636A1
22" W	907636A2
24" W	907637A3


Seat upholstery, black leatherette Includes mounting strips and screws

20" W x 18" D	907634B1
22" W x 18" D	907634B2
24" W x 18" D	907634B3

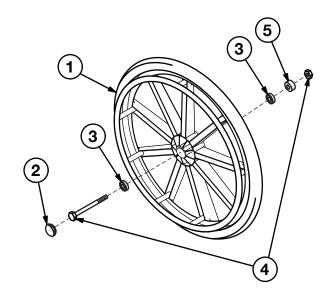


1 Rear wheel with handrim, 24" x 1", polyurethane tire Includes hub cap and bearing;
does not include axle or nut 90763450
2 Hub cap (10 each) 90763451
3 Wheel bearing (10 each) 90763452
4 Rear axle and nut (10 each) 90763453
5 Spacer (10 each) 90763454

CASTER

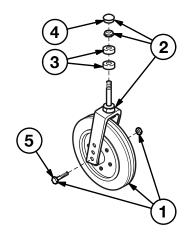


Complete caster (fork and wheel), 8" x 2" solid urethane tire 90763440
1 Caster wheel, axle and nut 90763441
2 Caster fork, stem nut and dust cap 90763042
3 Caster stem bearing (10 each) 90763043
4 Caster dust cap (10 each) 90763044
5 Caster axle and nut (10 each) 90763045


FRONT RIGGING, DETACHABLE

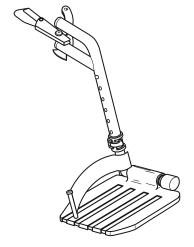

Aluminum footplate (pair)
(Fits all front rigging) 90763433
Anti-Theft nut (set of 4)
(Fits all front rigging) 90763630

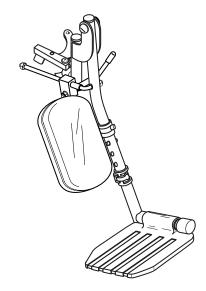

Elevating legrest panel kit (Fits all elevating legrests) Includes 2 black padded panels and hardware......90763032

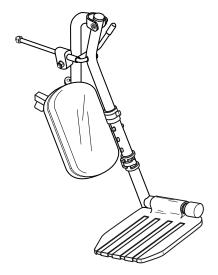

High-pivot elevating legrest, pair, all widths Includes right and left hanger with release, aluminum footplate with extension, and legrest panel (does not include heel loop)90763435

Traveler HTC accessories (not shown)

1 Heel loop, pair 90763064
2 Attendant wheel lock, pair (Includes hardware) 90763640
3 Rubber tip, wheel lock (10 each) 90763063
4 Positioning belt (metal buckle) 90133125
5 O ₂ cylinder holder B10325
6 O ₂ -IV holder * B10320F

* Note: O₂-IV Holder, black-painted, is standard and included on Traveler HTC models with serial numbers after HYT039WC.

If you wish to order the O₂-IV Holder individually, please order B10320F, chrome-plated O₂-IV holder (black-painted O₂-IV holder is not available separately from wheelchair).



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.

www.grahamfield.com









© 2012, GF Health Products, Inc. All Rights Reserved.

E&J, Everest & Jennings, Graham-Field, and Traveler are trademarks of GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140

GF Health Products, Inc. is an ISO 13485 Certified Company.