


Tools required: Phillips screwdriver

The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Bath Seat Basket. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the basket is not properly installed, personal injury and damage to the basket could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION (REFER TO PICTURES)

Components

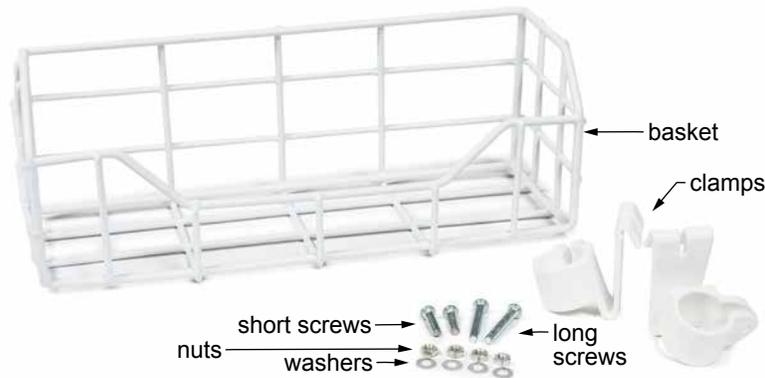

Basket Components


Components, shown above, are listed below.

COMPONENT	QUANTITY
Basket	1
Basket mounting clamp	2
Washer	4
Short screw (to mount clamps above joints)	4
Long screw (to mount clamps below joints)	4
Nut	4

The basket is shown in these instructions mounted **above the leg joint using short screws.**

Mounting the Basket

1. Mount one clamp on each front leg **at the same height** with hooks facing inward and clamps facing rear as shown at right in **Install Clamps**.
2. Install nuts in clamp recesses.
3. Install washers on appropriate screws: use short screws if you are mounting the clamp above the leg joint, as shown in these instructions, or long screws if you are mounting the clamp below the leg joint.
4. Insert screws through clamps into nuts as shown at right in **Fasten Clamp**.
5. Use Phillips Screwdriver to tighten as shown at right in **Fasten Clamp**. **DO NOT OVERTIGHTEN.**
6. Mount basket on clamp hooks as shown at right in **Basket Mounted on Clamp Hooks**. Ensure basket is securely mounted on both hooks as shown before use.

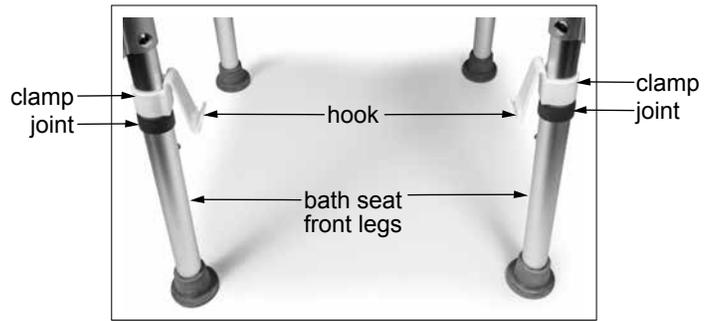

Install Clamps



Fasten Clamp
(Shown from Rear)



Basket Mounted
on Clamp Hooks

The basket is shown installed on bath seat (above leg joints) at right.

OPERATION

- ⚠ **WARNING:** Before each use, ensure all basket-mounting hardware is securely attached and basket is firmly installed on clamp hooks.
- ⚠ **WARNING:** The basket has a maximum weight capacity of 5 lb. Do not exceed this.
- ⚠ **WARNING:** Do not lean too far forward or sideward when accessing the basket; you could become unbalanced and serious personal injury could occur.

CLEANING AND MAINTENANCE

Cleaning

To clean the basket, use a clean, soft cloth dampened with a mild soap and water solution, rinse with clear water, then dry with a soft cloth.

- ▲ **NOTICE:** DO NOT use solvents, abrasive cleaners, or scouring pads to clean this product.

Maintenance

Check basket and mounting hardware periodically for corrosion and wear, and replace if found.



LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for one (1) year. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com



© 2017, GF Health Products, Inc. All Rights Reserved.
Graham-Field and Lumex are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.
Manufactured for GF Health Products, Inc.
Made in China