LUMEX®

by graham-field

Everyday Single-Release Folding Walker

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graham
field

QTY. 1 EA

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. Each individual should **ALWAYS** consult with their physician or other health care professional to determine proper adjustment and usage.
2. Before use, be sure that the walker is in its open and locked position.
3. All legs **MUST** be in contact with the floor at all times during use. This will ensure the walker is properly balanced.
4. This product has a maximum user weight limitation of 300 lbs - **EVENLY DISTRIBUTED. ALWAYS** observe the user weight limitation on the hangtag.
5. Ensure handgrips **DO NOT** twist on walker – otherwise damage or injury may occur.
6. **ALWAYS** test to see that the walker and attachments are properly and securely locked in the open position **BEFORE** using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. **ALWAYS** use caution when using the walker on wet and slippery surfaces. **DO NOT** use the walker on stairs and steep inclines.
9. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.
10. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
11. Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure, and that wheels (if applicable) and moving objects are in good working order before using this walker.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

STABILITY WARNINGS

Be careful when reaching for objects when using the walker. Reaching for objects can change the weight distribution of the walker and may cause the walker to tip, resulting in injury or damage.

MAINTENANCE

- For dust and dirt, wipe walker gently with damp, soft cloth.
- Inspect wheels (if applicable) periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the walker.
- Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ASSEMBLY OF THE WALKER

1. Remove closed walker from carton.
2. Install legs by depressing the height adjustment button and slide onto walker frame until the button engages in the desired adjustment hole. Adjust all leg attachments to the same height.
WARNING: Ensure all push buttons protrude fully from height-adjustment holes, all legs are adjusted to the same height, and wheeled leg attachments are positioned on the front and towards the outside of the walker.
3. To open walker, push handgrips and sides out away from each other until the release mechanisms lock into place.

CLOSING THE WALKER

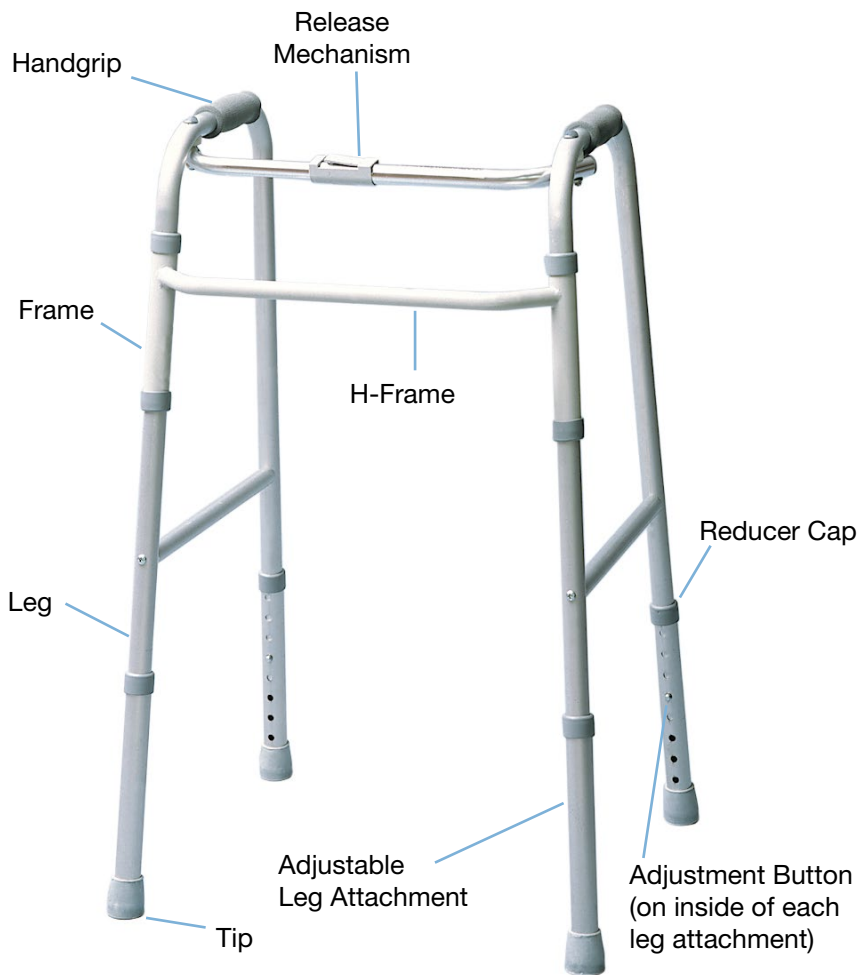
1. Push downward on release button and hold.
2. Grab one handgrip and push inwards towards center of walker.

ADJUSTING THE HEIGHT

NOTE: Height adjustment should be made to desired height before operation. Consult with healthcare professional for proper adjustment.

1. To adjust the height of the leg attachment, depress the adjustment button and slide upwards or downwards to the desired height.
2. Adjust the height of the walker by aligning all four leg attachments to the same position height.

When walking, the user should grasp one handgrip in each hand, positioning body between the two rear walker legs, and walking at a comfortable pace. The rear walker legs should be in contact with the floor at all times.

LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

Frame: one (1) year

Tips, Wheels/Casters, and Handgrips: three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing cs@grahamfield.com. Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

For additional information related to warranty claims, please visit GF's terms and conditions located on www.grahamfield.com

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Features

- Anodized aluminum frame supports a 300 lb weight capacity-
EVENLY DISTRIBUTED
- Single release folding mechanism is designed to aid users with limited hand dexterity
- Contoured, textured handgrips provide additional comfort
- Stable and lightweight frame makes it easy to maneuver

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com

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- Si los mangos están sueltos, **NO LOS** use. Llame a su distribuidor autorizado de GF Health Products, Inc.
- Para el mejor servicio de su andador, siempre use partes de reemplazo de GF Health Products, Inc.