



## **Platinum, Gold, Standard, Basic, and Bariatric Mattresses have Chemlux II and Chemsafe top/bottom covers.**

The following mattresses have **Chemlux II™** top covers and **Chemsafe™** bottom covers and meet the requirements of 16 CFR 1633\*:

Platinum Care Foam Mattresses, 416 and 519 Series  
Gold Care Foam Mattresses, 416 and 419 Series  
Standard Care Foam Mattresses, 316 and 319 Series  
Basic Care Foam Mattresses, 315 Series  
Bariatric Care Foam Mattresses, 650-1501-1633

### **Chemlux II top covers have the following benefits:**

- Bacteria resistant
- Flame Retardant
- Fluid resistant
- Near 0% vapor transmission – moisture will not build up under the cover in the foam from body heat
- Anti-fungal
- Anti-static
- Non-allergenic
- Soft, Durable

### **CARE AND MAINTENANCE OF CHEMLUX II TOP COVERS:**

A barrier sheet should be used whenever foam pads are used on mattresses.

Routine care: Wipe clean with neutral soap suds and lukewarm water. Rinse and dry.

Hard to clean areas: Use standard liquid cleaner and soft sponge. Solvents, harsh cleaners and detergents should not be used.

Disinfection: Germicides and disinfectants should be diluted as specified on product label of manufacturer. As recommended by the Centers for Disease Control, a 1:10 dilution of household bleach may be used.

### **Chemsafe bottom covers provide these benefits:**

- Bacteria Resistant
- Flame Retardant
- Fluid Resistant
- Anti-static
- Non-allergenic
- Stain resistant
- Soft, durable
- Micro Chek EPA Reg. #1486-19

### **CARE AND MAINTENANCE OF CHEMSAFE BOTTOM COVER:**

Soils and stains: Wash with neutral suds and lukewarm water. Rinse with water and dry.

Hard to clean areas: Use liquid vinyl cleaners and soft brush. Pre-soak, as needed. Rinse with water and dry.

Disinfections: Dilute disinfectants as per manufacturer's directions. Proper dilution and usage should be followed. Chemsafe may be cleaned as recommended by the Centers for Disease Control with a 1:10 dilution of household bleach.

Chemlux II and Chemsafe are produced by Chemtick Coated Fabrics, Inc.

\*Any Graham-Field Health Products mattress part number ending in "-1633" meets the requirements of 16 CFR 1633.