

Clamp Installation:

1. Position chair into semi-recline position.
2. Detach lower front end of side panel rod from chair frame by pulling rod toward rear.
3. Place clamp onto side rail with tube side in, and slide into place as far forward as possible.

Note: On the 565DG or 565TG, sandwich the coated metal spacer tube between the side rail and the insert bolts when assembling. When a higher footrest height is desired, the coated metal spacer may be placed on top of the side rail.

4. Insert bolts (head side out) while holding plastic inserts in-between sides of clamps so that the bolt is inside of the insert and the bottom of the bolt is showing through the square end of the clamp.
5. Place flat washers over the end of the bolts and then place split washers over the bolts against the flat washers.
6. Place wing nuts over the end of bolts and tighten.

Footrest Positioning:

Depress spring button on footrest tube and insert into receiving tube. Then, slide backward into position so that spring button engages the first hole on inside of clamp.

Note: The length of the footrests are adjustable. Proper length may be determined once both footrests are in place.



Graham-Field Health Products

2935 Northeast Parkway, Atlanta, Georgia 30360

telephone: 800-347-5678, 770-447-1609

fax: 800-726-0601, 678-291-3232

<http://www.grahamfield.com>