SIZE: 72\*97MM

**LUMISCOPE**  
For The Quality Of Life

**Advanced Wrist Blood Pressure Monitor Model 1147 Instruction Manual**

**Read these instructions before using your Blood Pressure Monitor. Save these instructions for future reference. The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).**

1147-INS-LAB-Rev.C14

**SAFETY PRECAUTIONS**

- WARNING: Babies, young children, or individuals who cannot express their consent are not suitable for blood pressure measurement.
- WARNING: Individuals with serious arteriosclerosis are not suitable for blood pressure measurement.
- WARNING: Blood pressure readings may fluctuate during pregnancy. Pregnant women should consult their doctor before taking measurement.
- WARNING: If there are unusual values, please consult your doctor before taking medical action.
- WARNING: Notice for California Customers-California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- CAUTION: Handling batteries properly:
  - As soon as batteries wear out, replace them with new batteries.
  - Do not use old and new batteries together.
  - Align the polarities of batteries correctly
  - When the unit will not be used for more than 3 months, remove the batteries. Otherwise, batteries may leak and cause damage to the unit.

**QUICK-START GUIDE**

Avoid smoking, eating or exercising for 30 minutes before taking a measurement. Do not move or talk during measurement. Measurement can be taken on either arm; however, if possible, always take measurements on the left arm.

- Sit upright in a chair with your feet on the floor.
- Remove tight-fitting or thick clothing from the lower arm.
- Pull on the end of the cuff until it wraps securely around your wrist. Place your arm on a table so that the cuff will be at the same level as your heart.
- Press the "O" button. The cuff will automatically inflate and the measurement will start.
- When the measurement is complete, the cuff will automatically deflate and your systolic and diastolic pressure values and pulse rate will display.

**OVERVIEW**

**Monitor**

**Features**

- One Touch Operation
- Irregular Heartbeat Detector
- Blood Pressure Mode and Clock Mode
- 4 User / 396 Total Memory
- 4 Alarm Clock Settings
- 5 Mode Averaging Function - Hour, Day, Week, Month, and Overall
- WHO Blood Pressure Indicator with 6-Color LED
- Auto-off Power Saving Feature
- Operated by 2 AAA batteries

**PREPARATION**

**2.1 Battery Installation**

- Remove the battery cover in the direction indicated on the cover.
- Insert 2 AAA batteries with the polarities as indicated.
- Slide the battery cover back to close it. The cover will click into place when completely closed.

Note: Replace batteries when the battery indicator "LO" shows low voltage.

**2.2 Setting the Date and Time**

Note:

- Set the correct date and time before taking a measurement.
- Reset the date and time after replacing the batteries.

- Press and hold the "OAVG" button for a few seconds until the Year display flashes.
- Press the "MEMO" or "PAGE" button to select the current year.
- Press the "OAVG" button to confirm Year Setting.
- Follow the same steps to set the Month, Date, Hour and Minute.

**2.3 Setting the Alarm**

Note:

- You will not be able to set alarm clock if Clock Function is set to "OFF".
- You can set a maximum of 4 alarms.

5. "ON" and "OFF" will then flash. This is to have the clock display when the unit is not in use; you can choose Clock Function "ON" or "OFF". Press "MEMO" or "PAGE" button to select. Press the "OAVG" button to confirm.

Note: If you select Clock Function "ON" you will continue at the Alarm Setting section. Otherwise, setting will end.

6. To disable Alarm, Press the "MEMO" or "PAGE" button to select "OFF" and press the "OAVG" button to confirm.

7. To set 1st Alarm, Press the "MEMO" or "PAGE" button to select "ON". Press the "OAVG" button to confirm.

8. When Hour digit flashes, press the "MEMO" or "PAGE" button to select Hour. Press the "OAVG" button to confirm.

**TAKING A MEASUREMENT**

Note:

- Avoid smoking, eating or exercising for 30 minutes before taking a measurement.
- Do not move or talk during measurement.
- Measurement can be taken on either arm; however, if possible, always take measurements on the left arm.

**3.1 Sitting Correctly**

- Relax.
- Sit upright in a chair with your feet on the floor.
- Remove thick or tight-fitting clothing from the lower arm. Do not roll up your sleeve if it is too tight.
- Place your arm on a table so that the cuff will be at the same level as your heart.

9. When Minute digit flashes, press the "MEMO" or "PAGE" button to select Minute. Press the "OAVG" button to confirm.

10. Follow the same steps to set the 2nd, 3rd, and 4th Alarm.

Note:

- Press the "O" button anytime during clock setting/ alarm setting to start over if an error occurs.
- Each alarm lasts 45 seconds, or you can turn it off by pressing the "O" button.

**3.2 Applying the Wrist Cuff**

- Place your elbow on a table with the palm facing upward. Wrap the wrist cuff around the left wrist as shown below.
- Pull on the end of the cuff until it wraps securely around your upper wrist. Do not over tighten the cuff. Allow 1-2 cm or 1/2 inch between the bottom of the cuff and your wrist joint.
- Make sure the cuff is at the same level as your heart.

**3.3 User Setting**

This unit allows a maximum of 4 users to store measurement records in different User Pages.

Press "PAGE" button to switch between User Page 1, 2, 3 and 4.

**3.4 Taking A Measurement**

- Press the "O" button. The cuff will automatically inflate and the measurement will start.
- When the measurement is complete, the cuff will automatically deflate and your systolic and diastolic pressure values and pulse rate will display.
- Remove the wrist cuff and take note of your measurement results. Note: You can press the "O" button to stop measurement at any time.

SIZE: 72\*97MM

**3.5 Heartbeat Readings**

Besides Systolic and Diastolic values, Pulse will also be displayed after a measurement

**Irregular Heartbeat**

This unit has a unique feature that alerts user of irregular heartbeat detection during measurement.

Note: An irregular heartbeat is defined as a heartbeat rhythm that has a variation of more than 25% from the average rhythm detected during measurement.

If an irregular rhythm occurs twice or more during the same measurement, the monitor will display the pulse value and "IH" symbol alternately. Example: (90 / IH)

Note: Please consult your doctor if irregular heartbeat occurs often.

**Regular but Below Normal Heartbeat**

If the monitor detects a pulse rate below 58 beats per minute during measurement, the monitor will display the pulse value and "LO" symbol alternately. Example: (58/LO)

**Regular but Above Normal Heartbeat**

If the monitor detects a pulse rate above 100 beats per minute during measurement, the monitor will display the pulse value and "HI" symbol alternately. Example: (102 / HI)

**3.6 WHO Light Indicator**

This unit features a 6-color LED light indicator to display blood pressure level in accordance with the WHO (World Health Organization) classification.

**WHO Classification**

Optimal	Blue
Normal	Sky Blue
High-normal	Green
Mild hypertension	Purple
Moderate hypertension	Yellow
Severe hypertension	Red

WHO Light Indicator

Example: If your measurement reading is 122/75, the LED light will display the color in blue.

**MEMORY FUNCTIONS**

After measurement, the last set of measured values will be saved automatically after measurement when the unit is turned off by pressing "O" button or by automatic power-off after 3 minutes.

**4.1 View Past Readings**

- Press the "MEMO" button to enter memory function.
- If the User Page is not what you want to view, press the "PAGE" button to select.
- Press the "MEMO" button to display the next sets of values for the user selected. Past readings are displayed from the most recent to the oldest.
- This blood pressure monitor can store up to 99 sets of past readings for each user (4 x 99).

**4.2 View Average Readings**

- Press the "OAVG" button to enter Average function.
- If the User Page is not what you want to view, press the "PAGE" button to select.
- Press the "OAVG" button to view different Average Values. 5 different Average Values are available:

Symbol on Display	Average Values of
1 h	One Hour
1 d	One Day
7 d	One Week
30 d	One Month
ALL	All Readings

**4.3 Delete Past Readings**

User can delete all the past readings stored in each User Page.

- Press the "PAGE" button to select the User Page you want to delete.
- Press and hold the "MEMO" & "PAGE" buttons simultaneously for 5 seconds. Symbol of "ALL EE" will be displayed and all past readings stored in that User Page will be deleted.

**TROUBLESHOOTING**

Symptom	Possible Cause	Solution
No display when you press the Power/Start button	Batteries are worn out Battery polarities are incorrect	Replace with new batteries Re-align batteries in correct polarities
"Err P" displays	Failure to inflate	Ensure the cuff is properly connected
"Err 1" displays	Deflating too rapidly	Return for service
"Err 2" displays	Movement during measurement Signal interference	Do not move during measurement Remove interfering source, e.g. mobile phones, magnets
"Err 3" displays	Incorrect results	Measure again

Note: If your problem cannot be solved by the above, consult a GF Health Products, Inc. representative. DO NOT disassemble the unit.

**SPECIFICATIONS**

Model No.	1147
Measurement Method	Oscillometric
Measurement Range	Pressure: 20 - 280 mmHg Pulse Rate: 40 - 195 beats/min
Accuracy	Pressure: +/- 3 mmHg Pulse Rate: +/- 5%
Inflation	Pump driven
Pressure Detection	Semi-conductor
Power Supply	2x AAA batteries
Auto Power Off	3 minutes after last operation
Storage Environment	-4°F to 140°F, 10% to 95% RH
Operation Environment	50°F to 104°F, 30% to 85% RH
External Dimensions	3.15" (L) x 3.35" (W) x 2.36" (H) or 80 x 85 x 60 mm
Net Weight	140g (batteries excluded)
Cuff Dimensions	Wrist circumference of 135-220mm
Package Contents	Blood Pressure Monitor, Instruction Manual, Batteries

**THIRTY-SIX MONTH LIMITED WARRANTY**

**SCOPE OF WARRANTY**

GF Health Products, Inc. ("GF") warrants to the original purchaser ("Customer") only that it will replace or repair components at GF's sole discretion, that are defective in material or workmanship under normal use for a period of thirty-six (36) months after the purchase date unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date. The warranty does not extend to non-durable parts and does not include labor or costs of shipping. This limited warranty is not transferable. All warranties are conditioned upon the proper use of the product strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. The warranty is void if the defect is caused by any other reason not related to defects in materials or workmanship.

**Obtaining Warranty Service**

GF's customer service team must be notified of any warranty claim within the applicable warranty period. Call 878-291-3207 or fax 770-368-2386 or e-mail [cs@grahamfield.com](mailto:cs@grahamfield.com). There is a \$15 fee for return shipping and handling. Please make checks payable to GF Health Products, Inc. Failure to follow the specific directions provided by the GF customer service team will result in denial of the warranty claim.

**ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

GF'S TOTAL LIABILITY FOR ANY PRODUCT OR SERVICE PROVIDED IS LIMITED TO THE COST OF THE PRODUCT GIVING RISE TO THE CLAIM. IN NO EVENT WHETHER IN CONTRACT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING BUT NOT LIMITED TO DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYER OR INDEPENDENT CONTRACTOR DAMAGES, PAIN AND SUFFERING. This warranty gives you specific legal rights. You may have additional rights which vary from state to state.

**Manufactured for:**  
**GF Health Products, Inc.**  
2935 Northeast Parkway  
Atlanta, Georgia 30360  
telephone: 770-368-4800  
fax: 770-368-2386

**© 2009 GF Health Products, Inc.**  
Made in China

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this document are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this manual, can be found online at [www.grahamfield.com](http://www.grahamfield.com).  
**Graham-Field, Lumiscope, and Lumiscope For The Quality Of Life** are registered trademarks of GF Health Products, Inc.

**STORAGE & MAINTENANCE**

- Keep away the unit from direct sunlight, extreme temperatures, humidity or moisture.
- Use a dry or slightly water-dampened cloth to clean the unit.
- Do not use alcohol, benzene, thinner or other volatile liquids to clean the unit.

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**Lumiscope For The Quality Of Life** son marcas registradas de GF Health Products, Inc.