

LUMISCOPE®

For The Quality Of Life™



Model 100-610

Adjustable Aneroid Sphygmomanometer

- Adjusting the Gauge -

The unit is properly calibrated when the needle rests at zero. (Fig A)



Fig A



Fig B

If the unit is not calibrated and the needle gauge points towards the left, use the screwdriver to turn the gauge counter-clockwise to zero the unit. (Fig B)

If the unit is not calibrated and the needle gauge points towards the right, use the screwdriver to turn the gauge clockwise to zero the unit. (Fig C)



Fig C

GF1200133RevB12