Garantía Limitada

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LUMEX®

Features

- Ideal for users who do not require a walker but need a wider base for support and for those limited to one hand or arm use
- Lighter than a walker; more stable than a cane
- Contoured, comfortable handgrips
- Folds easily with one hand
- Maximum Weight Capacity 300 lb - **EVENLY DISTRIBUTED**
- Adjustable in height from 29" - 34"
- Lifetime Limited Warranty

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

GF0600010RevD14

LUMEX®

Walkane

Andador Bastón

Item 6015A

User Instructions and Warranty Information

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Instructions for Use – Please Read All Instructions Before Use. Intended Use

This product is designed for use by those who:

- Require a wider base of support than that offered by most canes.

Please consult a healthcare professional for the proper height adjustment.

Height Adjustment

1. To adjust the height of the leg attachment, depress the adjustment button and slide upwards or downwards to the desired height.
2. Adjust the height of the Walkane by aligning all four leg attachments to the same position height.
3. Position the Walkane so that the closed end of the cross brace is facing away from you.

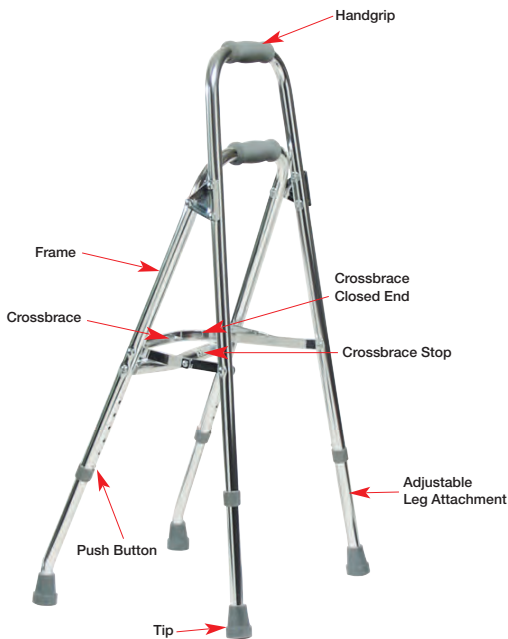
NOTE: The Walkane is packaged in its lowest height position. It should be adjusted to the desired height before operation.

To Open Walkane

1. To unfold Walkane, press down on crossbrace until it reaches the crossbrace stop.

To Close Walkane

1. Pull up on crossbrace.

Maintenance

- For dust and dirt, wipe Walkane gently with damp, soft cloth.
- Inspect periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the walker.
- Inspect Walkane weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, DO NOT use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

1. Each individual should ALWAYS consult with their physician or other health care professional to determine proper adjustment and usage.
2. Before use, be sure that the Walkane is in its open and locked position.
3. All legs MUST be in contact with the floor at all times during use. This will ensure the Walkane is properly balanced.
4. This product has a maximum user weight limitation of 300 lbs. ALWAYS observe the user weight limitation on the hangtag.
5. Ensure handgrips DO NOT twist on Walkane – otherwise damage or injury may occur.
6. ALWAYS test to see that the Walkane and attachments are properly and securely locked in the open position BEFORE using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. ALWAYS use caution when using the Walkane on wet and slippery surfaces. DO NOT use the Walkane on stairs and steep inclines.
9. DO NOT store or transport Walkane in open position. Fold Walkane when not in use.
10. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use EXTREME caution when reaching for any object.
11. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
12. Inspect Walkane weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner’s manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: If components are damaged or missing, contact your distributor immediately. DO NOT substitute parts.

WARNING: Before use, ensure that all assembled parts are secure and all hardware is tightened.

WARNING: Care should be taken to ensure that all height adjustments are secure, and that wheels (if applicable) and moving objects are in good working order before using this Walkane.

WARNING: ALWAYS observe the user weight limitation on the hangtag of this product.

WARNING: Notice for California Customers

California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: Be careful when reaching for objects. Reaching for these objects will cause a change to the weight distribution of the Walkane and may cause it to tip, resulting in injury or damage.

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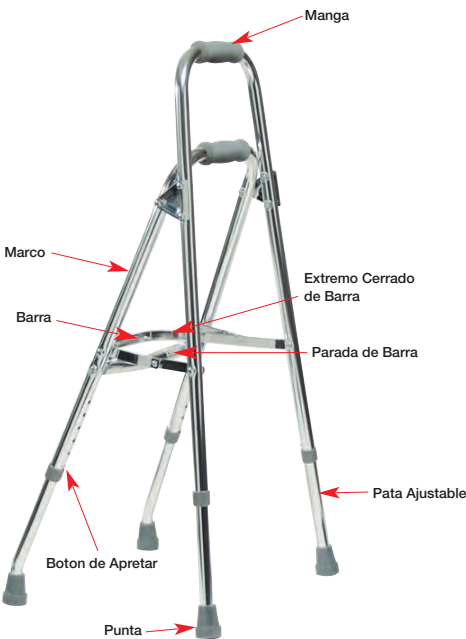
Cerrando el Walkane

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Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY. EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

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- Si los mangos están sueltos, NO LOS use. Llame a su distribuidor autorizado de GF Health Products, Inc.
- Para el mejor servicio de su Walkane, siempre use partes de reemplazo de GF Health Products, Inc..