Garantia Limitada

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Features

- Soft, cushion grip reduces hand stress and fatigue
- Easy height adjustment from 30" – 39" in 1" increments with locking mechanism for added security
- Attractive, designer finish
- 300 lb maximum weight capacity

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

GF1100100RevB14

Adjustable Offset Cane with Ortho-Ease Soft Grip

Baston Offset Ajustable con Manga Suave Ortho-Ease (6330 series)

Featuring the original
Therapist-preferred
Ortho-Ease® Grip

User Instructions and Warranty Information

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Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Instructions for Use – Please Read all Instructions Before Use.

The Lumex® Ortho-Ease® grip angle can be adjusted to comfortably fit the user’s hand and wrist angle, minimizing stress on the joints.

See instructions for grip adjustment below.

Adjusting Cane Height

1. Loosen locking collet by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height. The top of the cane should be even with the break in your wrist and/or your hip joint.

NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.

3. Secure locking collet by turning clockwise until tight.

Ortho-Ease® Adjustment

1. Loosen adjustment screw with slotted screwdriver.
2. Pull grip away from the cane until handgrip is free to rotate.
3. Rotate handgrip to desired position.
4. Push handgrip forward until it is no longer able to rotate and locks into place.
5. Tighten adjustment screw clockwise until secure.

Maintenance

- Wipe cane weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent.
- DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the cane.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

WARNING: DO NOT install or use this equipment without first reading and understanding these instructions. This product should not be used without proper instruction from a healthcare professional.

WARNING: Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

WARNING: This product has a maximum user weight limitation of 300 lb. The canes are NOT designed to support the total weight of individuals.

WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

WARNING: Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.

WARNING: Single push button height adjustment MUST be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. ALWAYS test to see that cane handle is properly and securely locked in place BEFORE using.

WARNING: Make sure locking collet is securely tightened at all times.

WARNING: Always use caution when using this product on wet or slippery surfaces.

WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

WARNING: Before use, inspect cane and ensure that it is locked securely in position. Inspect weekly.

WARNING: Notice for California Customers
California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: If any component is not in proper working order or appears worn, contact your dealer for repair and/or appropriate replacement parts.

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