



5532B: CHROME HAMPER ASSEMBLY INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Info: The most current version of these instructions can be found online at www.grahamfield.com.


SAFETY GUIDELINES - PLEASE READ BEFORE USE

- ⚠ **WARNING: Important: Read and understand these instructions before assembling or using the Chrome Hamper. Ensure that the Chrome Hamper is assembled as described in the following paragraphs before use. If the Chrome Hamper is not properly assembled, personal injury and/or property damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your Graham-Field authorized distributor immediately. DO NOT use substitute parts.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65**
WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

HAMPER ASSEMBLY (REFER TO PICTURE AT RIGHT)

Tools needed: 9/16" wrench, 1/2" wrench, 3/8" wrench

1. Install locking casters on the front end of the base as shown, upward through bottom tube. Install non-locking casters on the rear end of the base as shown, upward through bottom tube.
2. Install lock washers and 3/8" hex nuts on caster stems. Use 9/16" wrench to tighten.
3. Insert vertical tubes in the base as shown, until their snap pins lock into place in the holes in the base.
4. The bag holder rod is optional. If desired, insert it through the vertical tubes as shown. Install 1/4" lock nuts on the ends. Use 3/8" wrench to tighten.
5. Install the top frame and lid on the vertical tubes until vertical tube snap pins lock into place in the holes in the top frame and lid.
6. Insert top ends of connecting rods from the rear through holes in lid. Install 5/16" nuts on ends of connecting rods. Use 1/2" wrench to tighten.
7. Insert bottom ends of connecting rods through holes in foot pedal, facing inward. Install 5/16" nuts on ends of rods. Use 1/2" wrench to tighten.

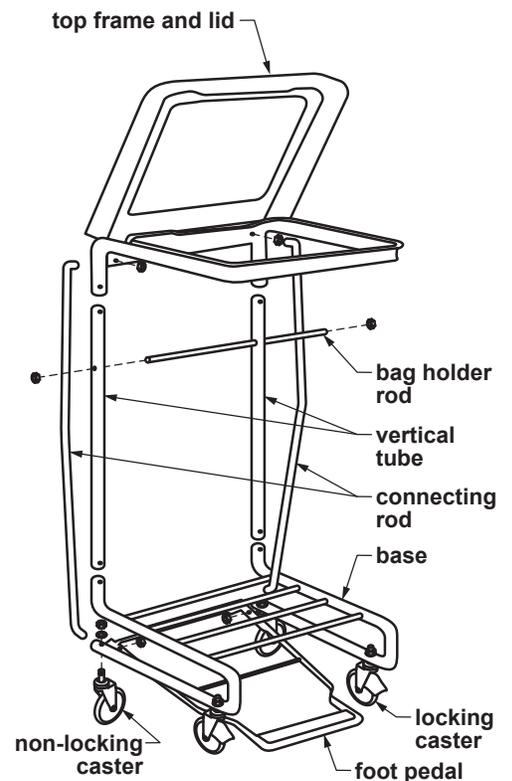

5532B Chrome Hamper

- ⚠ **WARNING: Ensure that all parts of Chrome Hamper are secure and assembled as instructed before use.**

MAINTENANCE / CLEANING

To clean the Chrome Hamper, wash it with soap and water or a mild detergent, then rinse.

▲ DO NOT use abrasive cleaners or scouring pads on this product.

LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES

DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.



GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com.

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2006 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in Taiwan