Info: The most current version of these instructions can be found online at www.grahamfield.com.

You will need: Screwdriver, drill, bit and mounting hardware as described in Installation type table below. Optional: Level, electronic stud locator.

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using grab bars. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If you are unable to perform this installation as described, please contact a licensed professional for installation. If the grab bar is not properly installed, personal injury and property damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.**
- ⚠ **WARNING: This product has a maximum weight capacity of 250 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION

Installation location

- ⚠ **WARNING: Ensure that there is no electrical wiring or plumbing at the chosen installation location.**
- ⚠ **WARNING: At least two screws in each flange MUST be mounted securely in a wood stud. If your installation site features standard 16"-centered wood studs, mount 12" grab bars vertically, and 24" grab bars vertically or at an angle, to ensure both ends are securely mounted in wood studs. Wood studs are located on 16" centers in most residential construction, but your installation may differ.**

1. Choose a mounting location most convenient for user. Place grab bar on wall, aligned with stud locations, and mark 3 mounting hole locations for each flange.
Note: If mounting grab bar horizontally or vertically, you may wish to use a level.
2. Determine which holes will be used for sheet metal screws and which for toggle bolts, if any.

Installation type

Installation type & wall surface	Drill bit needed	Mounting hardware required
Wood stud, tile wall	1/8" bit & 1/4" masonry bit	Sheet metal screw (provided)
Wood stud, untiled wall	1/8" bit	Sheet metal screw (provided)
Drywall (no wood stud), tiled or untiled wall	1/2" masonry bit	Toggle bolt: #10 or 3/16" flat head x at least 2 1/2" long

Drill

1. Choose the appropriate mounting method from a, b, or c.
 - a. **Sheet metal screw mount (into wood stud) on tile wall:** See figure 3. Drill 1/4" diameter hole *only* through tile with masonry bit, then drill 1/8" diameter approximately 1 3/4" deep through drywall and into wood stud.
 - b. **Sheet metal screw mount (into wood stud) on untiled wall:** See figure 3. Drill 1/8" diameter approximately 1 3/4" deep through outer wall, drywall, and into wood stud.
 - c. **Toggle bolt mount (no wood stud) on tile or untiled wall:** See figure 4. Drill 1/2" diameter through outer wall or tile and drywall with masonry bit.
2. After all holes are drilled, wipe wall with damp cloth to remove plaster powder.
3. Seal any unused holes in wall: Use finger to apply, press in, and smooth tub and tile caulking.

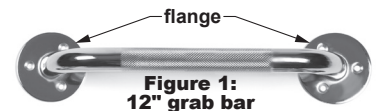
Figure 1:
12" grab bar

Figure 2:
flange (rubber gasket, mounted beneath flange, is not shown)

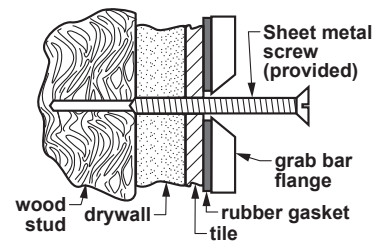
Figure 3:
sheet metal screw mount

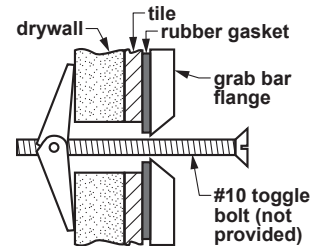
Figure 4:
toggle bolt mount

Mount

1. To seal, use finger to apply tub and tile caulking around installation edge of flange and outer edges of bolt or screw heads before installation.
 2. Attach grab bar and rubber gaskets to wall using appropriate mounting hardware as indicated in Figures 3 and 4. Install first one flange, then the other. Do not tighten completely until both flanges are installed. If installing toggle bolts, install them before sheet metal screws.
 - a. **Toggle bolt mount:** Insert toggle bolt through grab bar flange and gasket, then install toggle onto bolt and install bolt as specified in toggle bolt instructions.
 - b. **Sheet metal screw mount:** Insert supplied sheet metal screw through grab bar flange and gasket into wall.
 3. Use screwdriver to tighten all screws securely but do not overtighten, as this may damage tile or wall.
- ⚠ WARNING: Ensure that the grab bar is installed as described before use. If the grab bar is not properly installed, personal injury and property damage could result.**

OPERATION

- ⚠ WARNING: Users who have limited physical capabilities must be assisted or supervised when using the grab bar.**

MAINTENANCE

- ⚠ WARNING: Inspect grab bar weekly and tighten as needed to ensure secure installation. DO NOT use grab bar if looseness is detected.**

To clean grab bar, wash it with soap and water or a mild detergent, then rinse it with water. Ensure that grab bar is rinsed well before use.

- ▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the grab bar.**

THREE-YEAR LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, this product is warranted for three (3) years. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2005 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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