Available accessories for Walkers

For other options please visit us online at www.grahamfield.com

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Features

- Improves mobility and allows for greater independence for the user on the go
- Robust & easy to fold design makes rollator easy to use, fold and store for convenient transport
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with pouch for storage of personal items
- 8" wheels are ideal for indoor and outdoor use
- Maximum Weight Capacity 250 lb - EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2015, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

GF0700155RevD15

3-Wheel Cruiser

Items 609201P, 609201M, 609201B

User Instructions and Warranty Information
Important: Read Before Use
Instrucciones e Información de la Garantía
Importante: Lea Antes de Uso

Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE
This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS
The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER
THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Instructions for Use – Please Read All Instructions Before Use.

Please consult a healthcare professional for the proper adjustment.

Opening the Rollator

- Remove Rollator and other components from carton.
- Spread handgrips apart to open frame completely.
- Push down locking mechanism to secure the frame.

NOTE: Ensure locking mechanism is locked and secure before each use.

Folding the Rollator

- Pull up on the locking mechanism knob towards the rear part of the frame to release the lock.
- Push the two handgrips together.

Using the Hand Brake

Brakes should be used in one of the following methods:

- To Engage Brake: Push down on lower part of hand brake
- To Remain Stationary: Hold the hand brake up.
- To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.
- To unlock, pull up on the top portion of the brake handle.

CAUTION: Always lock hand brakes when user is immobile.

Adjusting the Height

- Consult with healthcare professional for proper adjustment.
- Install the adjustment knob into one of the adjustment holes by turning clockwise.
- Repeat steps 1-2 for the other side.

Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.

NOTE: After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

Attaching Basket, Pouch or Tote Bag

Basket

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

Pouch and Tote Bag

Hang the tote bag between the rollator frame tubes and secure with snap button flaps.

Maintenance

- For dust and dirt, wipe Rollator gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the rollator.
- Verify operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrip or hand brakes are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- For the best service from your rollator, always use replacement parts from GF Health Products, Inc.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

Safety Guidelines – Please Read Before Use

- Each individual should ALWAYS consult with their physician or other health care professional to determine proper adjustment and usage.
- The brakes should be in the locked position when pausing for extended time period.
- All wheels MUST be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
- Maximum Weight Capacity 250 lb - EVENLY DISTRIBUTED. ALWAYS observe the user weight limitation on the hangtag of this product and on the chart referenced at left.
- Ensure handgrips DO NOT twist on rollator handle - otherwise damage or injury may occur.
- ALWAYS test to see that the rollator and attachments are properly and securely locked in the open position BEFORE using.
- After installation and before use, ensure that all attaching hardware is securely tightened.
- ALWAYS use caution when using the rollator on wet and slippery surfaces. DO NOT use the rollator on stairs and steep inclines.
- DO NOT hang anything from the frame of the rollator. Items should be placed in the basket or tote bag.
- The rollator basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (4.5 kg). Items placed in either the basket or the bag should not protrude from the basket or bag.
- Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use EXTREME caution when reaching for any object.
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- If the hand brakes fail to stop the Rollator, contact your GF Health Products, Inc. authorized distributor immediately for adjustment or replacement.

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