
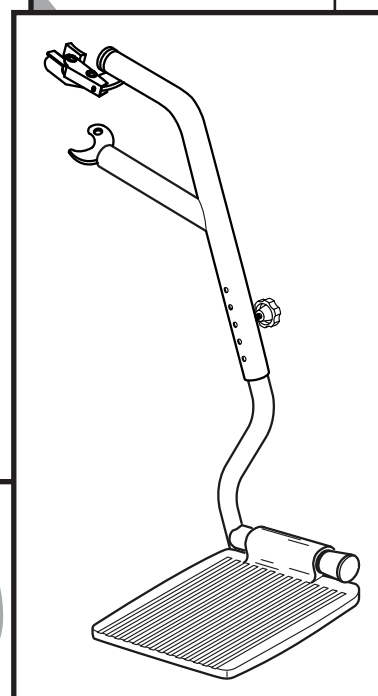
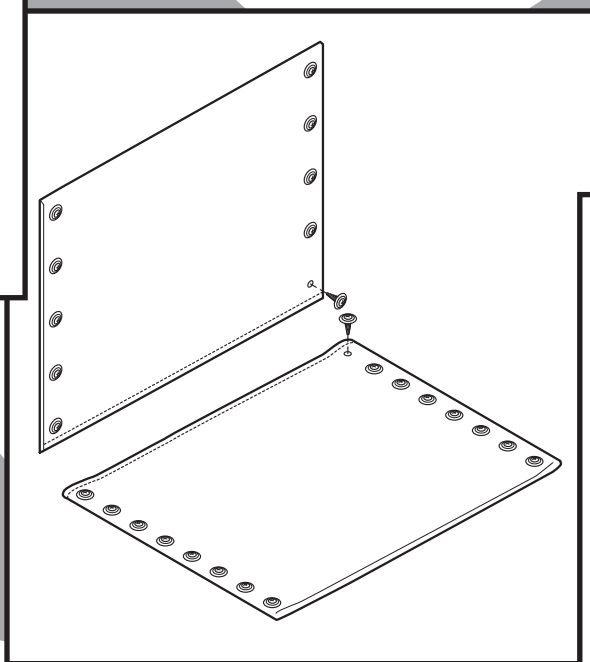


PARAMOUNT™ XD

PARTS CATALOG

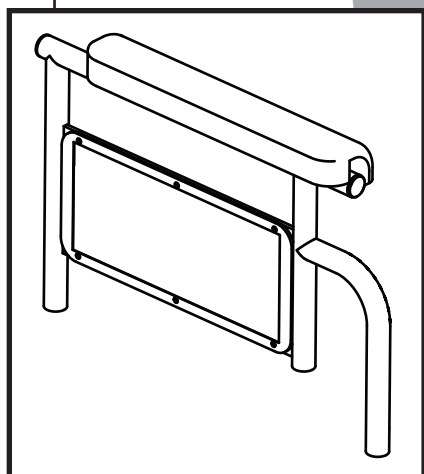

Everest & Jennings®

CONTENTS

SIDEFAME AND CROSSBRACE3

ARM4

CASTER.....4

FOOTREST5

ELEVATING LEGREST5

REAR WHEEL6

WHEEL LOCK6

BACK AND SEAT UPHOLSTERY.....7

ACCESSORIES7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386


Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.

GF Health Products, Inc. is not responsible for typographical errors. All design details, illustrations, specifications, packaging and warranties contained in this literature are based on the latest product information available at the time of printing and are subject to change. The most current product information can be found online at www.grahamfield.com. *Graham-Field* is a registered trademark; and *E&J, Everest & Jennings*, and *Paramount* are trademarks, of GF Health Products, Inc.

© 2011, GF Health Products, Inc., All Rights Reserved


Follow us on  

SIDEFRAME AND CROSSBRACE

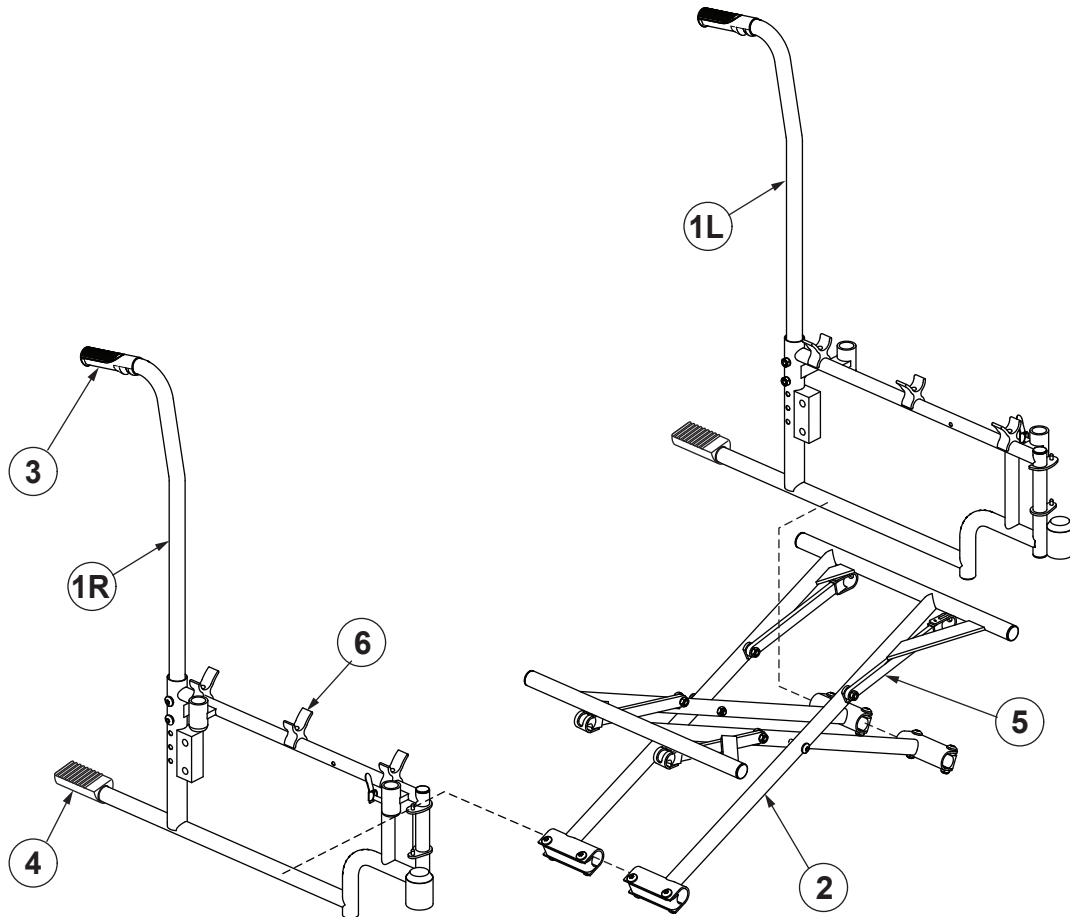


- | | |
|--|-----------------|
| 1 Sideframe, all widths; includes plugs | |
| Right | 90762901 |
| Left..... | 90762902 |
| 2 Crossbrace, pair; includes plugs and crossbrace assembly hardware | |
| 26" W x 18" D | 90762903 |
| 30" W x 18" D | 90762904 |
| 3 Grip, 10 each..... | 90762905 |
| 4 Tipping sleeve, 10 each | 90762906 |
| 5 Crossbrace link, 4 each; includes mounting hardware | |
| 26" W | 90762907 |
| 30" W | 90762908 |
| 6 Cradle, 6 each; includes mounting hardware | 90762909 |

ARM

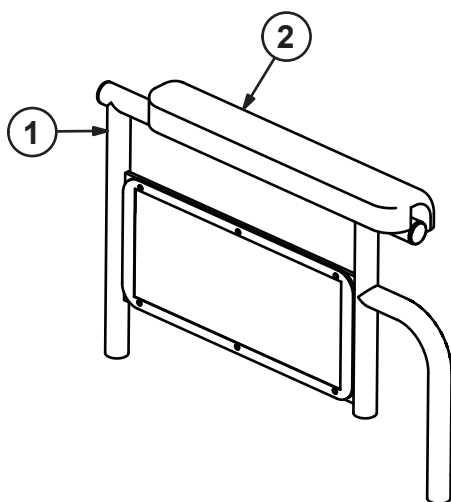


- 1 Arm, detachable fixed height desk-length, pair; includes left and right arm with side panel, spring lock, and black polyurethane arm pad 90762910
- 2 Replacement arm pad, desk-length, pair; includes 2 black polyurethane arm pads and mounting screws90762911

CASTER

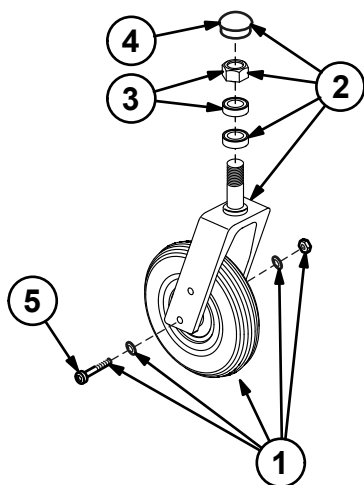


- Caster, complete; includes black aluminum fork and black plastic wheel with 8" gray polyurethane semi-pneumatic tire..... 90762912
- 1 Caster wheel, axle and nut; includes washers and axle bearings..... 90762913
 - 2 Caster fork, stem nut, dust cap and washer..... 90762914
 - 3 Caster stem bearing, 10 each..... 90762915
 - 4 Caster dust cap, 10 each 90762916
 - 5 Caster axle and nut, 10 each 90762917

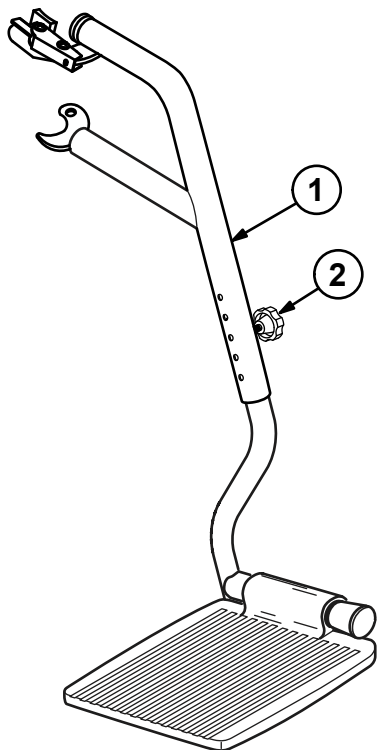


- 1 Footrest, swingaway and detachable, pair (left shown); includes right and left hanger, release, height-adjust knob, and aluminum footplate with extension90762918
- 2 Replacement height-adjust knob, black plastic, pair 90762938

ELEVATING LEGREST

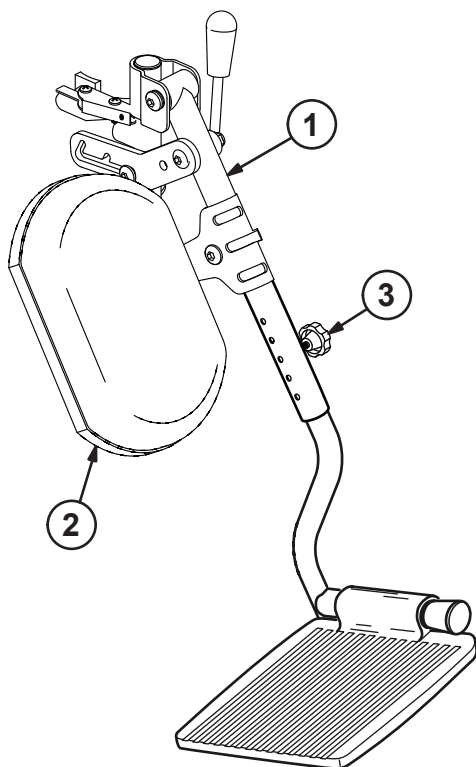


- 1 Elevating legrest, swingaway and detachable, pair (left shown); includes right and left hanger with release, height-adjust knob, panel, and aluminum footplate with extension 90762919
- 2 Elevating legrest panel; includes 2 black leatherette padded panels and hardware 90762920
- 3 Replacement height-adjust knob, black plastic, pair 90762938


REAR WHEEL

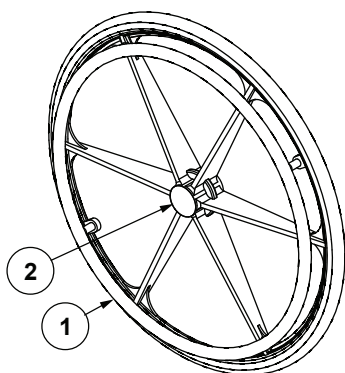

- | | |
|---|----------|
| 1 Rear wheel, 24" black composite, with solid gray rubber tire and chrome handrim; includes hub cap and wheel bearings..... | 90762922 |
| 2 Hub cap, 10 each..... | 90762923 |
| 3 Wheel bearing, 10 each..... | 90762924 |
| 4 Axle, nut and washer, 10 each | 90762925 |
| 5 Spacer, 10 each | 90762926 |

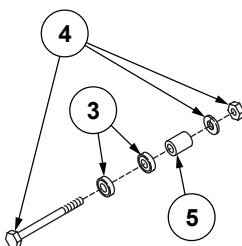


WHEEL LOCK

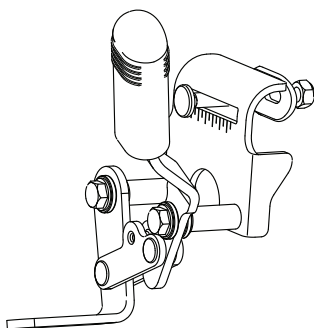

- | | |
|--|----------|
| Wheel lock, push to lock, pair; includes rubber tips | 90762927 |
|--|----------|

BACK AND SEAT UPHOLSTERY

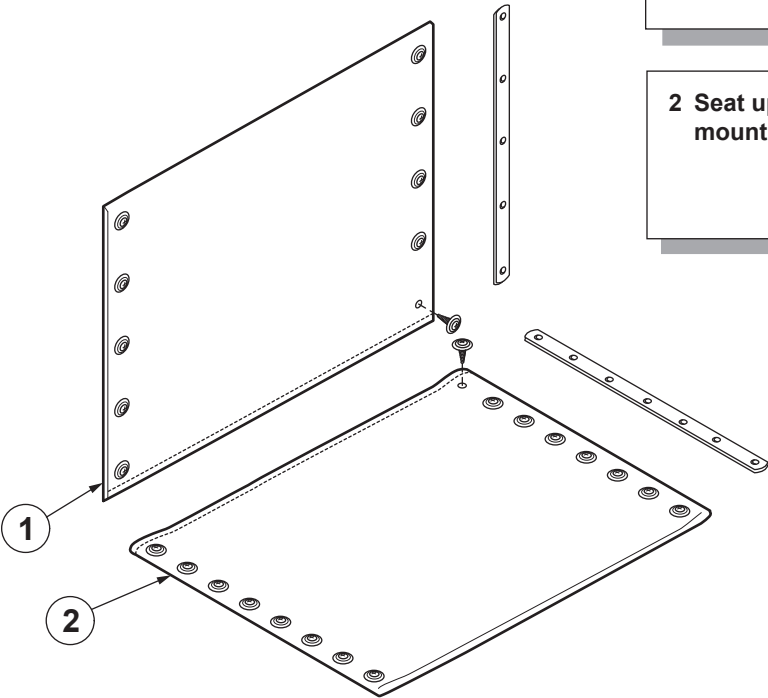

1 Back upholstery, black nylon; includes mounting strips and hardware

26" W..... 90762928
30" W..... 90762929

2 Seat upholstery, black nylon; includes mounting strips and hardware

26" W x 18" D 90762930
30" W x 18" D 90762931

ACCESSORIES

Paramount XD Accessories (Not Shown)

- | | |
|--|----------|
| 1 Heel loop, pair..... | 90762935 |
| 2 Wheel lock extension, 10 1/2", pair | 90762936 |
| 3 Positioning belt with metal buckle | 90762937 |
| 4 Anti-tipper, pair..... | 9A000022 |

Everest & Jennings®

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360 U.S.A.
telephone: 770-368-4700
fax: 770-368-2386**



GRAHAM-FIELD®

www.grahamfield.com

**Paramount XD Parts Catalog
GF0700096RevE15**