
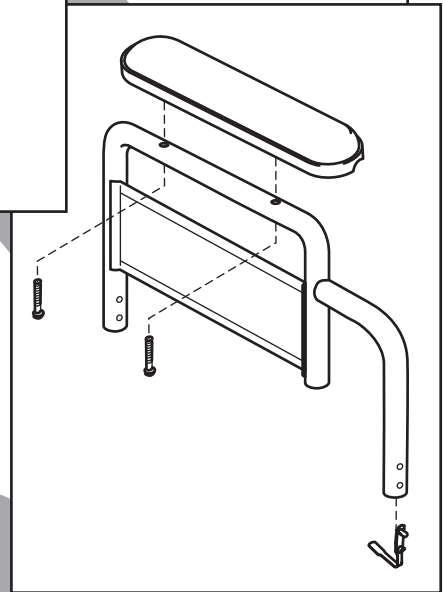
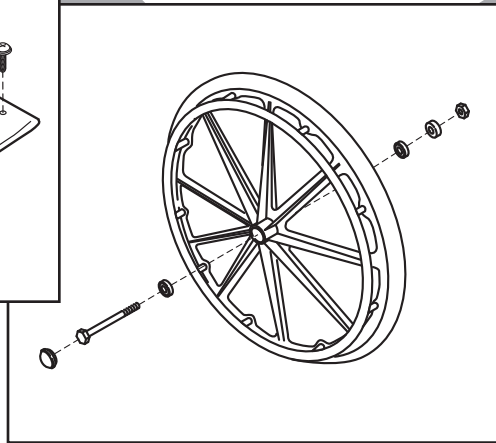


Traveler[®] ***SE***

PARTS CATALOG

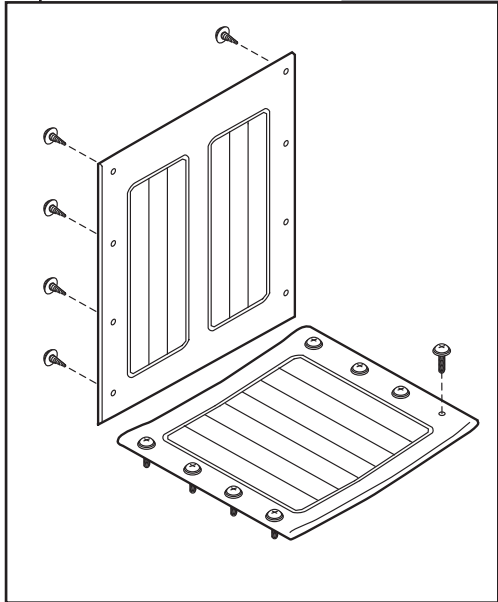

Everest & Jennings[®]

CONTENTS

SIDEFAME (FIXED ARM), CROSSBRACE, ARM PAD KIT3

SIDEFAME (DETACHABLE ARM), CROSSBRACE.....4

ARM, DETACHABLE5

CASTER.....5

REAR WHEEL5

FRONT RIGGING, DETACHABLE6

UPHOLSTERY (ALL SIDEFAMES)7

ACCESSORIES7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386


Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this catalog are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this catalog, can be found online at www.grahamfield.com.

Graham-Field and Traveler are registered trademarks; and Everest & Jennings and E&J are trademarks; of GF Health Products, Inc.

© 2008, GF Health Products, Inc., All Rights Reserved

SIDEFRAME (FIXED ARM), CROSSBRACE, ARM PAD KIT

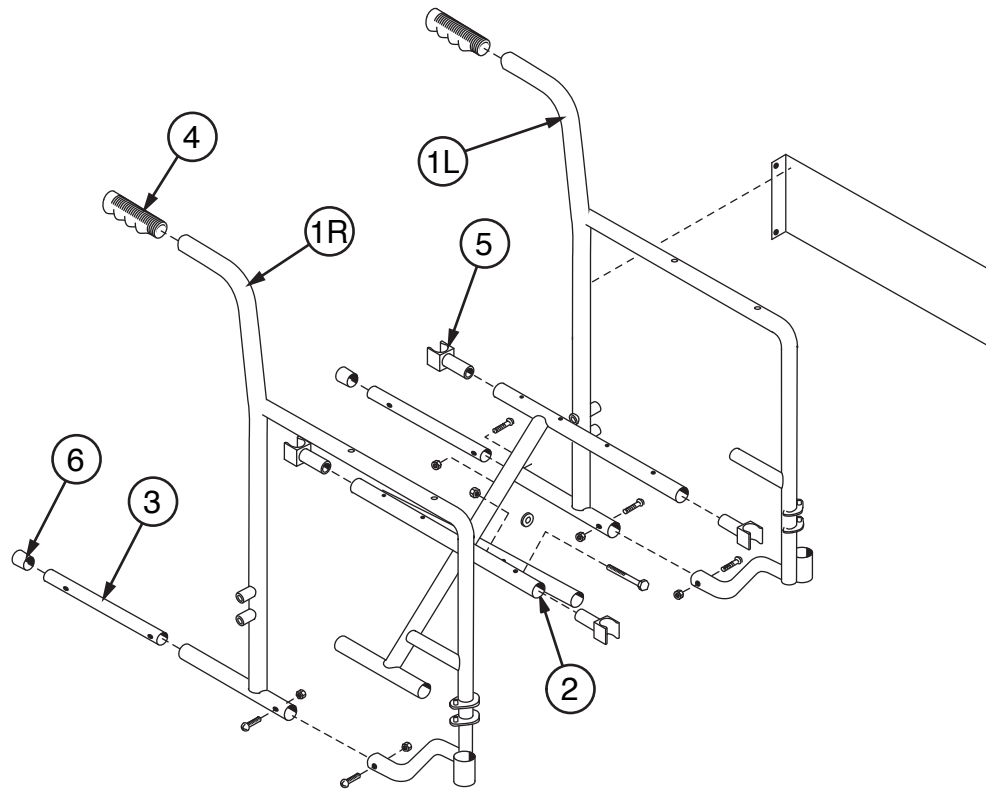

**1 Sideframe, fixed arm, all widths
(includes side panel)**

Right 90763001
Left 90763002

2 Crossbrace, pair (includes center bolt and nut)

16" W 907630C1
18" W 907630C2


**3 Bottom inner rail kit (includes bottom
inner rail (2 each) and screw and nut that
hold bottom inner rail (4 each)**

..... 90763003

4 Grip (10 each) 90763005

5 Seat rail guide (4 each) 90763004


6 Tipping sleeve (10 each)..... 90763006

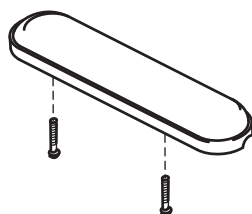

Arm pad kit

Includes 2 black full length
arm pads and mounting screws 90763023

SIDEFRADE (DETACHABLE ARM), CROSSBRACE

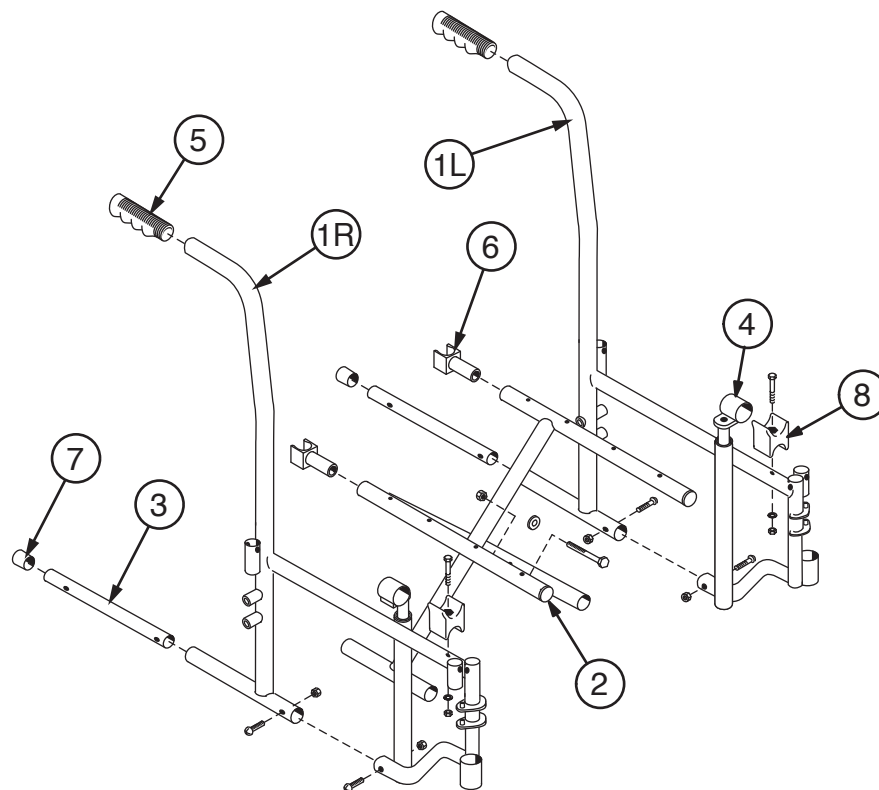

1 Sideframe, detachable arm

16" x 16", 18" x 16", 20" x 16"

Right 90763011

Left 90763012

20" x 18"

Right 90763113

Left 90763114

2 Crossbrace, pair (Includes center bolt and nut)

16" W x 16" D 907630C1

18" W x 16" D 907630C2

20" W x 16" D 907630C3

20" W x 18" D 907631D3

3 Bottom inner rail kit (includes bottom inner rail (2 each) and screw and nut that hold bottom inner rail (4 each))

..... 90763003

4 Seat slide (2 each) 90763013

5 Grip (10 each) 90763005


6 Seat rail guide, rear (2 each) 90763015

7 Tipping sleeve (10 each) 90763006

8 Seat rail, cradle, screw, washer and nut, front (2 each)


..... 90763403

ARM, DETACHABLE

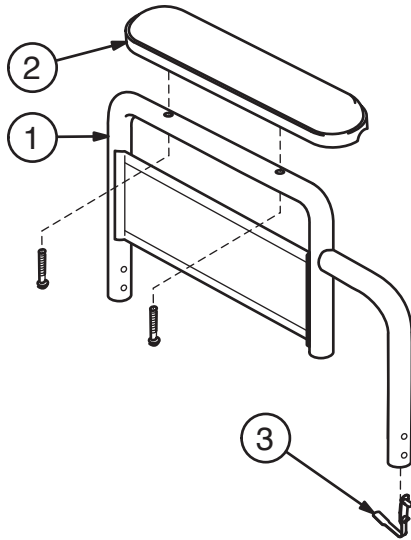

- 1 Arm, detachable fixed-height, pair (includes right and left arm with side panel and spring lock (does not include arm pad))
 - Desk length..... 90763020
 - Full length..... 90763021
- 2 Arm pad, pair (includes two black arm pads and mounting screws)
 - Desk length..... 90763022
 - Full length..... 90763023
- 3 Spring lock (10 each) 90763024


CASTER

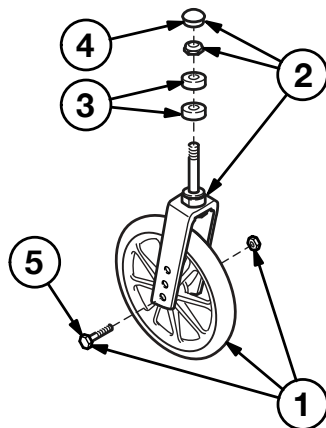

Complete caster (fork and wheel),
8" x 1" solid rubber tire..... 90763040


- 1 Caster wheel, axle and nut 90763041
- 2 Caster fork, stem nut and dust cap 90763042
- 3 Caster stem bearing (10 each) 90763043
- 4 Caster dust cap (10 each)..... 90763044
- 5 Caster axle and nut (10 each)..... 90763045

REAR WHEEL

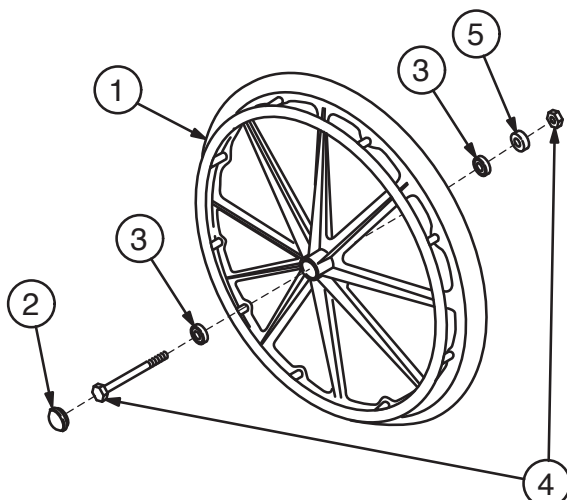


- 1 Rear wheel with handrim, 24" x 1", solid rubber tire (includes hub cap and bearing; does not include axle or nut) 90763050
- 2 Hub cap (10 each)..... 90763051
- 3 Wheel bearing (10 each) 90763052
- 4 Rear axle and nut (10 each)
 - Fixed-arm sideframe 90763053
 - Detachable-arm sideframe 90763054
- 5 Spacer (10 each)..... 90763055

FRONT RIGGING, DETACHABLE

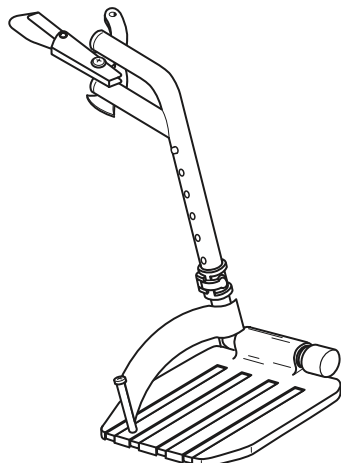


Footrest, pair, all widths
Includes right and left hanger with release
and composite footplate with extension
and heel loop 90763030

Composite footplate, pair
(Fits all front rigging) 90763033

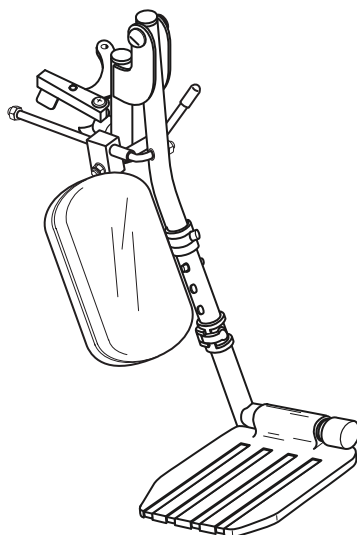


Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
composite footplate with extension, and
legrest panel (does not include
heel loop) 90763031

Elevating legrest panel kit
(Fits all elevating legrests)
Includes 2 black padded panels
and hardware 90763032

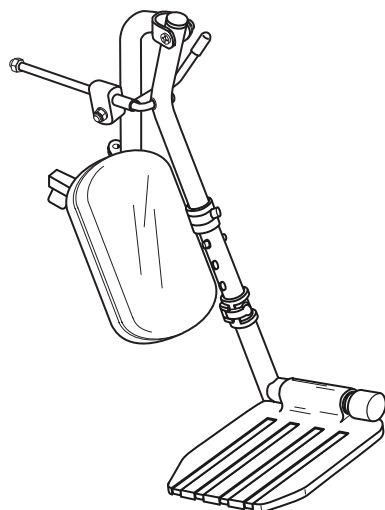

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop) 90763435

UPHOLSTERY

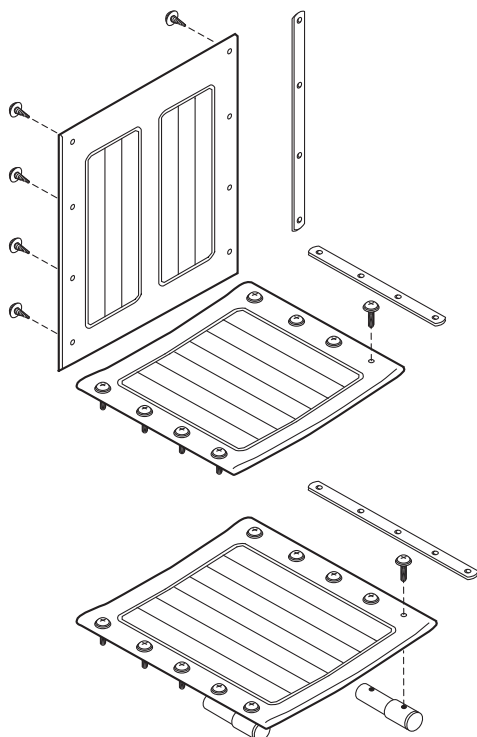

Back upholstery, black leatherette, with
built-in chart holder (includes mounting
strips and hardware)

16" W.....	907630A1
18" W.....	907630A2
20" W.....	907630A3

Seat upholstery black leatherette
(includes mounting strips and hardware)

16" W x 16" D.....	907630B1
18" W x 16" D.....	907630B2
20" W x 16" D.....	907630B3
20" W x 18" D.....	907630L3

Seat upholstery extension, 18" deep
(shown at bottom left), black leatherette
(includes seat extension tubes,
18" mounting strips and hardware)

16" W x 18" D.....	907630K1
18" W x 18" D.....	907630K2
20" W x 18" D.....	907630K3

ACCESSORIES

Traveler SE accessories (not shown)

1 Heel loop, pair.....	90763064
2 Wheel lock, pair (includes rubber tips)	
Fixed-arm chair	
Push to lock	90763070
Pull to lock	90763071
Detachable-arm chair	
Push to lock	90763060
Pull to lock	90763061
3 Rubber tip, wheel lock (10 each).....	90763063
4 Wheel lock extension, pair	90763062
5 Positioning belt (metal buckle)	92133040
6 Anti-Fold bar	92523850
7 O2-IV holder	B10320F
8 O2 cylinder holder.....	B10325
9 Anti-Tippers (pair)	
20" seat height	90763065
18" seat height	90763066

Everest & Jennings®

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360 USA**

**tel: 770-368-4700
fax: 770-368-2386**



GRAHAM-FIELD

www.grahamfield.com

**Traveler SE Parts Catalog
GF0800179RevH15**