



300DLX Series Lightweight Single-Head Stethoscope



Instructions for Use

Features:


- Unit weight 2.5 oz (70 g)
- 1 3/4" (4.5 cm) diameter diaphragm with non-chill ring
- White firm threaded PVC eartips
- Adjustable anodized aluminum binaurals
- Flexible PVC tubing
- 22" (56.5 cm) tubing; 31.5" (80 cm) total length
- One-year limited warranty

* This product is not made with natural rubber latex.

300DLX-INS-LAB-RevA16

INTRODUCTION

Congratulations on your purchase of the Labtron 300DLX Series Lightweight Single Head Stethoscope! The slim, lightweight, anodized aluminum chestpiece easily slides beneath a blood pressure cuff, and the binaurals are anatomically adjustable to ensure a comfortable, acoustically-sealed aural fit.

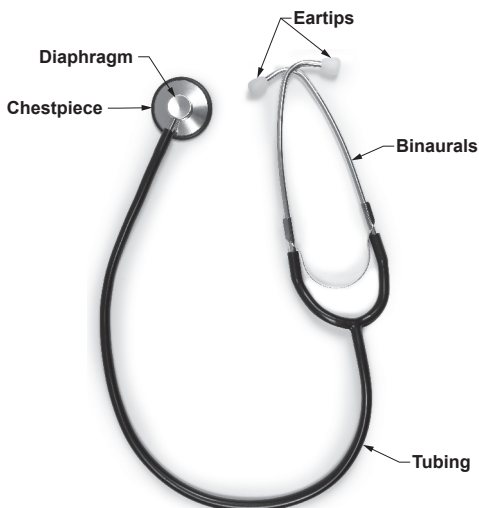

Important Safety Information

⚠ WARNING: Important!

Read and understand these instructions before assembling or using the Labtron 300DLX Series Stethoscope. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the stethoscope is not properly assembled, personal injury and damage to the stethoscope could result.

⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.

⚠ WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Labtron replacement parts. Non-Labtron replacement parts could cause personal injury and damage to the stethoscope.

⚠ WARNING: California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

Intended Use

The 300DLX Series Stethoscope is intended to facilitate the ability to hear internal sounds. This unit is ideal for general examination and blood pressure measurement.

Components

Included: stethoscope with diaphragm-only chestpiece, tubing, binaurals, and one set of eartips.

INSTRUCTIONS FOR USE

Adjusting the Binaural Tension


▲ **NOTICE:** The binaurals only need to be adjusted slightly — **DO NOT** squeeze or pull excessively.

To tighten the binaurals, hold binaurals at the base, at the bottom of the "Y", as shown at right. Squeeze together gently until the eartips cross and the desired tension is achieved.

To loosen the binaurals, grasp binaurals with both hands above the "Y", as shown at right. Pull binaurals apart gently until the desired tension is achieved.



Increase binaural tension (tighten)

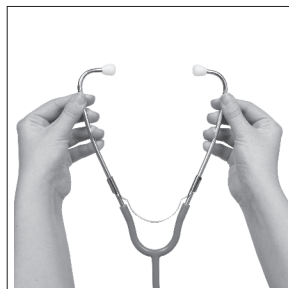

Decrease binaural tension (loosen)

Operation

1. If not already assembled, screw eartips into binaurals.
2. Clean the stethoscope (see the following instructions).
3. Ensure binaurals are adjusted to a comfortable tension. If not, follow previous instructions to adjust binaural tension.
4. Gently insert binaural eartips into ears, ensuring the eartips are pointing forward.
5. Tap the diaphragm softly; this should produce sounds clearly.
6. During use, in order to better hear different frequencies, vary the amount of pressure on the diaphragm.
7. To ensure your stethoscope remains comfortable, check frequently to ensure the eartips are tightly secured to the binaurals.

CARE, MAINTENANCE, AND CLEANING

▲ **NOTICE:** **DO NOT** subject the stethoscope to extreme temperatures or direct sunlight.

▲ **NOTICE:** **DO NOT** immerse in liquid, sterilize, or apply solvent or oil to the stethoscope.

Wipe stethoscope with a clean, soft, nonabrasive cloth dampened with a mild soap and water solution, then dry with a clean, soft, nonabrasive cloth. Clean stethoscope thoroughly before each use. If disinfection is required, wipe the stethoscope with a 70% isopropyl alcohol solution.

Remove and clean eartips as needed to prevent body secretions or other foreign matter from entering the eartips or tubing. To remove, grasp eartips firmly and unscrew them from binaurals. Reinstall eartips immediately after cleaning.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for one (1) year. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto.

The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com

Follow us on



© 2016, GF Health Products, Inc. All Rights Reserved.
Graham-Field and Labtron are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.
Made in China