



GFHealth Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360-2808
t: 1.770.368.4700 • f: 1.770.368.4954
www.grahamfield.com

3DL Thermoformed Casegoods Cleaning

3DL laminates can be safely cleaned and disinfected with a number of cleaners and disinfectants.

In laboratory testing using both NEMA LD 3.4 and ASTM D3450 test methods, 3DL laminates were shown to have no discoloration or damage to the finish when cleaned and disinfected with the products shown below.

Acceptable Cleansers:

- Asepticare TB-II
- Bleach 5:1
- Bleach Rite® disinfecting spray
- Cavicide® Surface disinfectant
- Citrace® germicide
- Clorox® germicidal wipes
- Dispatch® spray hospital cleaner with bleach
- Fade-a-Dyne® stain remover
- Lysol® spray disinfectant
- Precise® hospital cleaner
- PDI® Super Sani-Cloth® germicidal wipes
- PDI® Sani-Cloth HB germicidal wipes
- PDI® Sani-Cloth Plus germicidal wipes
- SaniZide® Plus germicidal solution
- Virex® II 256 cleaner
- Virox® AHP 5 disinfectant cleaner surface wipes

Always use a soft, cotton cloth or non-abrasive sponge.

Before using any of the above on the entire surface, be sure to test it on a non-visual spot.

TaTCleaners to Avoid:

- Any abrasive cleaner
- Most solvent-based cleaners and harsh chemicals, ex. sink and toilet bowl cleaners
- Any cleaner in combination with a brush
- Cleaners containing ammonia, acetone, or lacquer thinners
- Baking soda