## Garantía Limitada

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# LUMEX®

### Features:

- Offset handle places user’s weight over the base for maximum balance and control
- Stability is enhanced with the low profile, 4-point base
- Simple height adjustment from 29" – 38" in 1" increments with locking mechanism for added security
- Strong and durable rounded base protects user’s legs when walking
- Maximum Weight Capacity 500 lb  
- EVENLY DISTRIBUTED

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1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

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[www.grahamfield.com](http://www.grahamfield.com)

Follow us on [in](#) [YouTube](#) [f](#)

© 2016, GF Health Products, Inc. All Rights Reserved.  
Lumex is a trademark of GF Health Products, Inc.  
GF Health Products, Inc. is an ISO 13485 Certified Company.

GF090015SRREVCI6

# LUMEX®

## Silver Collection Low Profile Bariatric Quad Cane

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Items 6130

### User Instructions and Warranty Information

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[grahamfield.com](http://www.grahamfield.com)

## Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

## Instructions for Use – Please Read all Instructions Before Use.

Please consult a healthcare professional for the proper height adjustment and usage.

### Adjusting Cane Height

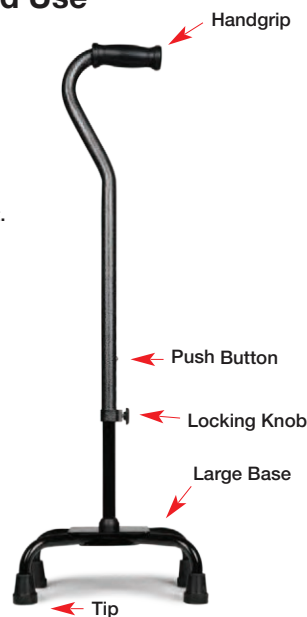
1. Remove locking knob by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height.

**NOTE:** An audible “click” will be heard when push button securely protrudes through the adjustment hole.

3. Secure locking knob by reinserting and turning clockwise til tight.

### Adjusting Quad Canes for Right or Left Handed Use

1. Loosen locking knob by turning screw knob counter-clockwise.
2. Press the push button and turn cane handle 180° until push button locks in adjustment hole.
3. Secure locking knob by turning screw knob clockwise until tight. DO NOT over tighten.

### Maintenance

- Wipe cane weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent.
- DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the cane.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your cane, always specify Lumex replacement parts, available from your GF Health Products, Inc. Distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

### Safety Guidelines – Please Read Before Use

**WARNING:** DO NOT install or use this equipment without first reading and understanding these instructions. This product should not be used without proper instruction from a healthcare professional.

**WARNING:** Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

**WARNING:** This product has a maximum user weight limitation of 500 lb. The canes are NOT designed to support the total weight of individuals.

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.

**WARNING:** Single push button height adjustment MUST be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. ALWAYS test to see that cane handle is properly and securely locked in place BEFORE using.

**WARNING:** Make sure locking collet is securely tightened at all times.

**WARNING:** When using a Quad cane, the cane should be placed on a level walking surface with all four legs in contact with the surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury.

**WARNING:** For stability, ensure that shorter legs of the base are closest to foot - otherwise, injury or damage may occur. It may be necessary to adjust cane for right-hand or left-hand use.

**WARNING:** Always use caution when using this product on wet or slippery surfaces.

**WARNING:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

**WARNING:** Inspect cane weekly for proper function and wear.

**WARNING:** Notice for California Customers  
California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** If any component is not in proper working order or appears worn, contact your dealer for repair and/or appropriate replacement parts.

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