The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ WARNING: Important! Read and understand these instructions before installing or using the Lumex Walker Attachments. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the Walker Attachments are not properly installed, personal injury and damage to the Walker Attachments could result.**
- ⚠ WARNING: Do not install the Walker Attachments unless prescribed by a healthcare professional.**
- ⚠ WARNING: Do not use the Walker Attachments with any other walker accessories unless prescribed by a healthcare professional.**
- ⚠ WARNING: The Walker Attachments are recommended for use on Lumex walkers.**
- ⚠ WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts.**
- ⚠ WARNING: The Walker Attachments do not have their own weight capacity limitation; the weight capacity limitation is that of the walker on which they are installed.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION (REFER TO PICTURES)


Forearm Walker Attachments are shown installed on walker in front view picture at right. Use this for an installation guide; note orientation of attachments.

- ⚠ WARNING: GF recommends installing the platform walker attachments on both sides for better stability.**
- ⚠ WARNING: Ensure walker is securely locked into the open position before installing the Walker Attachments, as shown at right, or serious personal injury may result.**

LOWER ATTACHMENT

Install Walker Attachment support tube clamps on walker crossbars as shown in pictures at right. Install knobs and hardware as shown; tighten until secure. Repeat for other side.

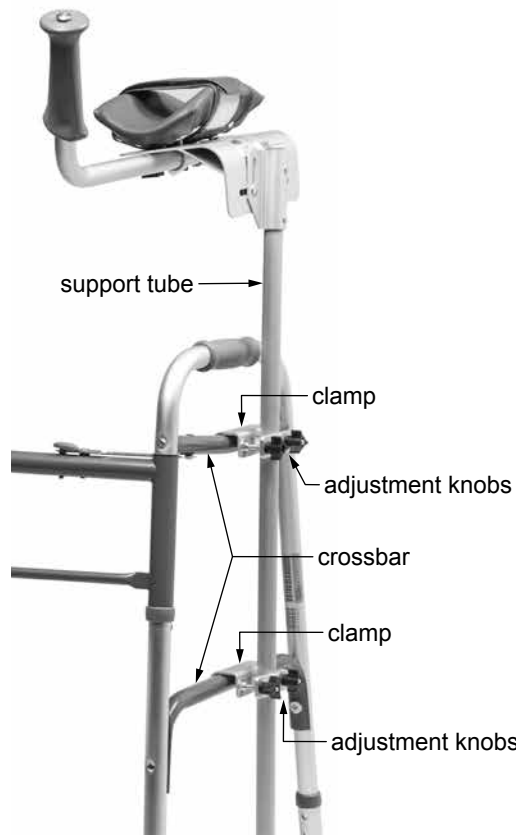Install Walker Attachment
on Walker Crossbars

Closeup of Walker
Attachment Installation
on Walker Crossbars

UPPER ATTACHMENT


Install Armrest Tube (left side shown)

Install armrest tube below platform as shown in left rear view picture at right. Insert carriage bolts, plain washers, and knobs; tighten until secure. Repeat for other side.

Install Armrest Assembly (left side shown)

Install armrest assembly on support tube. Insert carriage bolts, eight-square washers, and knobs as shown in left rear view picture at right; tighten until secure. Repeat for other side.

⚠ WARNING: Ensure each Walker Attachment is securely installed as instructed before use.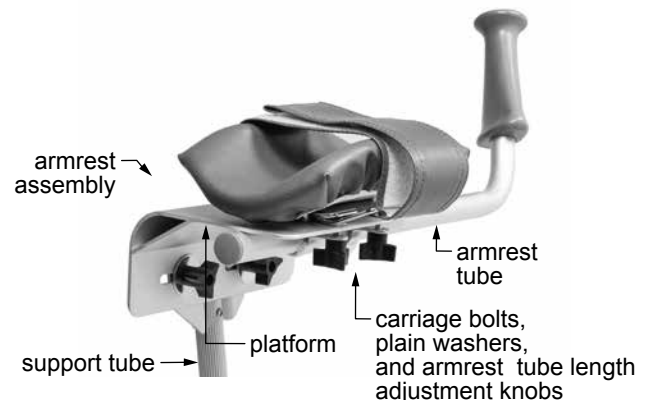

Install Armrest Tube
Install Armrest Assembly on Support Tube
Adjust Armrest Tube Length

ADJUSTMENT (LEFT SIDE SHOWN)

Adjust Armrest Tube Length

Loosen the arm tube adjustment knobs shown at right and slide arm tube in or out to the desired position. Tighten knobs securely to secure arm tube. Repeat for other side.

Adjust Armrest Tilt

Loosen the armrest tilt adjustment knobs shown at right and adjust platform tilt up and down along slots in backside of platform to desired tilt. Tighten knobs securely to secure arm tube. Repeat for other side.

Adjust / Secure Arm Strap

Pull hook and loop fastener apart to free arm strap. Slide arm into position so that armrest is comfortable and you have a comfortable hold on handgrip. Secure hook and loop fastener well so that it is tight but not uncomfortable around arm. Repeat for other side.

Adjust Attachment Support Height

Loosen both sets of height adjustment knobs shown in picture at right. Slide support tube up or down to the desired height. Tighten both sets of knobs securely to secure support tube. Repeat for other side.

Adjust Attachment Support Angle

Loosen both height angle adjustment knobs shown in picture at right. Slide both clamps forward or backward along crossbars to desired angle. Tighten both sets of knobs securely to secure clamps. Repeat for other side.

OPERATION


- ⚠ **WARNING:** Before use, practice using the Walker Attachments with the guidance of a healthcare professional until you are comfortable with their operation.
- ⚠ **WARNING:** Before each use, ensure all Walker Attachment hardware is securely attached.
- ⚠ **WARNING:** Do not lean too far forward or sideward when using the Walker Attachments; the walker could become unbalanced and serious personal injury could occur.
- ⚠ **WARNING:** Ensure all four legs are in contact with the floor when weight is exerted on the walker with the Walker Attachments.

MAINTENANCE / STORAGE


Maintenance: To remove dust and dirt, wipe the Walker Attachments gently with a clean, soft, damp cloth. To clean them, use a clean, soft cloth dampened with a mild soap and water solution, rinse with clear water, then dry with a soft cloth. To aid in rust prevention, lubricate adjustment knob threads.

▲ **NOTICE:** DO NOT use solvents, abrasive cleaners, or scouring pads to clean this product.

Storage: To remove platform attachments for storage, remove lower clamps and hardware from walker crossbars; package and keep them with the Walker Attachments for future use. Store in a clean, dust-free environment.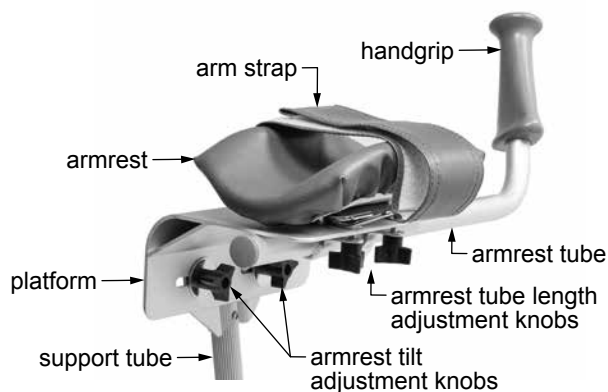

Adjust Armrest

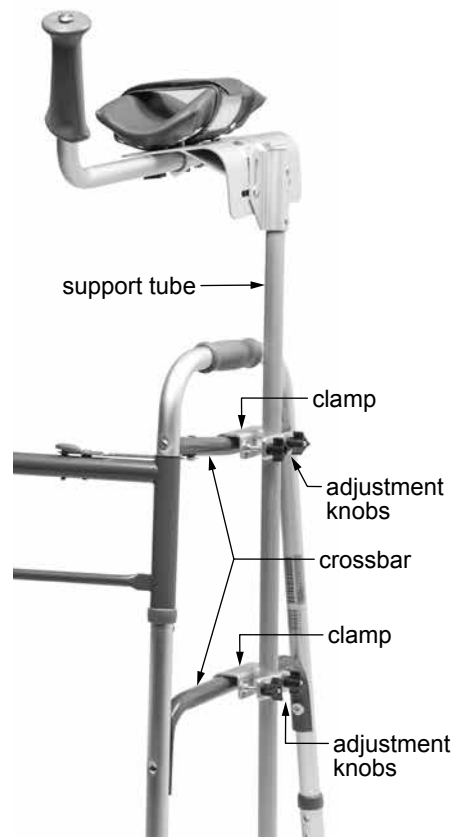Adjust Walker Attachment Height

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) years. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com

© 2004, GF Health Products, Inc. All Rights Reserved.
Graham-Field and Lumex are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.
Manufactured for GF Health Products, Inc.
Made in China

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© 2004, GF Health Products, Inc. Todos los derechos reservados.
Graham-Field and Lumex son marcas de GF Health Products, Inc.
GF Health Products, Inc. es una empresa certificada ISO 13485.
Fabricado para GF Health Products, Inc.