

Info: The most current version of these instructions can be found online at www.grahamfield.com.


- ⚠ **WARNING:** Important! Read and understand these instructions before installing or using the Raised Toilet Seat with Locking Clamp. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the raised toilet seat.
- ⚠ **WARNING:** This product has a weight limit of 300 lb, **EVENLY DISTRIBUTED**.
- ⚠ **WARNING:** Notice for California Customers - California Proposition 65 **WARNING:** This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov/product.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

INTENDED USE

The raised toilet seat is intended for use by those who have difficulty sitting down on or standing up from the toilet.

INSTALLATION

1. Remove toilet seat and arms from carton and protective packaging.
2. Insert each end of arm into the raised toilet seat as shown at near right.
3. Push arm downward until you hear it click into place.
4. Pull arm upward slightly to secure proper installation. If loose, repeat step 3.
5. Repeat steps 2-4 for other arm.
6. Lift lid and seat of toilet. This raised toilet seat fits most current, standard toilets.
7. Place raised toilet seat on toilet bowl rim, hooking rear flanges under rear edge of bowl. Ensure paddle is back far enough to clear inner rim of bowl. If not, turn knob counterclockwise.
8. Ensure raised toilet seat rests flat on toilet bowl rim; straighten unit on bowl rim.
9. To tighten, rotate adjustment knob clockwise. Push unit back against rear of toilet rim while tightening knob. When you feel resistance, turn an additional 1/8 turn to snug. If top rear of unit hits the toilet seat or lid, you must remove the seat and lid for proper installation.

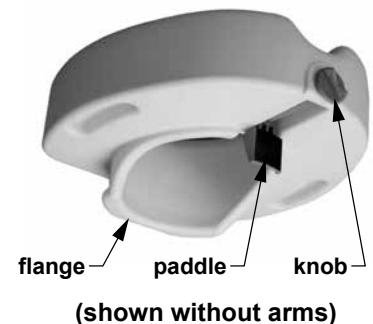

Info: The adjustment knob head must be flush with the front of the toilet seat for proper fit. On smaller bowls, if paddle is moved to rear of its travel, the knob may pop out of its original position; the knob will automatically re-engage when you re-tighten. **DO NOT** force the knob to turn. If necessary, jiggle the knob a little to start the threads.

10. The raised toilet seat should now be securely installed on the toilet bowl. To ensure that you have a proper fit, intentionally shake the raised toilet seat to ensure proper placement and fit. The raised toilet seat should not move; if it does, re-install.

- ⚠ **WARNING:** Ensure the raised toilet seat is installed as described. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.

REMOVAL

1. Rotate knob counter-clockwise until raised toilet seat is free.
2. Lift front of raised toilet seat, then remove from toilet bowl.

OPERATION

- ⚠ **WARNING:** Always ensure the raised toilet seat is securely installed before use.
- ⚠ **WARNING:** Users with limited physical capabilities should be supervised or assisted when using raised toilet seat.

MAINTENANCE / CLEANING

⚠ WARNING: Check the raised toilet seat weekly for signs of damage or cracking. If damage is found, DO NOT use the raised toilet seat. Contact your Lumex authorized distributor immediately.

Loosen, retighten, and check for secure fit of raised toilet seat weekly. To clean raised toilet seat, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the raised toilet seat is rinsed well before use.

If moisture accumulates inside raised toilet seat, do the following:

1. Remove raised toilet seat from toilet bowl.
2. Turn raised toilet seat upside down and tilt toward drain opening.
3. Rotate until drain opening is facing downward and moisture drains from raised toilet seat.
4. Allow raised toilet seat to air dry for several hours.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the raised toilet seat.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com



© 2005, GF Health Products, Inc. All Rights Reserved.
Lumex and Graham-Field are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.
Made in China