

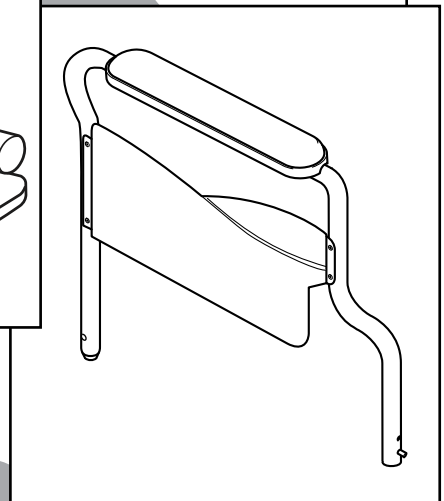


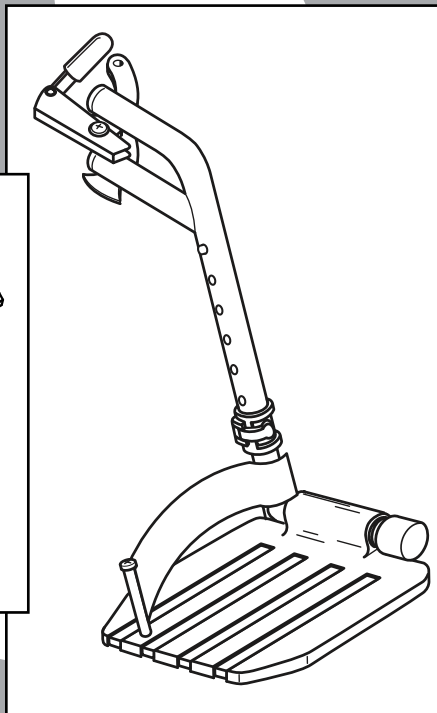

PARTS CATALOG

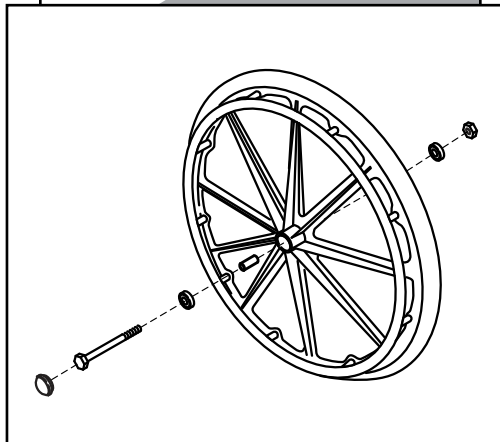

CONTENTS

SIDEFAME AND CROSSBRACE 3

ARM: FLIP-BACK FIXED-HEIGHT 4

UPHOLSTERY 4

REAR WHEEL 5

CASTER..... 5

FRONT RIGGING, SWINGAWAY AND DETACHABLE 6


ACCESSORIES 7

TO ORDER

Tel: 1-770-368-4700 • Fax: 1-770-368-2386

Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. [Terms and Conditions](#) for further information.


SIDEFRAME AND CROSSBRACE

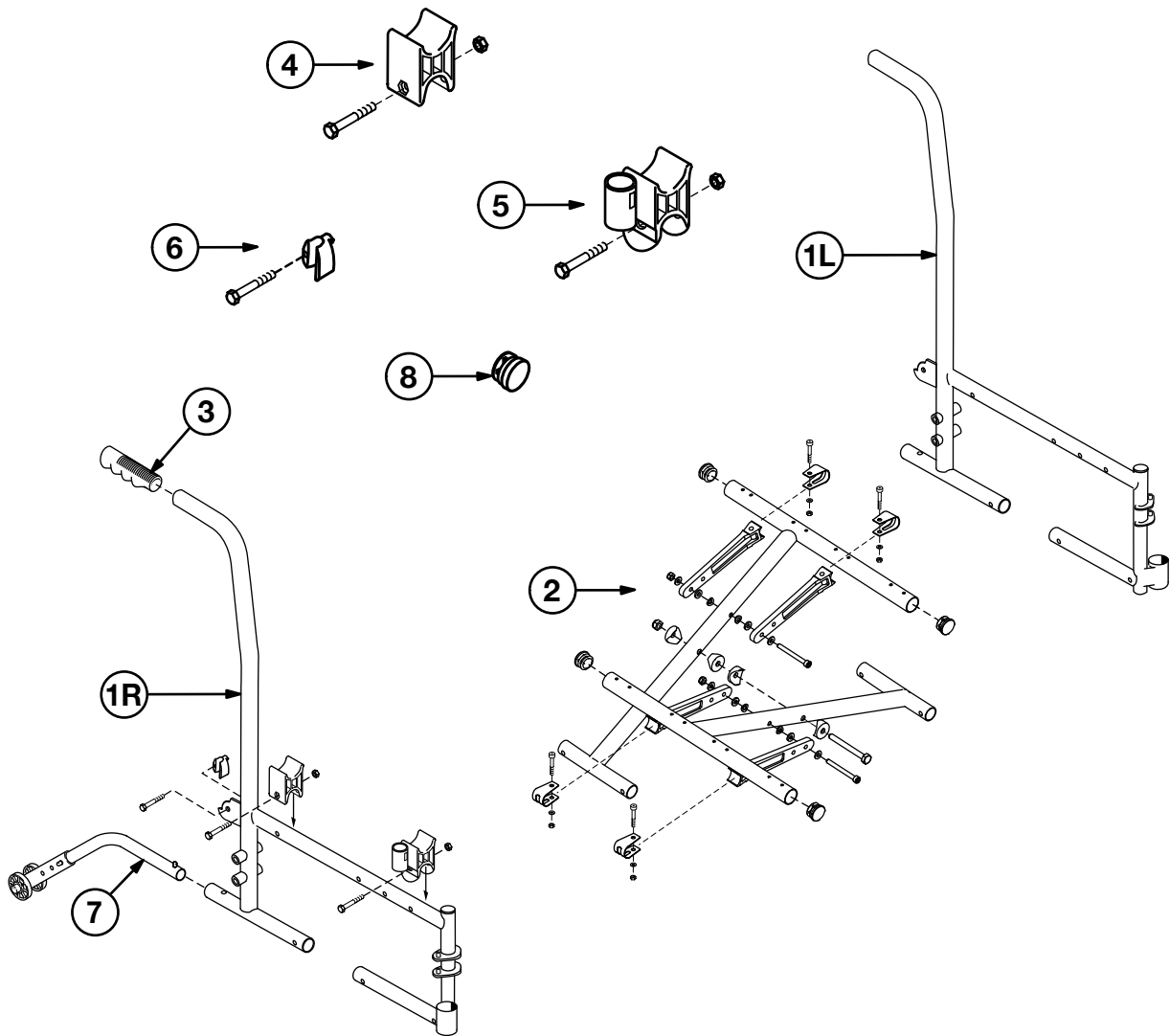

- 1 Sideframe and backpost, all widths**
 16" deep Right..... 90763201A
 16" deep Left..... 90763202A
- 2 Crossbrace, pair (Includes center bolt and nut, 4 crossbrace links, 4 plugs and mounting hardware)**
 16" W x 16" deep..... 907632C1A
 18" W x 16" deep..... 907632C2A
 20" W x 16" deep..... 907632C3A
- 3 Grip (10 each) 90763005A**


- 4 Rear seat rail cradle, pair (includes 2 each and mounting hardware) 90763206A**
- 5 Front seat rail and arm cradle, pair (includes 2 each and mounting hardware) 90763208A**
- 6 Arm bracket, pair (includes 2 each and mounting hardware) 90763210A**
- 7 Anti-Tippers, pair..... 90763230A**
- 8 Seat rail plug (4 each) 907633CSE**

ARM: FLIP-BACK FIXED-HEIGHT

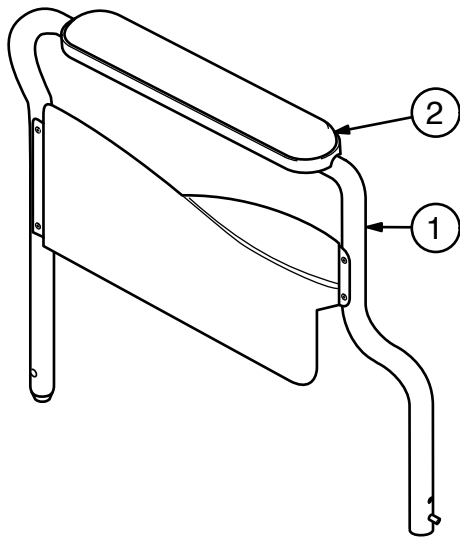


- | | | |
|---|--|-----------|
| 1 | Arm, flip-back fixed-height, desk length, pair (includes right and left arm with mounting hardware, side panel, and arm pad) | 90763231A |
| 2 | Arm pad only, desk length, pair (includes two black arm pads and mounting screws) | 90763022A |

UPHOLSTERY

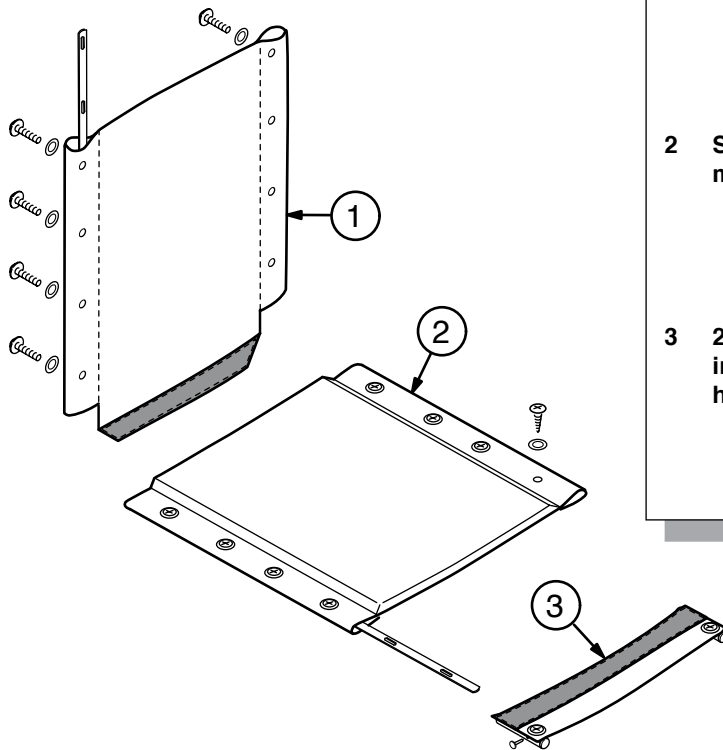


- | | | |
|---|--|-----------|
| 1 | Back upholstery, black nylon, and mounting hardware | |
| | 16" W..... | 907632A1A |
| | 18" W..... | 907632A2A |
| | 20" W..... | 907632A3A |
| 2 | Seat upholstery, black nylon, and mounting hardware | |
| | 16" W x 16" D..... | 907632B1A |
| | 18" W x 16" D..... | 907632B2A |
| | 20" W x 16" D..... | 907632B3A |
| 3 | 2" Seat extension kit, black nylon, includes tubes and mounting hardware | |
| | 16" W..... | 90763SE16 |
| | 18" W..... | 90763SE18 |
| | 20" W..... | 90763SE20 |

REAR WHEEL

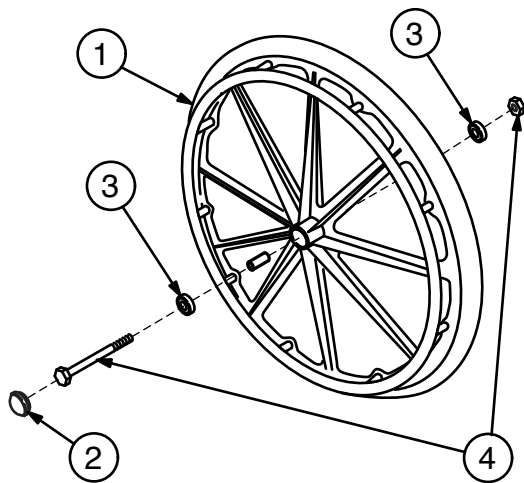


- | | |
|--|-----------|
| 1 Rear wheel with handrim, 24" x 1",
polyurethane tire (includes hub cap,
bearings, and spacer; does not include
axle or nut) | 90763150A |
| 2 Hub cap (10 each)..... | 90763051A |
| 3 Wheel bearing (10 each) | 90763052A |
| 4 Rear axle and nut (10 each)..... | 90763053A |

CASTER

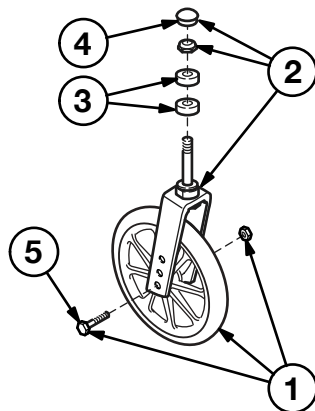


- | | |
|---|-----------|
| Complete caster (fork and wheel),
8" x 1" solid rubber tire..... | 90763040A |
| 1 Caster wheel, axle bearings,
axle and nut | 90763041A |
| 2 Caster fork, stem nut and dust cap | 90763042A |
| 3 Caster stem bearing (10 each) | 90763043A |
| 4 Caster dust cap (10 each)..... | 90763044A |
| 5 Caster axle and nut (10 each)..... | 90763045A |

FRONT RIGGING, SWINGAWAY AND DETACHABLE

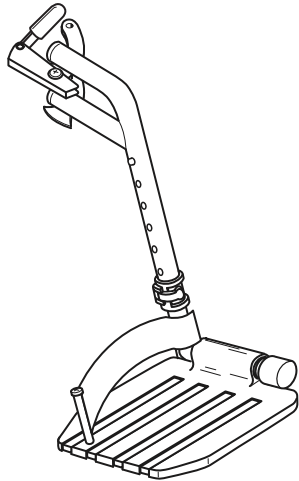

Footrest, pair, all widths
Includes right and left hanger with
release and black composite footplate
with extension and heel loop 90763030A


Black composite footplate, pair 90763033A

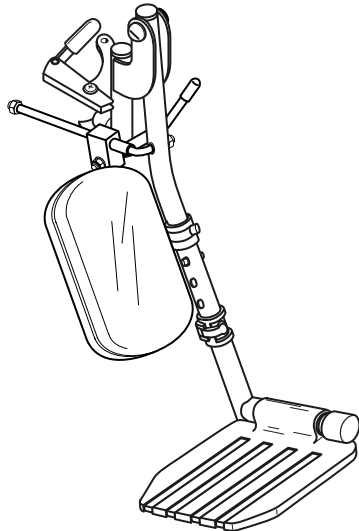

Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
composite footplate with extension, and
legrest panel (does not include
heel loop) 90763031A

Black composite footplate, pair90763033A

Elevating legrest panel kit
Includes 2 black padded panels
and hardware90763032A

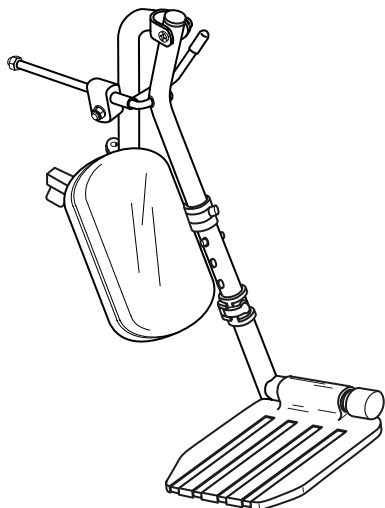

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop)90763435

Black composite footplate, pair90763033

Elevating legrest panel kit
Includes 2 black padded panels
and hardware90763032

Traveler L3 Plus accessories (not shown)

1	Heel loop, pair.....	90763064A
2	Wheel lock, pair, push to lock (includes rubber tips).....	90763070A
3	Wheel lock extension, pair	90763062A
4	Positioning belt (auto style)	92133040
5	Anti-Fold bar	92523850
6	O2-IV holder	B10320F
7	O2 cylinder holder.....	B10325
8	Tipping sleeve (10 each).....	90763006



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com



© 2016, GF Health Products, Inc. All Rights Reserved.

Everest & Jennings, Graham-Field, and Traveler are trademarks of
GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140

GF Health Products, Inc. is an ISO 13485 Certified Company.