

**Congratulations on your purchase of the Lumex or Gendron Bariatric Mattress. This guide covers the unpacking instructions.**

1. Check for any obvious damage to the packing or its contents before opening. If damage is evident, **DO NOT USE**. Contact carrier / distributor for further instruction.
2. Remove mattress from the box.
3. Position the mattress near the bed base.
4. Locate the shrink-wrap seam, and peel back to unroll mattress (see photos below). **DO NOT use knife or scissors to cut shrink-wrap.**
5. The mattress will start to unwrap. Be careful as sometimes it can start to expand rapidly.
6. Once unwrapped, the compressed mattress foam may take up to 72 hours to fully recover. Environmental factors such as cooler temperatures may cause expansion to take longer. Mattress will expand to + / -1" dimensions from the size mattress that you ordered.

