Universal Aluminum Crutches

Muletas Universales De Aluminio

Items 3610LF-8, 3611LF-8, 3612LF-8

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 300 lb. - EVENLY DISTRIBUTED.
3. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts.
5. Before use, ensure that crutch tips are not ripped, worn or missing, are properly seated and straight on each crutch.
6. Always use caution when using this product on wet or slippery surfaces. DO NOT use crutches on stairs or steep inclines.
7. Inspect crutches weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
8. Before use, ensure that push buttons protrude fully through adjustment holes.
9. Proper height adjustments must be made to both the leg extension and the crutch handle. A dealer or therapist should aid in the proper placement for maximum balance and support.
10. Wing nuts MUST be tight at all times.

GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

MAINTENANCE

- Wipe crutches weekly with a damp cloth.
- Grip and underarm pad may be cleaned with mild soap or detergent. DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check hardware and positioning weekly for tightness.
- Replace crutch tips as needed.
- To guarantee the best service from your crutches, always specify Lumex replacement parts, available from your GF Health Products distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ALUMINUM CRUTCHES SIZE

Selection Chart

Item Number	Size	User Height
3610LF-8	Adult	5' 2" – 5' 10"
3611LF-8	Tall	5' 10" – 6' 6"
3612LF-8	Youth	4' 6" – 5' 2"
3613C-8	Combination Pack of Adult, Tall and Youth crutches	

ADJUSTING HANDLE POSITION

1. Remove the wing nut and bolt securing the handle to the crutch.
2. Align the handle with the desired set of adjustment holes.
3. Secure with wing nut and bolt.
4. Repeat for other crutch.

HEIGHT ADJUSTMENT

1. Use double push buttons in center tube to adjust height in 1" increments.

Caution: Be sure that push buttons are fully protruding through the same respective adjustment hole of each leg extension. This ensures that the adjustable legs are securely locked into position and an even height adjustment is obtained.

LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

Frame: three (3) years


Tips, Wheels/Casters, and Handgrips: three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing cs@grahamfield.com. Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

For additional information related to warranty claims, please visit GF's terms and conditions located on www.grahamfield.com

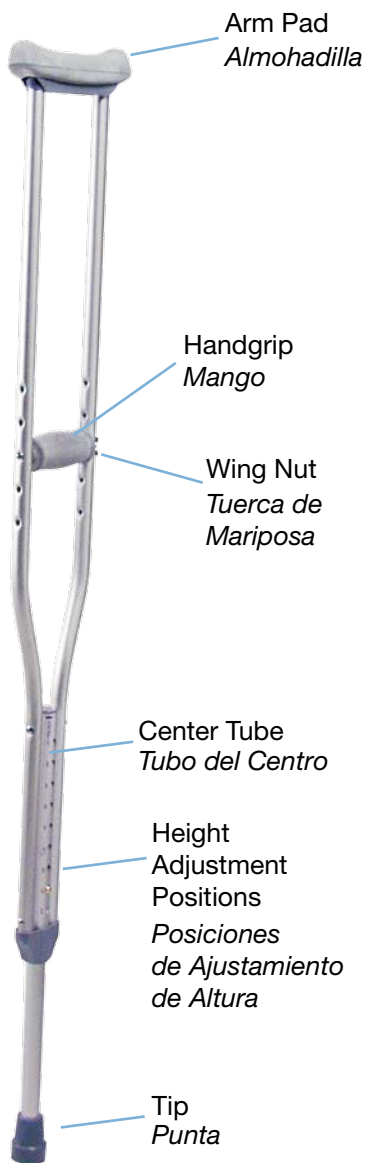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Features

- Anodized aluminum frame makes crutches a lightweight solution for user
- Reinforced center tube gives a stronger support to secure and maintain proper balance and stability
- Easy height and handle adjustment lets you adjust to best fit your needs
- Padded underarm pads and grips provide you with a comfortable and durable fit
- Maximum Weight Capacity 300 lb - **EVENLY DISTRIBUTED**

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2006, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com