

# **PVC Shower Bed**

Item 8005

### **Contents:**

Partially assembled shower bed Six (6) casters (4 locking, 2 non-locking) Push/pull handles Close cell mattress Ten (10) ½" stainless steel screws

# etings on the r bed. Secure each crew.

### **Assembly Instructions:**

## Step 1

**Installing Casters:** 

- 1. Install the four locking casters into the female fittings on the base of the head and foot sections of the shower bed. Secure each caster to the fitting with one  $\frac{1}{2}$ " stainless steel screw.
- 2. Install the two non-locking casters to the female fittings on the base in the center of the frame. Secure with stainless steel screws.

### Step 2

Install one push/pull handle to each bed end. Secure with ½" stainless steel screws.

Safety rails can be held in place by one safety locking pin.

Close cell mattress serves as a buffer/comfort for patient when using the shower bed.

### **Instructions for Use:**

The main purpose for the shower bed is to bathe residents/patients that are confined to a bed. The easy rolling casters allow the shower bed to transfer bed ridden residents/patients to and from the shower room.

- Roll the shower bed next to the individual's bed
- Drop the side rail facing the bed down, keeping the opposite side rail up
- If it's possible and the resident/patient has no medical requirements, transfer the resident/patient with head elevation in down/flat position.
- Have a minimum of 2 persons (nurses, aids, etc.) transfer the resident/patient onto the shower bed.
- Once the resident/patient is safely transferred and in a comfortable position, the head elevation can be raised if needed. Both side rails must be up and locked prior to the shower bed being moved.
- When using the shower bed, direct the drain hose to a drain system. All excess water must drain properly.

**Graham-Field Health Products** 

2935 Northeast Parkway, Atlanta, Georgia 30360 telephone: 800-347-5678, 770-447-1609 fax: 800-726-0601, 678-291-3232 http://www.grahamfield.com